


Wheel Locks - Hill Holder Assembly (A1598)

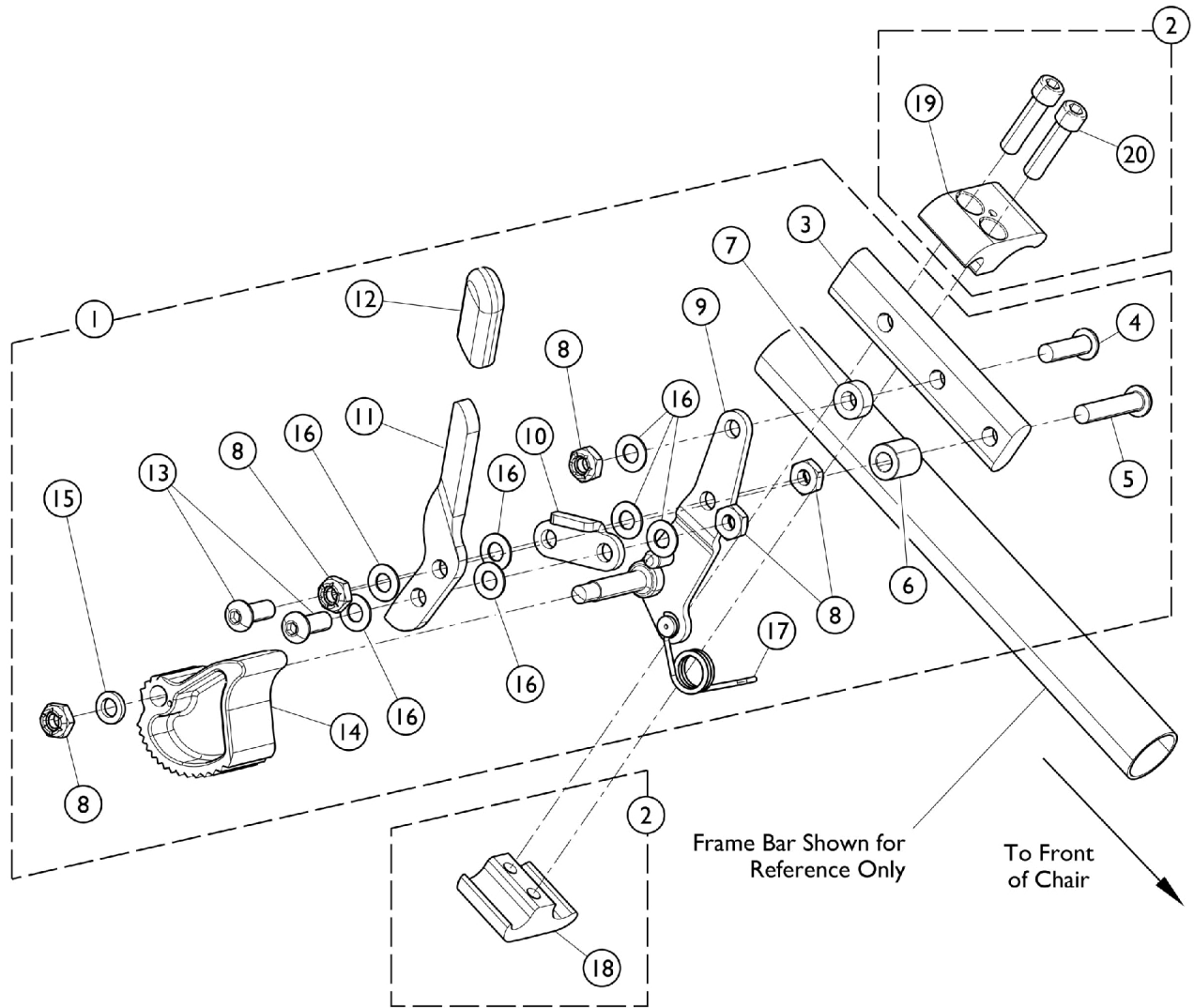

Wheel Locks - Hill Holder Assembly (A1598)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1039323-NLA	NLA - Hill Holder Assembly w/ Hardware - Right	1	1
3			NLA - Strap, Wheel Lock	1	1
4			NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1	1
5			NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
6			NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
7			NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
8			Locknut (1/4-28)	1	5
9			NLA - Bracket Assembly, Black - Right	1	1
10			NLA - Link, Stop, Black	1	1
11			NLA - Handle, Wheel Lock, Black - Right	1	1
12			Tip, Plastic, Black	1	1
13			Screw, Socket Button Head (1/4-28 x 5/8")	1	2
14			Shoe, Wheel Lock	1	1
15			Washer, Nylon (1/4 x 7/16 x 5/64")	1	1
16			Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
17			NLA - Spring, Torsion - Right	1	1
1			B	1039324-NLA	NLA - Hill Holder Assembly w/ Hardware - Left
3	NLA - Strap, Wheel Lock	1			1
4	NLA - Screw, Button Head Cap (1/4-28 x 7/8")	1			1
5	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1			1
6	NLA - Spacer (1/4 x 1/2 x 1/2")	1			1
7	NLA - Spacer (1/4 x 1/2 x 5/32")	1			1
8	Locknut (1/4-28)	1			5
9	NLA - Bracket Assembly, Black - Left	1			1
10	NLA - Link, Stop, Black	1			1
11	NLA - Handle, Wheel Lock, Black - Left	1			1
12	Tip, Plastic, Black	1			1
13	Screw, Socket Button Head (1/4-28 x 5/8")	1			2
14	Shoe, Wheel Lock	1			1
15	Washer, Nylon (1/4 x 7/16 x 5/64")	1			1
16	Washer, Nylon (1/4 x 1/2 x 1/32")	1			7
17	NLA - Spring, Torsion - Left	1			1
2		1039343			Kit, Clamp Wheel Lock Hardware (Pair)
18			Clamp, Wheel Lock, Threaded (7/8")	1	2
19			NLA - Clamp, Wheel Lock, Non-Threaded (7/8")	1	2
20			Screw, Allen Head (1/4-20 x 1")	1	4

NOTE: A - Includes items 3 - 17.
 B - Includes items 18 - 20.