


PHAL4A (Hemi) Articulating Legrest Supports (Before 1/6/03) Chrome

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1068649) when ordering replacement Calf Pads.

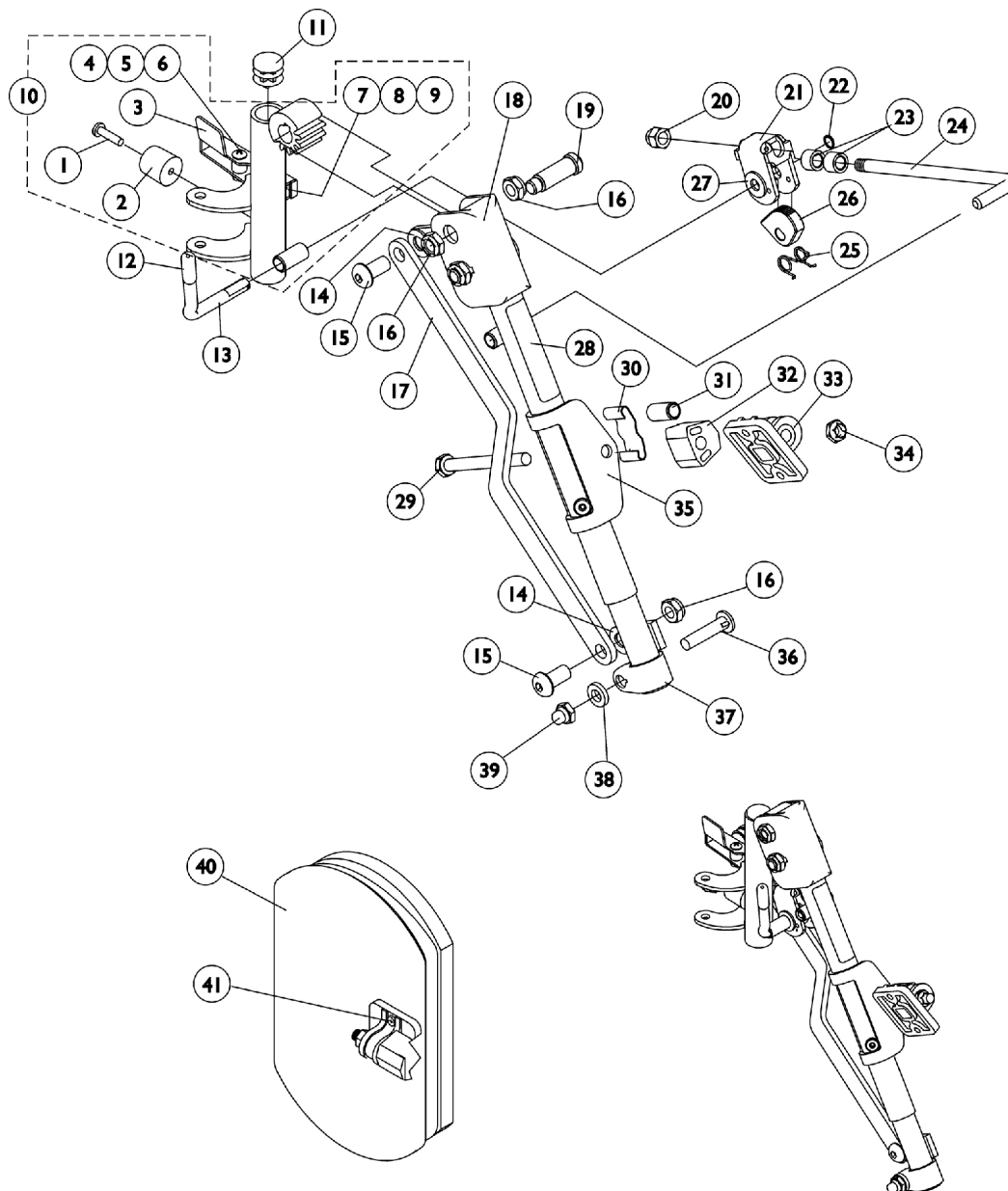

Assembled View
Right Side Shown

PHAL4A (Hemi) Articulating Legrest Supports

(Before 1/6/03) Chrome

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1068649) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
2		1022874	Bumper	1	1
3	B,D	1087698-NLA	NLA - Handle, Release, Chrome - Right	1	1
3	B,D	1087699-NLA	NLA - Handle, Release, Chrome - Left	1	1
4		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
5		60279X003	Locknut (#10-32)	1	1
6		40163X008	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
7		1008935	Spring, Release - Right	1	1
7		1008936	Spring, Release - Left	1	1
8		67005X000	Rivnut (#10-32)	1	1
9		40163X002	Screw, Phillips Round Head (#10-32 x 1/2")	1	1
10	E	1114918-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Right	1	1
10	E,F	1114919-NLA	NLA - Legrest Support Tube Assembly, Hemi, Chrome - Left	1	1
11		1027097	Plug Button, Black (3/4")	1	1
12		1040060	Tip, Plastic, Black	1	1
13		1037126	Handle, Release	1	1
14		1049527	Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
15		1022866-NLA	NLA - Screw, Socket Button Head (3/8-16 x 3/4")	1	2
16		40363X000	Hex Nut (3/8"-16)	1	3
17		1022846-NLA	NLA - Bar, Link, Black	1	1
18		1022877-NLA	NLA - Tube, Support and Drive Link, Chrome - Right	1	1
18		1022878-NLA	NLA - Tube, Support and Drive Link, Chrome - Left	1	1
19		1022854-NLA	NLA - Screw, Shoulder Slotted Head (1/2 x 1-1/8", 3/8-16)	1	1
20		10261X000	Nut, Acorn (5/16-24)	1	1
21		04528X002	Bracket, Release, Black	1	1
22		1040831	Ring, Snap	1	1
23		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	2
24		42326X004	Bar, Ratchet - Right/Left	1	1
25		1037485	Spring, Release	1	1
26		1037125	Cam, Legrest	1	1
27		1013519	Washer, Nylon (5/16 x 11/16 x 1/16")	1	1
28		1024911-NLA	NLA - Decal, Smartleg	1	1
29		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
30		48823X008	Spring, Leaf	1	1
31		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
32		1025133	Spacer, Legrest Pad, Black	1	1
33		04526X019	Plate, Legrest Pad	1	1
34		60264X001	Locknut (5/16-24)	1	1
35		1025104	Bracket, Legrest, Chrome	1	1
36		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
37		1022848	Tube, Slide, Articulating, Chrome - Right	1	1
37		1022849	Tube, Slide, Articulating, Chrome - Left	1	1

PHAL4A (Hemi) Articulating Legrest Supports

(Before 1/6/03) Chrome

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1068649) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
38		1027377	Washer (#20)	1	1
39		1107321	Nut, Low Cap (5/16"-24)	1	1
40	C,H	8881091764	Pad, Legrest, Vinyl - Right/Left	1	1
41	G	1068649	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
	A	1114995-NLA	NLA - Legrest Support Assembly, Articulating, Hemi, Chrome - Right	1	1
	F,A	1114996-NLA	NLA - Legrest Support Assembly, Articulating, Hemi, Chrome - Left	1	1
NOTE: A - Not Shown B - Right release handle is used on the left side and the left release handle is used on the right side. C - Specify upholstery color U67 black when ordering. D - Includes items 4-6 E - Includes items 1-9 F - No longer available as a service part. There is no replacement. G - Includes Screws and Instruction Sheet. H - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1068649) when ordering replacement Calf Pads.					