
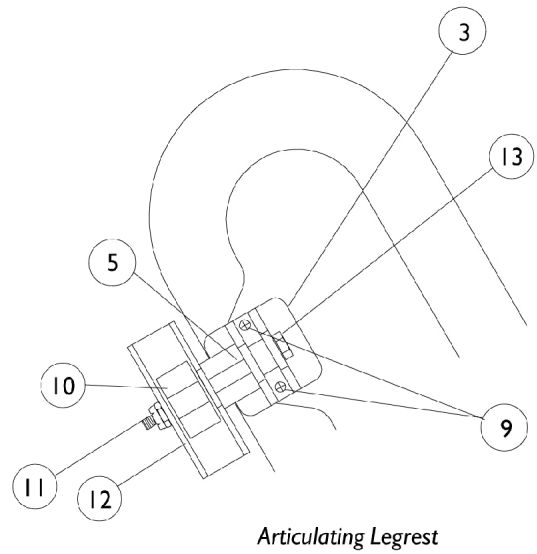


# Calf Pads and Attaching Hardware

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

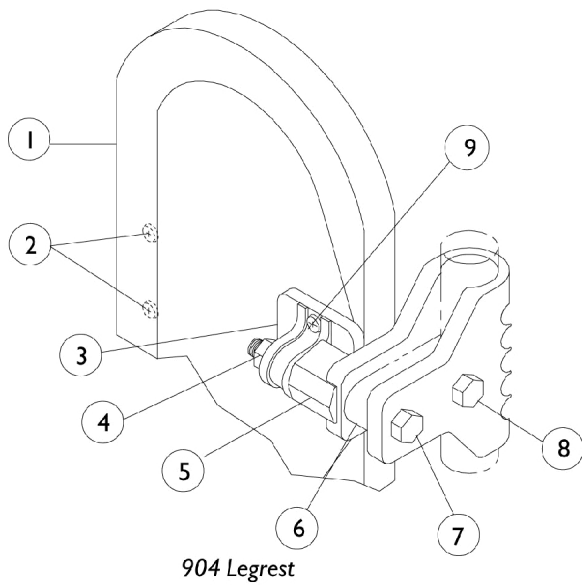

# Calf Pads and Attaching Hardware

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,G	88804529X003	Pad, Legrest, Vinyl - Right	1	1
1	A,B,G	88804529X004	Pad, Legrest, Vinyl - Left	1	1
1	A,C,D, G	1091767-NLA	NLA - Pad with Screws, Legrest (Pair)	2	1
1	A,C,E, G	8881091764	Pad, Legrest, Vinyl - Right/Left	1	1
1		01415X000-NLA	NLA - Cover, Legrest Pad	1	1
2		40163X006-NLA	NLA - Screw, Phillips Pan Head (#10-14 x 1/16")	1	1
3		04526X019	Plate, Legrest Pad	1	1
4		1025730	Locknut (5/16-24)	1	1
5		1025133	Spacer, Legrest Pad, Black	1	1
6		1032267	Spacer, Legrest Pad Mounting, Black	1	1
7		1037110	Screw, Hex Head (5/16-24 x 3-1/2")	1	1
8		1028514	Screw, Hex Head (5/16-24 x 1-1/2")	1	1
9	F	1066591	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
10		48823X008	Spring, Leaf	1	1
11		60264X001	Locknut (5/16-24)	1	1
12		1025104	Bracket, Legrest, Chrome	1	1
13		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1

NOTE: A - Specify upholstery color when ordering. See page titled "Frame Finishes, Touch-Up Paints and Upholstery Colors."  
 B - 14-15" Wide  
 C - 16-20" Wide  
 D - Before 9/5/00  
 E - After 9/4/00  
 F - Includes Screws and Instruction Sheet.  
 G - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.