
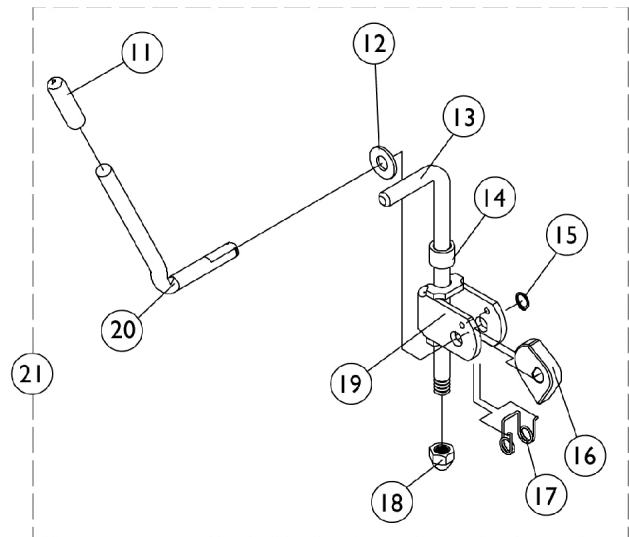


Legrest Calf Pads and Ratchet Assembly

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

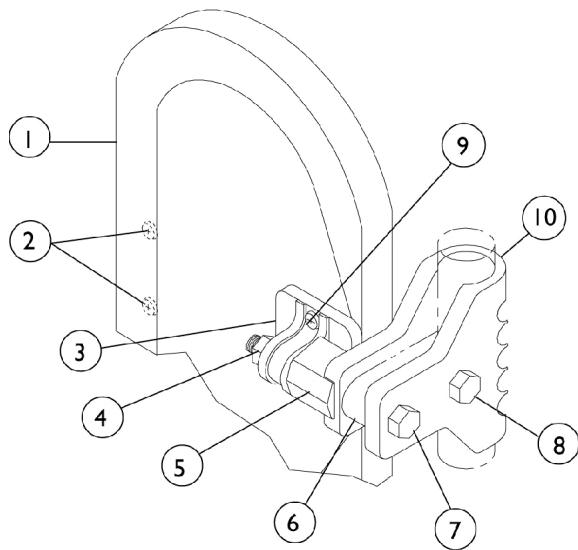

Legrest Calf Pads and Ratchet Assembly

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,G,J	8881091759	Pad w/ Cover and Screws, Legrest Vinyl (Pair)	2	1
1	A,C,G,J	8881091764	Pad, Legrest, Vinyl - Right/Left	1	1
1	A,H,J	88804529X003	Pad, Legrest, Vinyl - Right	1	1
1	A,H,J	88804529X004	Pad, Legrest, Vinyl - Left	1	1
1		01415X000-NLA	NLA - Cover, Legrest Pad	1	1
2		40163X006-NLA	NLA - Screw, Phillips Pan Head (#10-14 x 1/16")	1	1
3		04526X019	Plate, Legrest Pad	1	1
4		7930330-NLA	NLA - Locknut (5/16-24)	25	1
5		1025133	Spacer, Legrest Pad, Black	1	1
6		1032267	Spacer, Legrest Pad Mounting, Black	1	1
7		1037110	Screw, Hex Head (5/16-24 x 3-1/2")	1	1
8		1028514	Screw, Hex Head (5/16-24 x 1-1/2")	1	1
9	I	1066591	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
10		1032266	Bracket, Legrest Pad Mounting, Black	1	1
11		1040060	Tip, Plastic, Black	1	1
12		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
13	E	42326X004	Bar, Ratchet - Right/Left	1	1
13	F	42326X001	Bar, Ratchet	1	1
14		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	2
15		1040831	Ring, Snap	1	1
16		1037125	Cam, Legrest	1	1
17		1037485	Spring, Release	1	1
18		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
19		04528X002	Bracket, Release, Black	1	1
20		1037126	Handle, Release	1	1
21	E	1023812-NLA	NLA - Bar, Ratchet - Left	2	1
21	F	1096974	Kit, Elevating Legrest Ratchet Hardware (Pair)	2	1
	D,E,K	1042798-NLA	NLA - Instruction Sheet, Ratchet Kit	1	1
	D,F,L	1096943-NLA	NLA - Instruction Sheet, Ratchet Kit	1	1

NOTE: A - Specify U67 upholstery color when ordering.
 B - Before 9/00
 C - After 8/00
 D - Not Shown
 E - ATL4A
 F - AT5044
 G - 16-22" Wide
 H - 14-15" Wide
 I - Item includes Screws and Instruction Sheet.
 J - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.
 K - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.
 L - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.