



# Calf Pads and Ratchet Assemblies

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

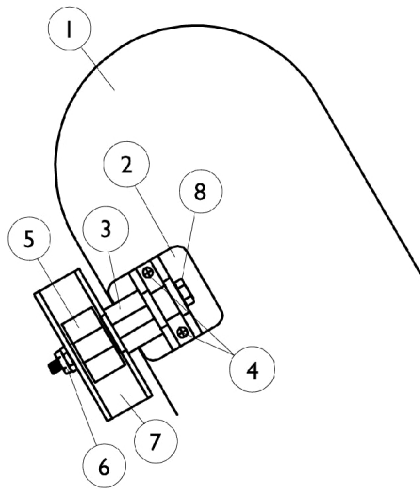

ATL4A  
Articulating Legrest

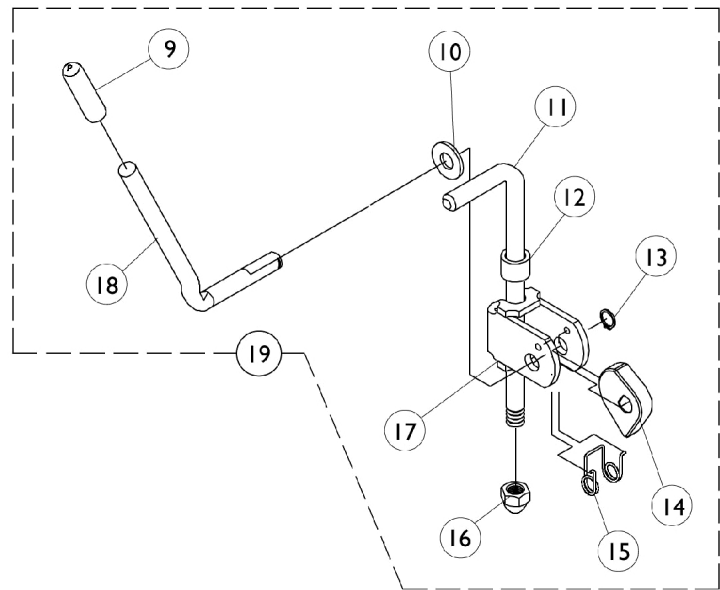

# Calf Pads and Ratchet Assemblies

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B,D, F	8881091759	Pad w/ Cover and Screws, Legrest Vinyl (Pair)	2	1
1	A,C,D, F	8881091764	Pad, Legrest, Vinyl - Right/Left	1	1
1		01415X000-NLA	NLA - Cover, Legrest Pad	1	1
2		04526X019	Plate, Legrest Pad	1	1
3		1025133	Spacer, Legrest Pad, Black	1	1
4	E	1066591	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
5		48823X008	Spring, Leaf	1	1
6		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	2
7		1025104	Bracket, Legrest, Chrome	1	1
8		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
9		1040060	Tip, Plastic, Black	1	1
10		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
11		42326X004	Bar, Ratchet - Right/Left	1	1
12		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
13		1040831	Ring, Snap	1	1
14		1037125	Cam, Legrest	1	1
15		1037485	Spring, Release	1	1
16		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
17		04528X002	Bracket, Release, Black	1	1
18		1037126	Handle, Release	1	1
19		1046329	Kit, Elevating Legrest Ratchet Hardware (Pair)	2	1

NOTE: A - Specify upholstery color when ordering.  
 B - Before 9/5/00 (pair)  
 C - After 9/4/00 (single)  
 D - 16" Wide  
 E - Includes Screws and Instruction Sheet.  
 F - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.