


# Back Flow Preventer and Drain Components

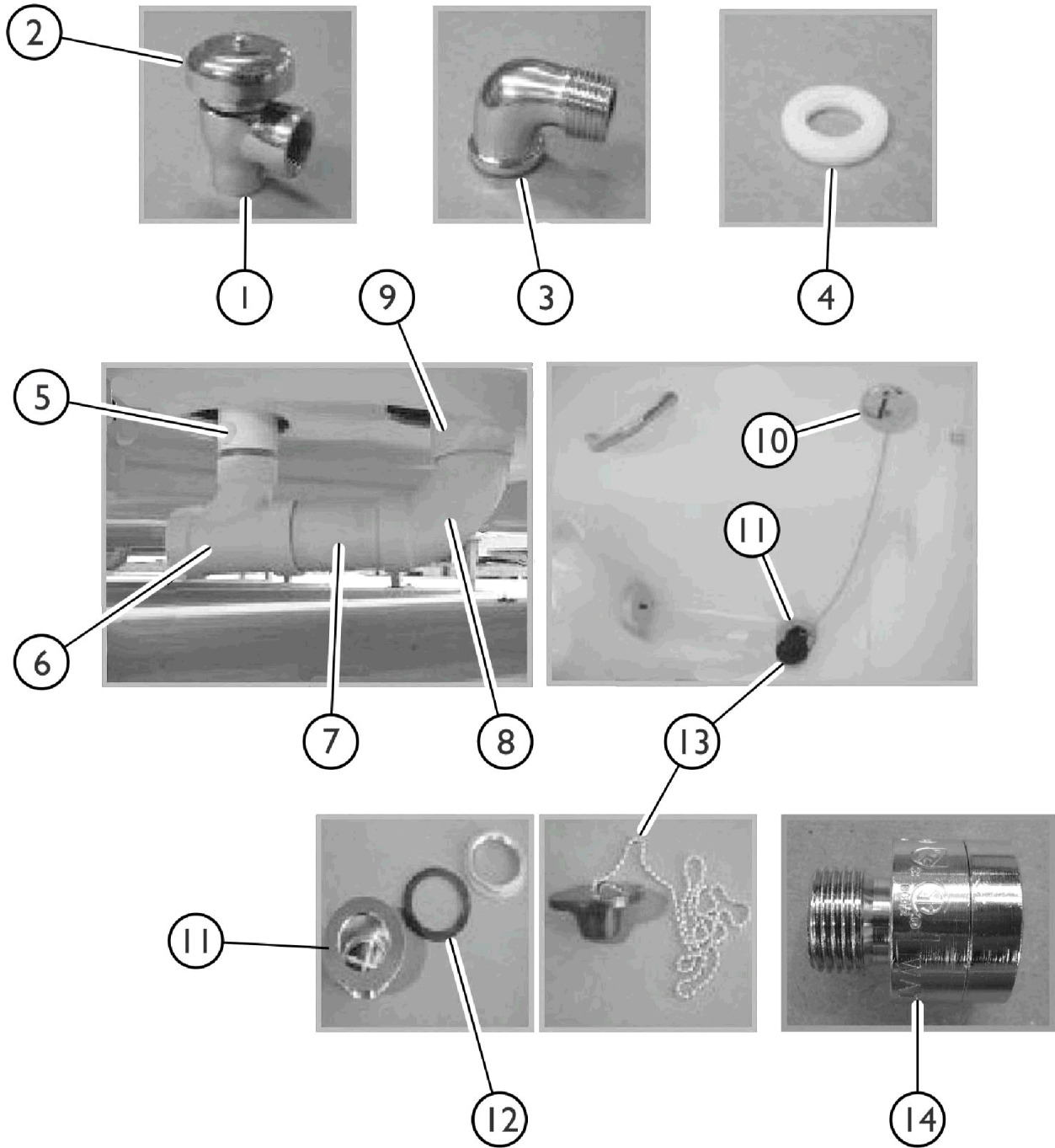

SICC\_005A

## Back Flow Preventer and Drain Components

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		PAR7380AG	Back Flow Preventer - Conbraco (1/2") (One Screw in Cap)	1	1
2	A	IHPAR7383C	Bonnet	1	1
3		IHPAR7384C	Kit, Repair	1	1
4		PAR1136	Elbow, 90 Degrees - Chrome	1	1
5		PAR6160D	Washer, Nylon, Backflow Preventer and Chrome Elbow	1	1
6		IHR2430295-NL A	NLA - Elbow, 45 Degree (1-1/2") (Connects to Overflow)	1	1
7		PAR5780BC	Reducing "T", Sanitary (2 x 2 x 1-1/2")	1	1
8		PAR8081-NLA	NLA - PVC Pipe (2") (Sold by the foot)	1	1
9		PAR8082WT	Elbow, 90 Degree, Wide Turn (2")	1	1
10		PAR8083	Threaded Adapter (2") (Connects to Drain)	1	1
11	B	IHR2430051	Cover, Overflow	1	1
12		60106483-NLA	NLA - Drain Assembly (2")	1	1
13		PAR7188	Gasket	1	1
14	C	PAR9817A	Plug, Drain with Chain	1	1
		1146438	Preventer, External, Back Flow	1	1

NOTE: A - Not shown in picture.  
 B - Part includes Item 13.  
 C - Used on Tubs built for LA County, Wisconsin, and Oregon.