



# Mixing Valve - RADA 222

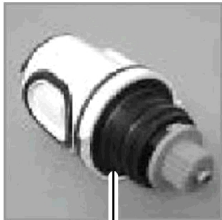


1




2

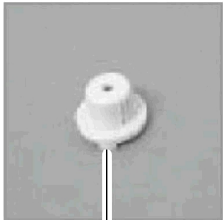

3

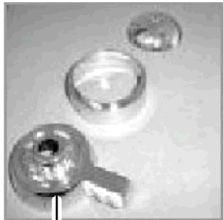


4

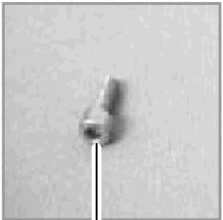

5



6

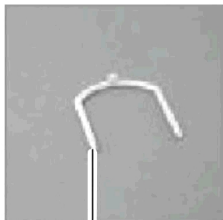

7



8



9



10

## Mixing Valve - RADA 222

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		PAR6670CG	NLA - Valve, Complete RADA Mixing	1	1
2		PAR6676CG	Cartridge, Replacement	1	1
3		PAR6674CG	Hub, Yellow	1	1
4		PAR6673CG-C	Handle Assembly, Chrome	1	1
4		PAR6673CG-W-NLA	NLA - Handle Assembly, White	1	1
5		PAR6672CG-NL A	NLA - Set Screw Handle	1	1
6		IHPAR6675	Nut, Head	1	1
7		PAR6675CG	Clip, Removal, Yellow	1	1
8		PAR6677CG	Valve, Inlet Check	1	1
9		PAR6669CG-NL A	NLA - Elbow, 90 Degree, Compression Fitting	1	1
10		PAR6679CG-NL A	NLA - Fitting, Straight Compression	1	1
	A	IHPAR6681-CG	Service Pack, Mixing Valve, RADA 222	1	1

NOTE: A - Not shown in photos.