


Seat Frame Assembly

Junior 12"-16" Wide

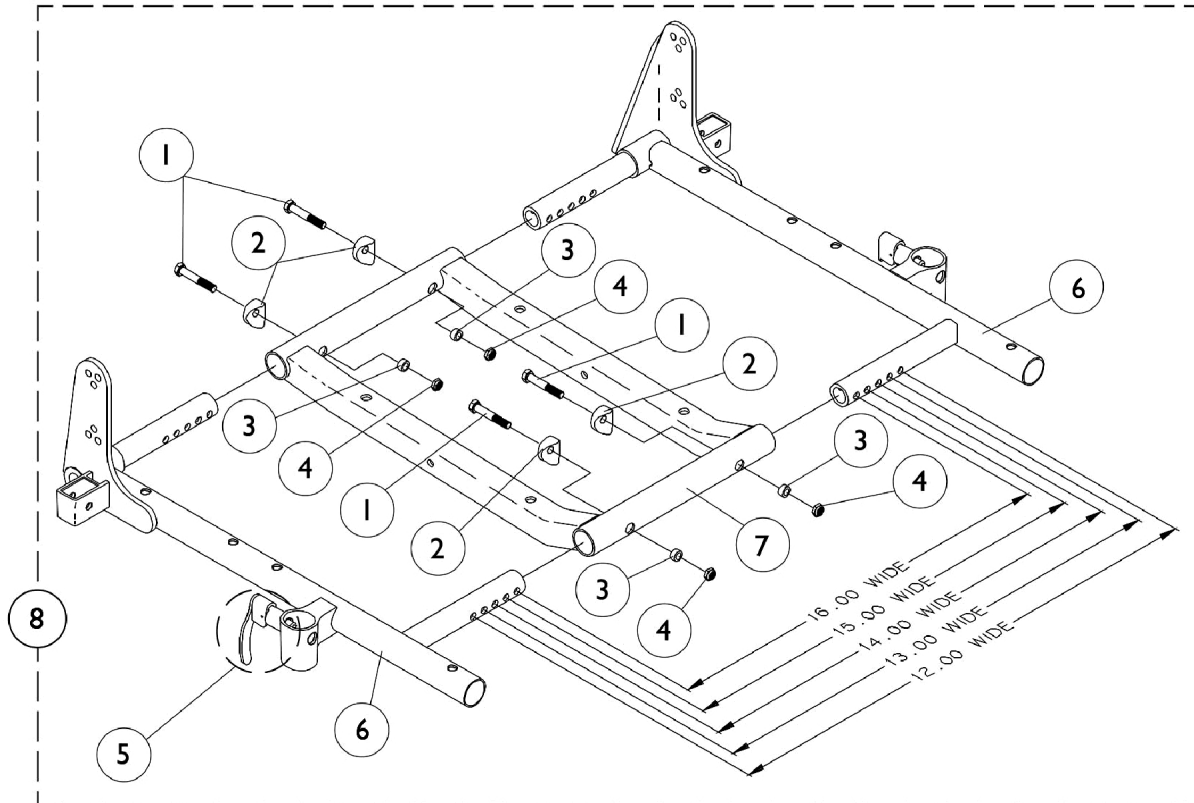

Seat Frame Assembly Junior 12"-16" Wide

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1037610	Screw, Hex Head (1/4-20 x 1-5/8")	1	4
2		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
3		1047449-NLA	NLA - Spacer (1/4 x 3/8 x 3/16")	1	4
4		1025736	Locknut (1/4-20)	1	4
5		47471X074	Spring, Arm Lock	1	1
5		1028594	Pin, Arm Lock, Black	1	1
5		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
6	A,B	1070473-NLA	NLA - Frame Seat, Adult/Junior, Black - Right	1	1
6	A,B	1070474-NLA	NLA - Frame Seat, Adult/Junior, Black - Left	1	1
7	A	1070476-NLA	NLA - Frame, Seat, Adult/ Junior, Center, Black	1	1
8	C	1070469-NLA	NLA - Frame Assembly Seat, Adult/Junior, Black (14-15" W x 16-17" D)	1	1

NOTE: A - Junior 12-16" Wide and 12-16" Deep
 B - Includes seat frame and item 5 arm lock assembly
 C - Includes items 1-7