
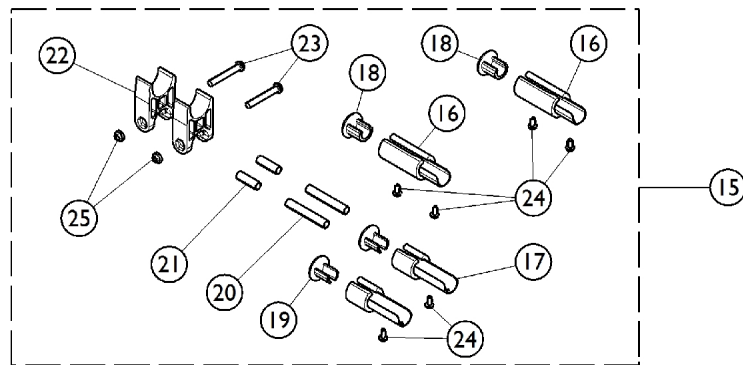
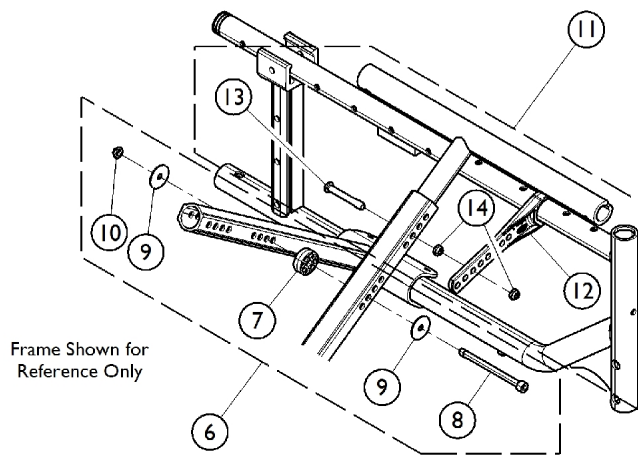
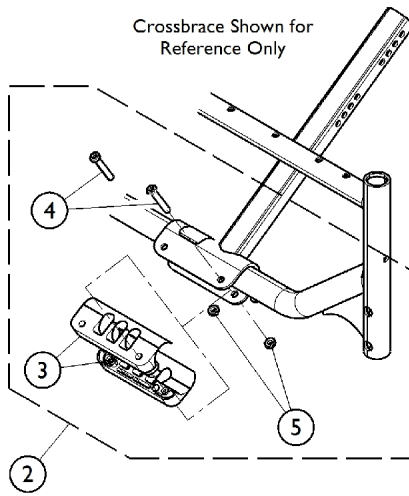


Crossbrace - Conventional Growable Seat Depth

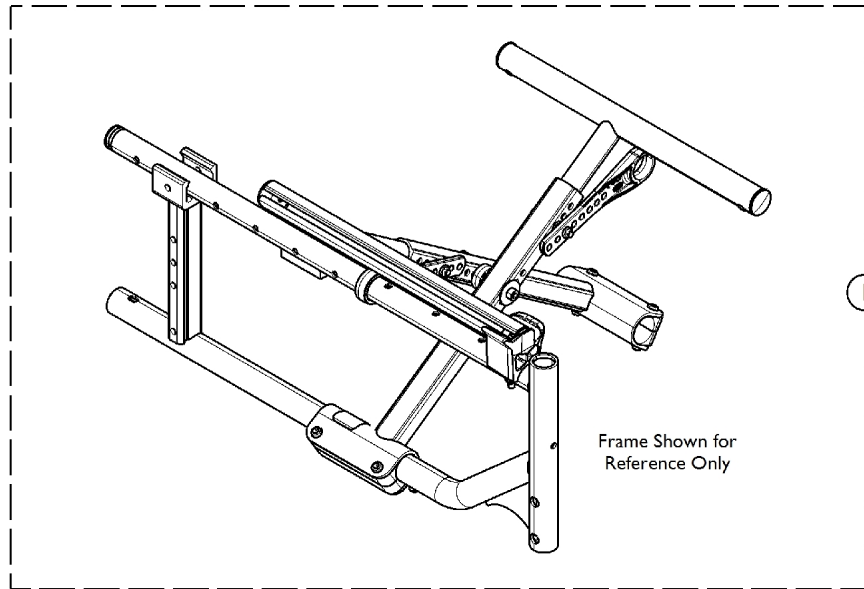

Crossbrace - Conventional Growable Seat Depth

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	See Chart	#1 Crossbrace Assembly - Growable Conventional Adjustable Seat Depth	1	1
2	B	1173765	Kit, Saddle Clamp w/ Hardware (Pair)	1	1
3			Clamp, Saddle	1	2
4	C	1173777	NLA - Screw, Socket Button Head (M6 x 40mm)	1	4
5			NLA - Locknut (M6 x 1)	1	4
6			Kit, Crossbrace w/ Hardware (Pair)	1	1
7			Washer, Plastic	1	2
8			Bolt (M6 x 80mm)	1	2
9	D	1173776	Washer (6.6 x 22 x 2)	1	4
10			NLA - Locknut (M6 x 1)	1	2
11			Kit, Pivot Link w/ Hardware (12 - 15" W)	1	1
12			Link, Pivot, Crossbrace (12-15" W)	1	2
13			Screw (M6 x 45mm)	1	2
14			NLA - Locknut (M6 x 1)	1	4
15			E	1172274	Kit, Seat Rail Extensions (16 - 18" D)
16			Extension, Seat Rail (2")	1	2
17			Extension, Seat Rail (1")	1	2
18			NLA - Plug Button	1	2
19			Cap, Extension, Seat Rail (1")	1	2
20			Rod, Upholstery (2")	1	2
21			Rod, Upholstery (1")	1	2
22			Guide, Seat	1	2
23			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
24			NLA - Screw, Phillips Pan Head (M3.5 x .6 x 9.5mm)	1	6
25			NLA - Locknut (M6 x 1)	1	2
			Instruction Sheet, Seat Rail Extensions	1	1

NOTE: A - Includes items 2 - 14.
 B - Includes items 3 - 5.
 C - Includes items 7 - 10.
 D - Includes items 12 - 14.
 E - Includes items 16 - 25.

#1 Growable Conventional Adjustable Seat Depth

Seat Depth	8 - 11" W	12 - 15" W
8 - 10" Deep	1511958	N/A
10 - 12" Deep	1511959	N/A
11 - 13" Deep	N/A	1512008
12 - 14" Deep	1513260	N/A
13 - 15" Deep	N/A	1512009