
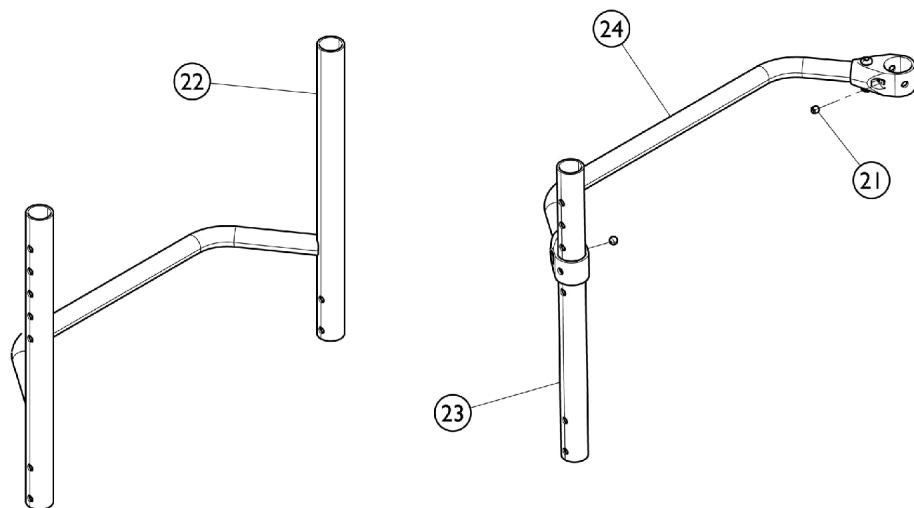


Backrests and Hardware w/ Fold Down Side Guards

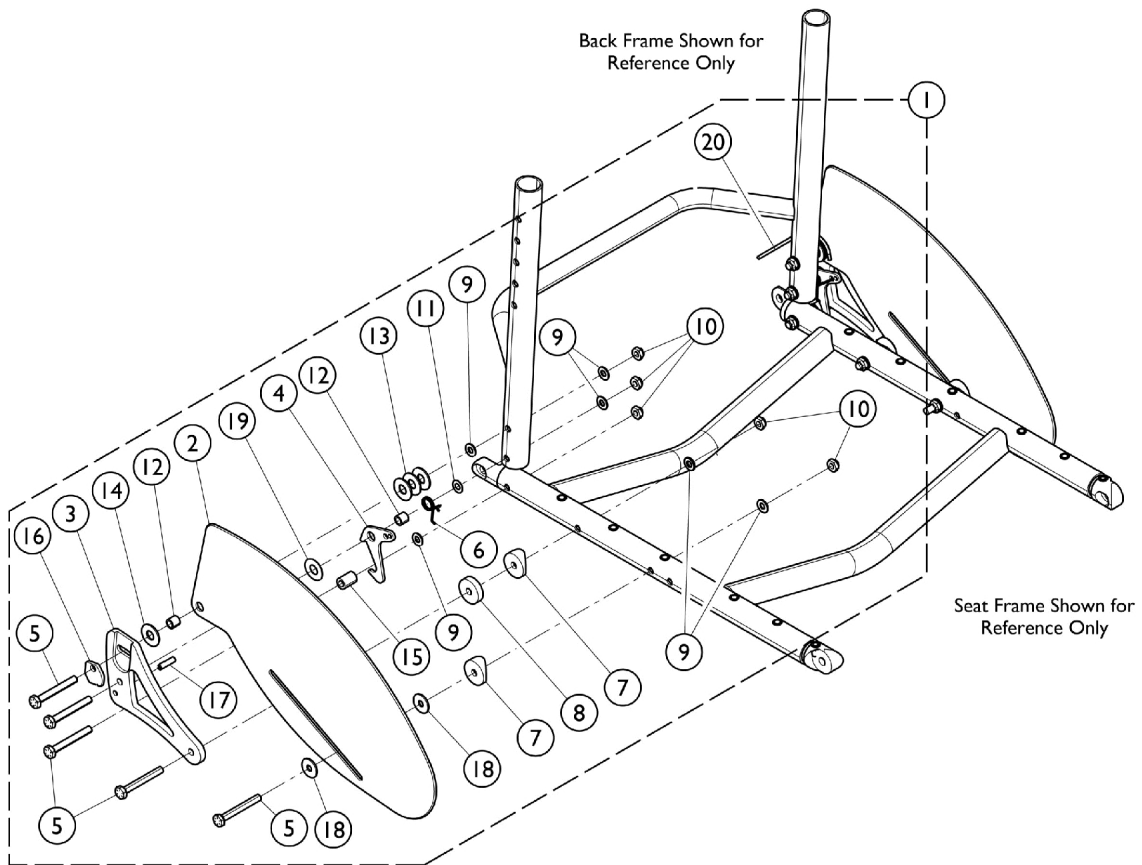

Backrests and Hardware w/ Fold Down Side Guards

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1114097-NLA	NLA - Kit, Fold Down Back Mounting Hardware with Side Guards	1	1
2		1114087-NLA	NLA - Guard, Side, Folding, Black	1	2
3		1114084-NLA	NLA - Bracket, Support Back, Black -Right	1	1
3		1114085-NLA	NLA - Bracket, Support Back, Black -Left	1	1
4		1116005-NLA	NLA - Bracket, Latch, Black	1	2
5		1114099	NLA - Screw, Hex Head (1/4-28 x 2")	1	8
6		1114077-NLA	NLA - Latch, Torsion Spring - Right	1	1
6		1114078-NLA	NLA - Latch, Torsion Spring - Left	1	1
7		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	4
8		1040776-NLA	NLA - Spacer, Nylon (1/4 x 1 x 1/4")	1	4
9		1026907	Washer (1/4 x 1/2 x 1/16")	1	14
10		1025765	Locknut (1/4-28)	1	10
11		1025728	Washer (1/4 x 1/2 x 1/32")	1	2
12		1026162	Spacer (1/4 x 3/8 x 11/32")	1	4
14		1058636-NLA	NLA - Washer, Nylon (25/64 x 7/8 x 1/32")	1	2
15		1039233	Spacer (1/4 x 7/16 x 5/8")	1	2
16		1039232-NLA	NLA - Cam, Adjustment	1	2
17		1071496-NLA	NLA - Pin, Roll (3/16 x 3/4")	1	2
18		1108477-NLA	NLA - Washer, Nylon (1/4 x 3/4 x 5/8")	1	4
19		1012371	Washer, Nylon (3/8 x 7/8 x 1/16")	1	2
20		1141633	Cord, Nylon (6-1/2 ft.)	1	1
21		1045495-NLA	NLA - Screw, Set Socket (1/4-20 x 1/4")	1	2
22		See Chart	#22 - One Piece Backrest Frame	1	1
23		See Chart	#23 - Upright Tube	1	2
24		See Chart	#24 - Backrest Crossmember	1	1

NOTE: A - Includes items 2 - 20.

#22 - One Piece Backrest Frame

Chair Width	8-11" High	10-14" High	12-16" High	14-18" High	16-20" High
12"	1071788-NLA	1071789-NLA	1071790-NLA	1071791-NLA	1071792-NLA
13"	1071793-NLA	1071794-NLA	1071795-NLA	1071796-NLA	1071797-NLA
14"	1024706-NLA	1026188-NLA	1042982-NLA	1026757-NLA	1026764-NLA
15"	1024707-NLA	1026189-NLA	1042986-NLA	1026758-NLA	1026765-NLA
16"	1024708-NLA	1026190-NLA	1042987-NLA	1026759-NLA	1026766-NLA
17"	1024709-NLA	1026191-NLA	1042988-NLA	1026760-NLA	1026767-NLA
18"	1024710-NLA	1026192-NLA	1042989-NLA	1026761-NLA	1026768-NLA
19"	1024711-NLA	1026193-NLA	1042990-NLA	1026762-NLA	1026769-NLA
20"	1024712-NLA	1026194-NLA	1042991	1026763-NLA	1026770-NLA
21"	1082654-NLA	1082952-NLA	1082953-NLA	1082954-NLA	1082955-NLA
22"	1082956-NLA	1082957-NLA	1082958-NLA	1082959-NLA	1082960-NLA

Backrests and Hardware
w/ Fold Down Side Guards
#23 - Upright Tube

Back Height	8 - 11" High	10 - 14" High	12 - 16" High	14 - 18" High	15 - 19" High
Part Number	1042514-NLA	1042513-NLA	1043358	1042515-NLA	1052226-NLA

Backrests and Hardware
w/ Fold Down Side Guards
#24 - Backrest Crossmember

Chair Width	Part Number	Chair Width	Part Number
12" Wide	1042508-NLA	18" W	1050080
13" Wide	1042509-NLA	19" W	1050077
14" Wide	1042510	20" W	1050074
15" Wide	1042511	21" W	1085292-NLA
16" Wide	1042512	22" W	1085293-NLA
17" Wide	1050083		