


Bedside Cabinet - Contemporary (Model IH23030)

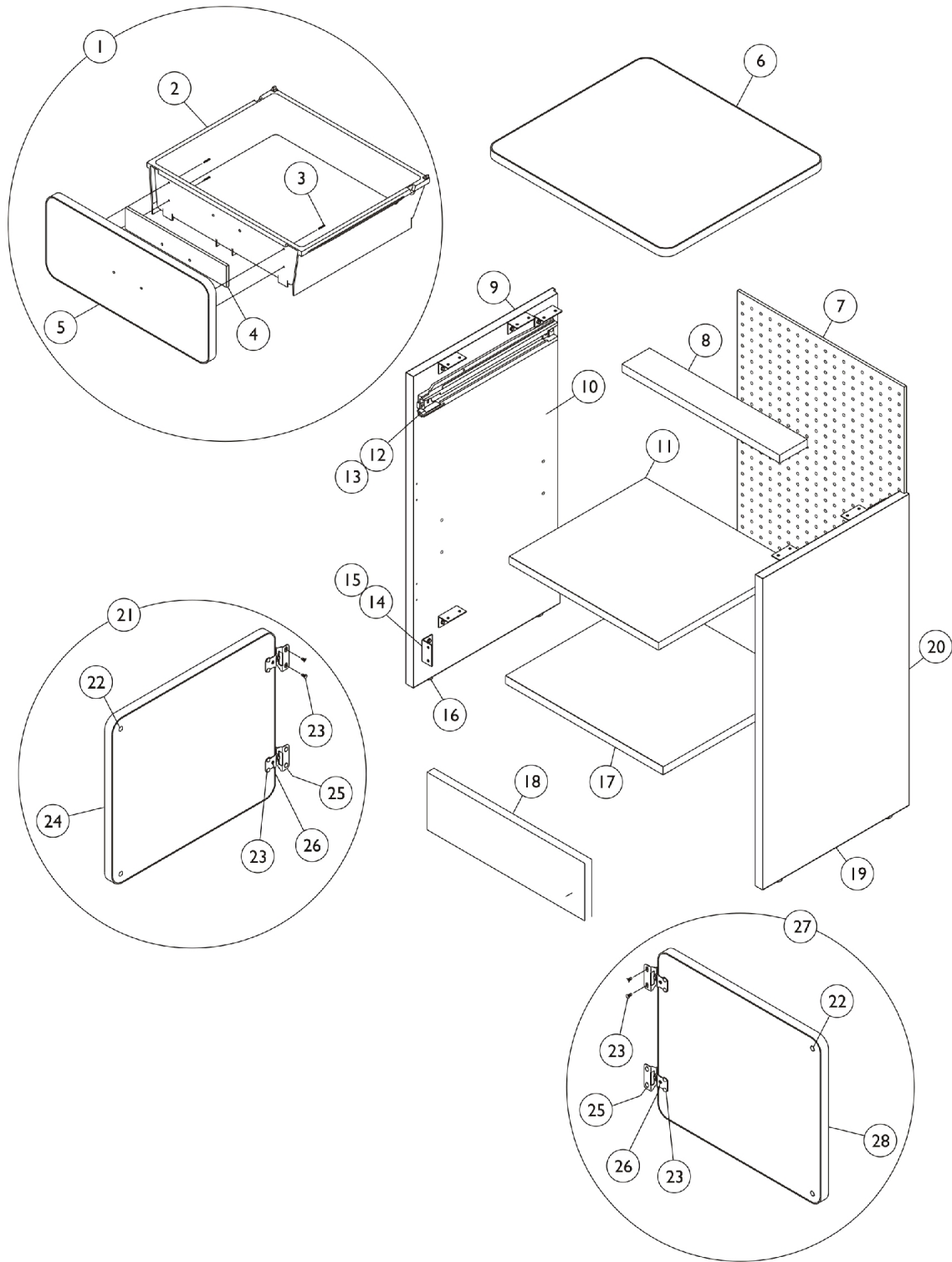

Bedside Cabinet - Contemporary (Model IH23030)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		A021402	Drawer Box Assembly	1	1
2		IHB007308	Drawer, Plastic (16-1/2")	1	1
3		IHB011227	Screw, Phillips Flat Head (#6 x 1")	1	4
4		IHP015366	Strip, Filler	1	1
5		A021403	Drawer Assembly - Front	1	1
6		A014613	Complete Assembly - Top	1	1
7		IHP014639	Core, Back	1	1
8		IHA024145	Complete Crosspiece Assembly	1	1
9		A024142	Side Panel Assembly - Left	1	1
10		A024141	Panel, Side - Left	1	1
11		A021406	Complete Shelf Assembly	1	1
12		IHB007311	Glide, Plastic Drawer - Left	1	1
13		IHB015929	Screw, Phillips Pan Head (#8 x 3/4")	1	4
14		IHB004330	Bracket, Angle	1	12
15		IHB007651	Screw, Hex Head (#10 x 5/8")	1	24
16		IHB003531	Glide, Plastic Tack (3/4 x 1/4")	1	4
17		A023006	Complete Bottom Assembly	1	1
18		A021398	Complete Toe Panel Assembly	1	1
19		A024144	Side Panel Assembly - Right	1	1
20		A024143	Side Panel - Right	1	1
	A	IHB007309	Glide, Drawer Plastic - Right	1	1
21		A024828	Door Assembly - Left	1	1
22		IHB023733	Pad, Felt	1	4
23		IHB004149	Screw, Phillips Pan Head (#8 x 5/8")	1	8
24		A024829	Door, Panel - Left	1	1
25		IHB003895	Hinge, Ajax	1	2
26		IHB003054	Screw, Phillips Flat Head (#6 x 5/8")	1	2
27		A024832	Door Assembly - Right	1	1
28		A024831	Door, Panel - Right	1	1
	A	IHB003571	Pin, Shelf Support	1	4

NOTE: A - Not Shown.