
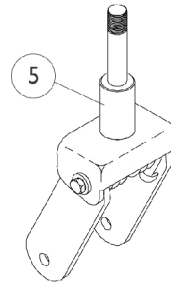


6" Caster Assembly

NOTE: 6" caster assemblies available On-Chair as of 11/19/01

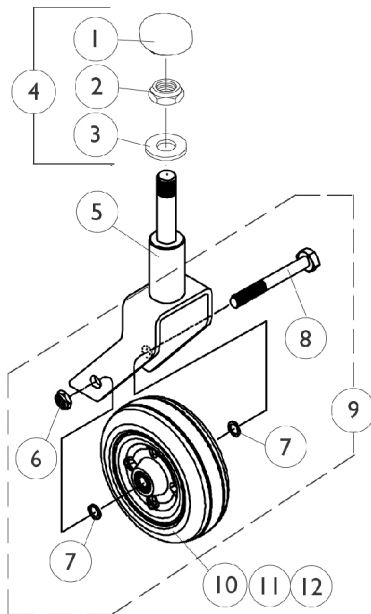

6" Caster Shock Fork

6" Caster Assembly

NOTE: 6" caster assemblies available On-Chair as of 11/19/01

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1101737	Cap, End, Tube, Black	1	1
2		1028550	Locknut (5/8-18)	1	1
3		1041444	Washer, Nylon (5/8 x 1-5/16 x 1/8")	1	1
4	A	1104451	Hardware, Caster Fork Mounting	1	1
5	C,D	8881041458	Fork and Stem Assembly (6" Caster)	1	1
5	C,D	8881056379	Fork, Shock, Color (6")	1	1
6		7930331-NLA	NLA - Locknut (7/16-20")	25	1
7		7930329	Washer, Steel (15/32 x 5/8 x 3/64")	25	2
8		1025726	Screw, Hex Head (7/16-20 x 3-1/4")	1	1
9	B	1042489	Hardware, Caster Axle	1	2
9	B	1058890-NLA	NLA - Hardware, Caster, Shock Fork (Pair)	2	1
10	E	1072014-NLA	NLA - Wheel Assembly, Semi-Pneumatic, Urethane (6" x 2")	1	1
10	F	1115179	Wheel Assembly, Semi-Pneumatic, Urethane, Deep Profile, Grey Tire/Black Hub (6" x 2")	1	1
10	F,G	1122066	Wheel Assembly, Semi-Pneumatic, Urethane, Deep Profile, Grey Tire/Black Hub (6" x 2") (Pair)	2	1
10	H	1175881	Wheel Assembly, Urethane, Deep Profile, Grey Tire/Silver Hub (6" x 2")	1	1
10	H	1180077	Wheel Assembly, Urethane, Deep Profile, Grey Tire/Silver Hub (6" x 2") (Pair)	2	1
11		1006049	Bearing (7/16")	1	2
12		1020592	Spacer, Bearing (7/16 x 5/8 x 1-3/4")	1	1

NOTE: A - Includes items 1-3
 B - Includes items 6-8 to accommodate 1 side.
 C - Specify frame color when ordering. See page titled "Frame Finishes and Touch-Up Paints".
 D - 300 Lb. weight limit and under
 E - Service part No Longer Available 8/15/03. Due to product improvement, order (pair) part number 1122066 dated after 8/14/03 so each side matches. The deep profile caster has 2 tread grooves w/ round corners instead of 4 tread grooves w/ square corners.
 F - Service part 8/13/03-2/29/12.
 G - Part number 1122066 is a pair number that can be used to replace part number 1072014 that was discontinued 8/15/03
 H - After 2/29/12. Caster changed from Grey Tire/Black Hub to Grey Tire/Silver Hub.