


Footrest Assembly

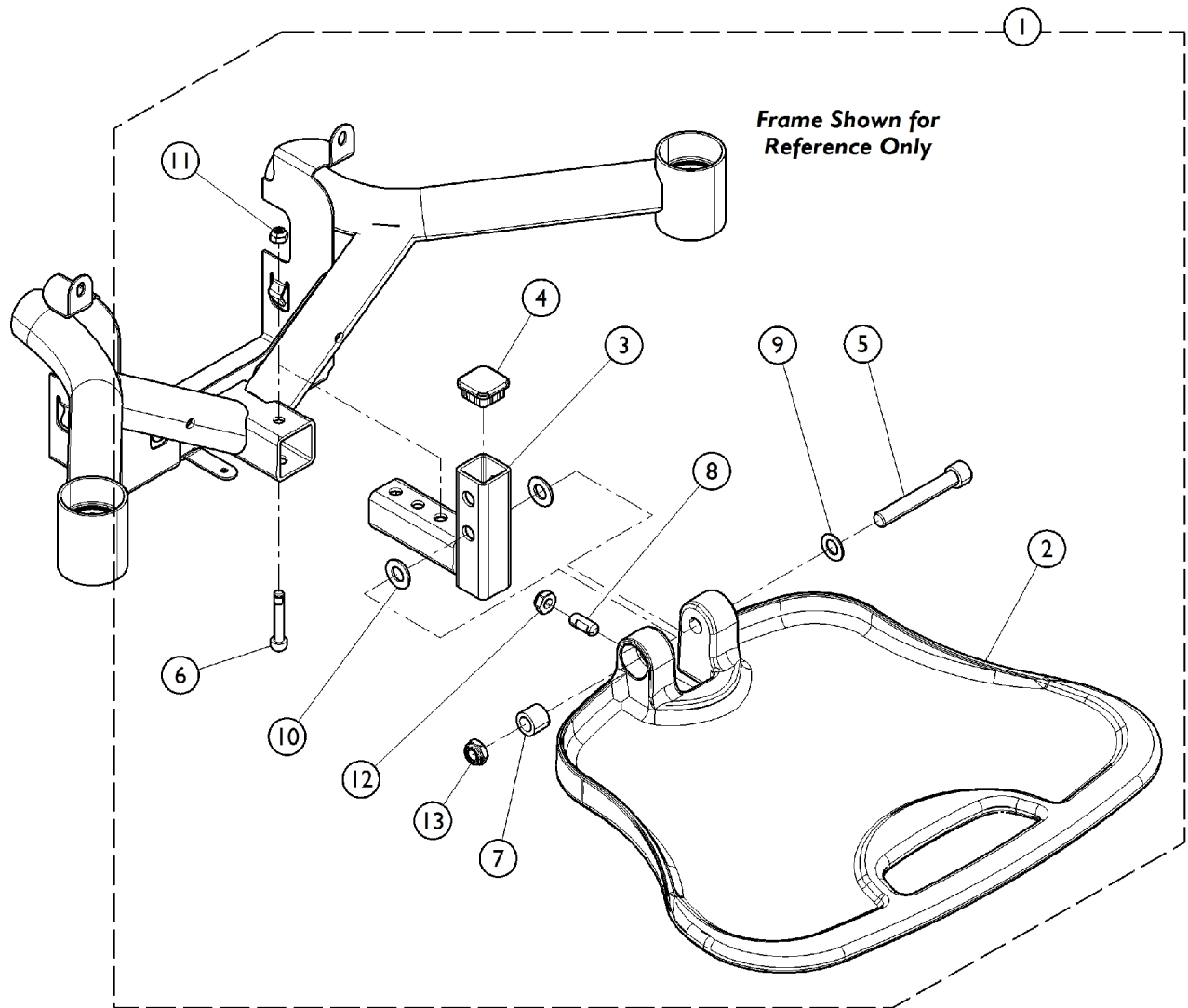

Footrest Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1179363-NLA	NLA - Kit, Footrest Assembly	1	1
2			Footplate	1	1
3			Footrest Support	1	1
4			Plug Button, Black (1" Square)	1	1
5			Screw, Socket Head (3/8-16 x 2-1/2")	1	1
6			NLA - Screw, Socket w/ Patch (M6 x 1 x 35mm)	1	1
7			Bushing (3/8 x 5/8 x 1/2")	1	1
8			Screw, Set Socket, Flat Point (5/16-18 x 3/4")	1	1
9			Washer, Flat (3/8 x 11/16 x 1/16")	1	1
10			Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
11			Nut, Lock (M6 x 1mm)	1	1
12			Locknut (5/16-18)	1	1
13			Locknut, Low Profile (3/8-16)	1	1
			Instruction Sheet, Footrest Assembly	1	1
NOTE: A - Includes items 2 - 14.					