



Wheels, Drive and Hub Mount

NOTE: The new chrome painted 14" x 3" spoke wheels are retrofitable to chairs made before 10/17/01. Wheel hub assembly part number (1103010) must be ordered as well.

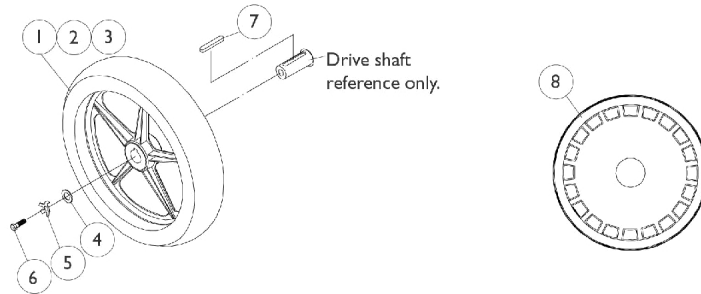


Drive Wheel Assembly (12-1/2" x 2-1/4")

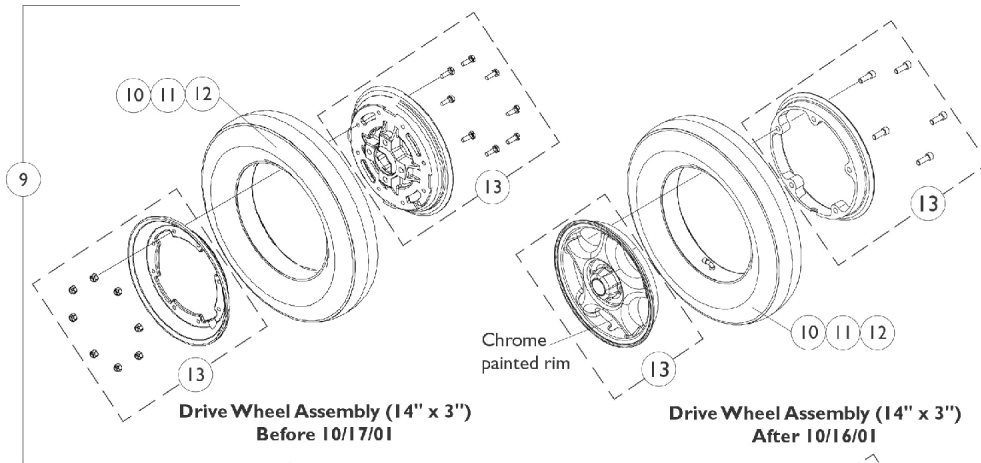

**Drive Wheel Assembly (14" x 3")
Before 10/17/01**

**Drive Wheel Assembly (14" x 3")
After 10/16/01**

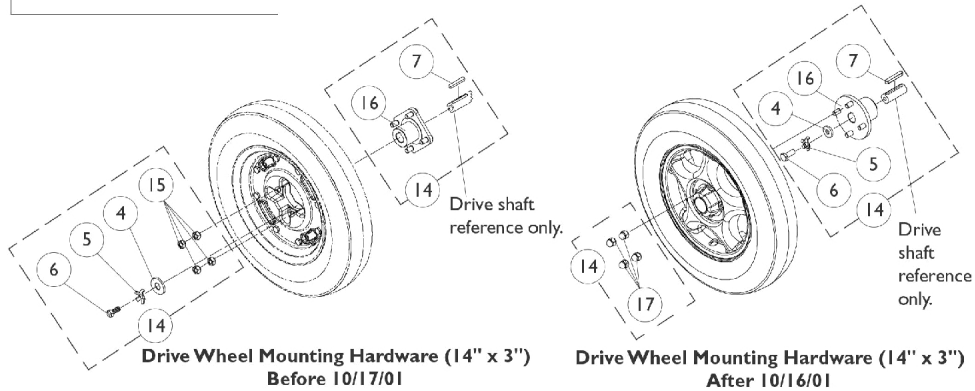

**Drive Wheel Mounting Hardware (14" x 3")
Before 10/17/01**

**Drive Wheel Mounting Hardware (14" x 3")
After 10/16/01**

Wheels, Drive and Hub Mount

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|-------|-------------|--|--------------|----------|
| | | | | Package | Assembly |
| 1 | | 1064177 | Drive Wheel Assembly, 5 Spoke w/ Foam Fill Insert (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet | 1 | 1 |
| 1 | | 1064178-NLA | NLA - Drive Wheel Assembly, 5 Spoke, Pneumatic (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet | 1 | 1 |
| 2 | | 1011524 | Tire, Pneumatic, Grey (12-1/2" x 2-1/4") | 1 | 1 |
| 3 | | 1011525 | Tube, Inner, Black (12-1/2 x 2-1/4") | 1 | 1 |
| 4 | | 1079247 | Washer, Flat (21/64 x 53/64 x 7/64") | 1 | 1 |
| 5 | | 1064179 | Kit, Locking Tab Washer w/ Instruction Sheet (Pair) | 2 | 1 |
| 6 | | 7930322-NLA | NLA - Package, Screw, Hex Head (M8 x 1.25 x 20mm) | 25 | 1 |
| 7 | | 1023264 | Key (6mm x 6mm x 40mm) | 1 | 1 |
| 8 | I | 1074974-NLA | NLA - Cover, Shroud (Black), Drive Wheel | 1 | 1 |
| 9 | A | 1090649-NLA | NLA - Pneumatic Wheel Drive Assembly (14" x 3") | 1 | 1 |
| 9 | B | 1104438 | Drive Wheel Assembly, Pneumatic, Grey Tire/Silver Rim (14 x 3") | 1 | 1 |
| 9 | A | 1090650-NLA | NLA - Drive Wheel Assembly w/ Flat Free Insert, Grey Tire/Black Rim (14 x 3") | 1 | 1 |
| 9 | B | 1104437 | Drive Wheel Assembly, Flat Free, Grey Tire/Silver Rim (14 x 3") | 1 | 1 |
| 10 | | 1101754 | Tire w/ Foam Fill Insert, Grey (14" x 3") | 1 | 1 |
| 11 | | 1039652 | Tire, Pneumatic, Grey (14" x 3") | 1 | 1 |
| 12 | | 1039651 | Tube, Inner, Black (14 x 3") | 1 | 1 |
| 13 | G,A | 1090651-NLA | NLA - Kit, 8" Rim w/ Hardware (14" x 3" Wheel) | 1 | 1 |
| 13 | H,B | 1104882-NLA | NLA - Kit, 8" Rim w/ Hardware (14 x 3" Wheel) | 1 | 1 |
| 14 | C,A | 1090648-NLA | NLA - Hardware, Drive Wheel Mounting w/ Instruction Sheet (14 x 3") | 1 | 1 |
| 14 | D,B | 1103011-NLA | NLA - Hardware, Drive Wheel Mounting w/ Instruction Sheet (14" x 3") | 1 | 1 |
| 15 | | 1024444 | Locknut (5/16-24) | 1 | 4 |
| 16 | A | 1086681-NLA | NLA - Hub, Wheel (14" x 3") | 1 | 1 |
| 16 | B | 1100576-NLA | NLA - Hub, Wheel (14" x 3") | 1 | 1 |
| 17 | | 1100580 | Nut, Beveled Hex (5/16-24) | 1 | 4 |
| | E,F,B | 1103010-NLA | NLA - Wheel Hub Assembly | 1 | 1 |

NOTE: A - Before 10/17/01
 B - After 10/16/01
 C - Includes items 4-7, 15 and 16
 D - Includes items 4-7, 16 and 17
 E - Includes items 16 and 17
 F - Not Shown
 G - Sold complete. Includes the Rims, (8) ea. Hex Head Screws (1/4-20 x 3/4") and (8) ea. Locknuts (1/4-20).
 H - Sold complete. Includes the Rims and (5) ea. Socket Head Screws w/ Patch (5/16-18 x 7/8").
 I - Shroud cover not available on drive wheels offered after 2/13/03