


Adult Elevating Leg Rest Calf Pads (ELRSS)

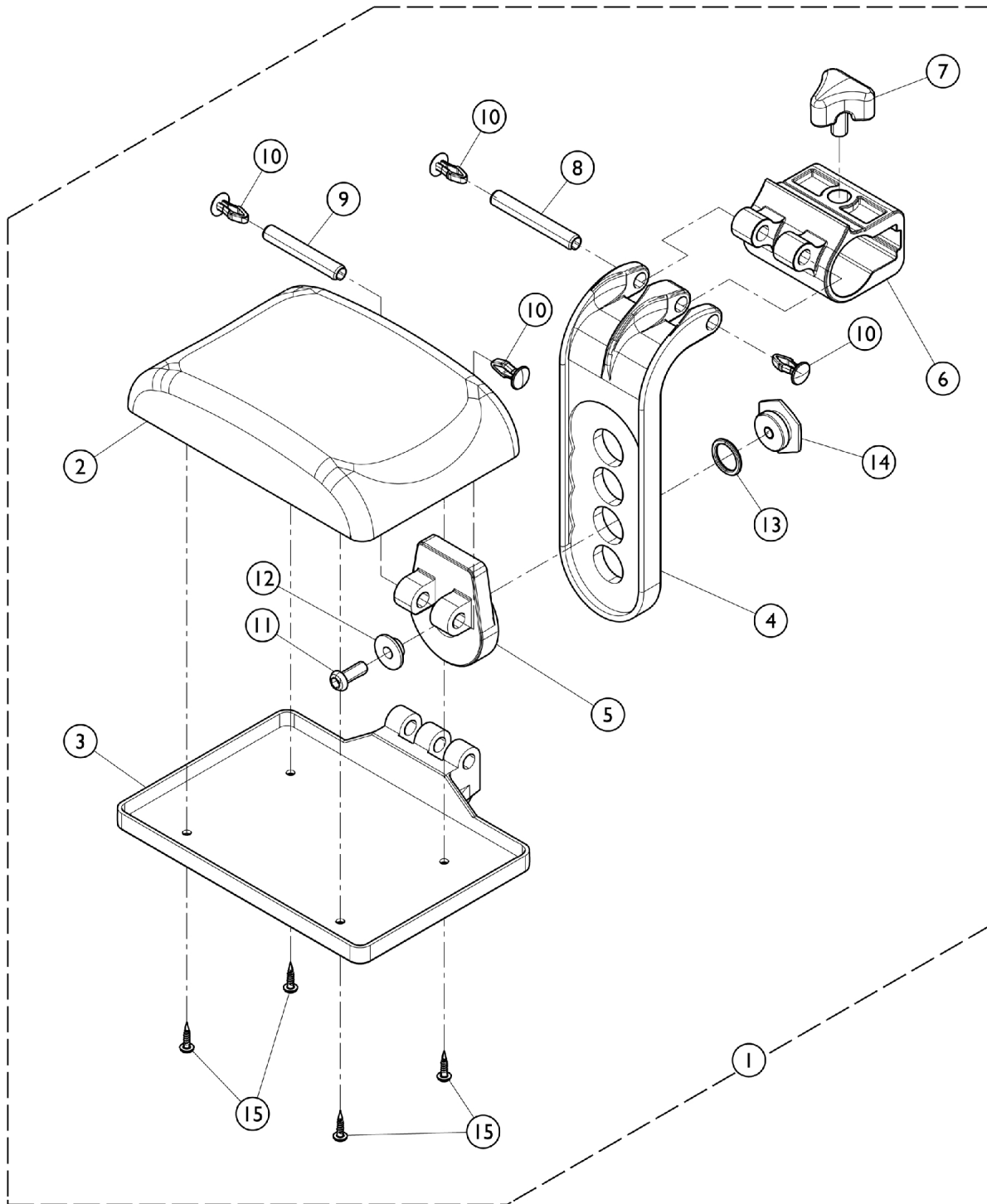

Adult Elevating Leg Rest Calf Pads (ELRSS)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1169052	Calf Pad Assembly w/ Hardware (15 - 17" W)	1	1
2			Pad, Calf (15 - 17" W)	1	1
3			Plate, Attaching, Calf Pad (15 - 17" W)	1	1
4			Plate, Articulation, Female	1	1
5			Plate, Adjustment Angle	1	1
6			Plate, Articulation	1	1
7			Knob, Locking, Star	1	1
8			Pin, Spring (M8 x 55mm)	1	1
9			Pin, Spring (M8 x 50mm)	1	1
10			Clamp, Calf Pad Plate	1	4
11			Screw, Socket, Button Head (M6 x 16mm)	1	1
12			Spacer, Sleeve	1	1
13			Washer (2.4mm x 15.6mm)	1	1
14	Ring, Locking	1	1		
15	Screw, VBA (M3.5 x 15)	1	4		
1	A	1169053	Calf Pad Assembly w/ Hardware (18 - 20" W)	1	1
2			Pad, Calf (18 - 20" W)	1	1
3			Plate, Attaching, Calf Pad (18 - 20" W)	1	1
4			Plate, Articulation, Female	1	1
5			Plate, Adjustment Angle	1	1
6			Plate, Articulation	1	1
7			Knob, Locking, Star	1	1
8			Pin, Spring (M8 x 50mm)	1	1
9			Pin, Spring (M8 x 55mm)	1	1
10			Clamp, Calf Pad Plate	1	4
11			Screw, Socket, Button Head (M6 x 16mm)	1	1
12			Spacer, Sleeve	1	1
13			Washer (2.4mm x 15.6mm)	1	1
14			Ring, Locking	1	1
15			Screw, VBA (M3.5 x 15)	1	4

NOTE: A - Includes items 2 - 15.