



One Arm Drive

Urethane (DUHALU, DUHARU), Pneumatic (DUHARP, DUHARL)

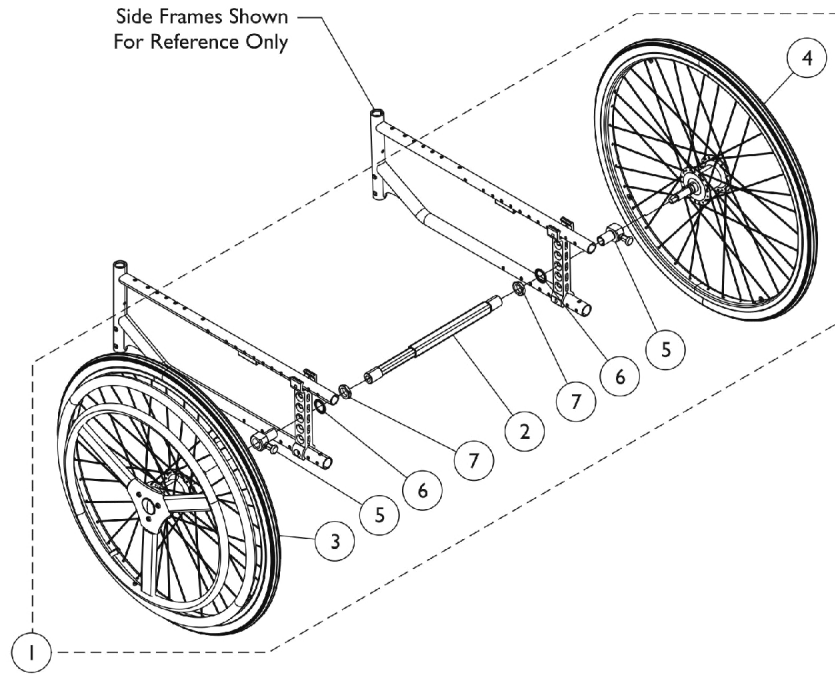

Solid Tire

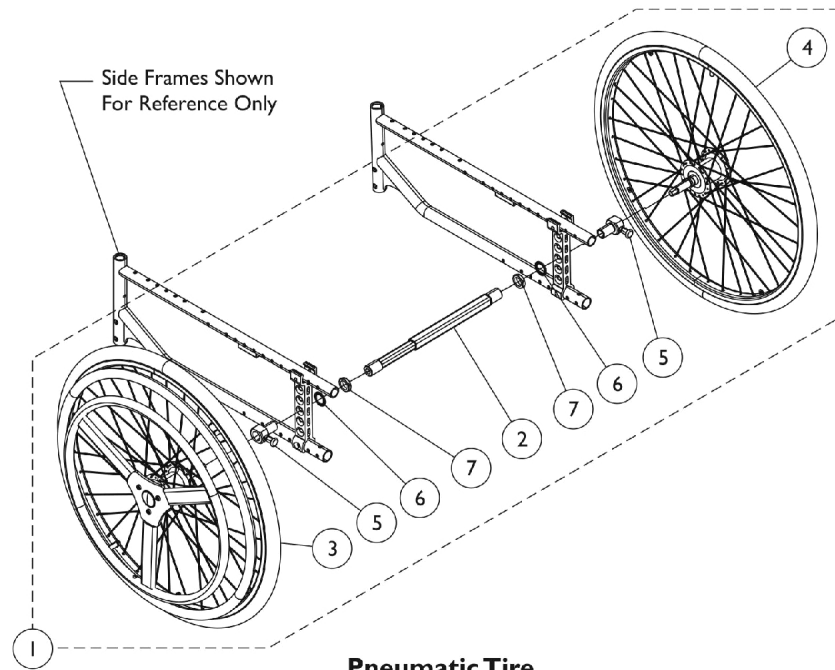

Pneumatic Tire

One Arm Drive

Urethane (DUHALU, DUHARU), Pneumatic (DUHARP, DUHARL)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1513269	One Arm Drive Assembly, Pneumatic w/ Hardware (15 - 17" W)	1	1
2			Shaft, Telescopic, One Arm Drive (15 - 17" W)	1	1
3			Wheel, Active, Pneumatic, DHR (24")	1	1
4			Wheel, Passive, Pneumatic, DHR (24")	1	1
5			Bushing, Quick Release	1	2
6			Washer, Serrated (M20.3 x 30mm x 1.5mm)	1	2
7			Nut, One Arm Drive	1	2
1	A	1513270	One Arm Drive Assembly, Pneumatic w/ Hardware (18 - 20" W)	1	1
2			Shaft, Telescopic, One Arm Drive (18 - 20" W)	1	1
3			Wheel, Active, Pneumatic, DHR (24")	1	1
4			Wheel, Passive, Pneumatic, DHR (24")	1	1
5			Bushing, Quick Release	1	2
6			Washer, Serrated (M20.3 x 30mm x 1.5mm)	1	2
7			Nut, One Arm Drive	1	2
1	A	1513271	One Arm Drive Assembly, Solid w/ Hardware (15 - 17" W)	1	1
2			Shaft, Telescopic, One Arm Drive (15 - 17" W)	1	1
3			Wheel, Solid, Active, DHR (24")	1	1
4			Wheel, Solid, Passive (24")	1	1
5			Bushing, Quick Release	1	2
6			Washer, Serrated (M20.3 x 30mm x 1.5mm)	1	2
7			Nut, One Arm Drive	1	2
1	A	1513272	One Arm Drive Assembly, Solid w/ Hardware (18 - 20" W)	1	1
2			Shaft, Telescopic, One Arm Drive (18 - 20" W)	1	1
3			Wheel, Solid, Active, DHR (24")	1	1
4			Wheel, Solid, Passive (24")	1	1
5			Bushing, Quick Release	1	2
6			Washer, Serrated (M20.3 x 30mm x 1.5mm)	1	2
7			Nut, One Arm Drive	1	2

NOTE: A - Includes items 3 - 7.