


Bath Board (Model 9664 - Discontinued 6/07)

Model 9664
Bath Board - Discontinued

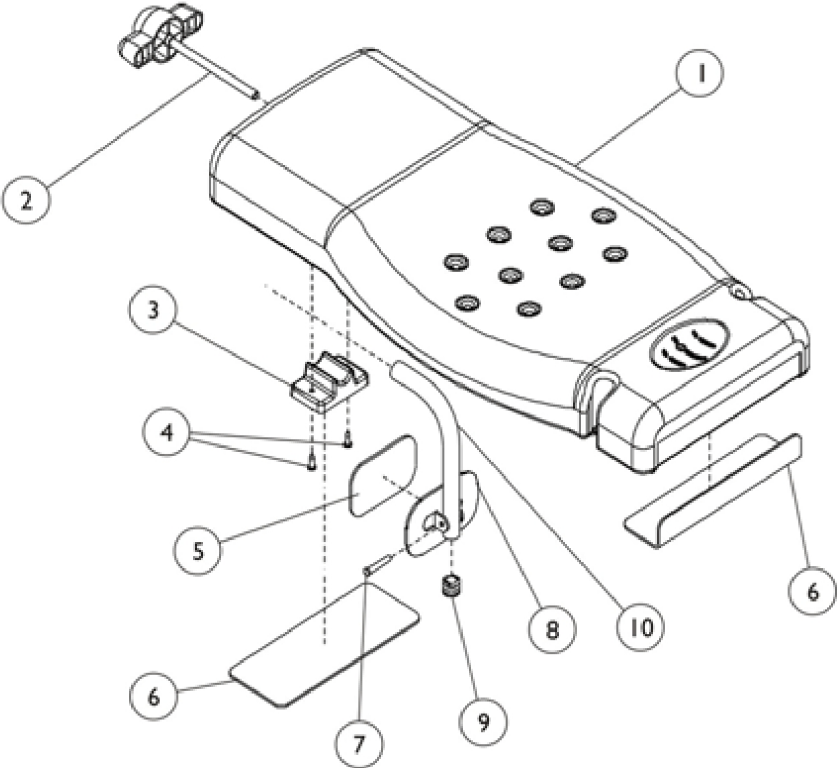

Bath Board (Model 9664 - Discontinued 6/07)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1060311-NLA	NLA - Board, Bath	1	1
2		1052984-NLA	NLA - Knob with Threaded Rod, White	1	1
3		1060378-NLA	NLA - Bracket, Support, White	1	1
4		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	2
5		1052960-NLA	NLA - Pad, Non-Skid without Holes	1	1
6		1060380-NLA	NLA - Pad, Non-Skid without Holes	1	2
7		1061430-NLA	NLA - Pin, Self Locking Clevis	1	1
8		1052981-NLA	NLA - Bracket, Adjustable, White	1	1
9		1040840-NLA	NLA - Plug Button, Grey (7/8")	1	1
10		1061428-NLA	NLA - Tube, Bracket, White	1	1

NOTE: Replacement parts for 1364P, 1365P are No Longer Available. Please contact Customer Service for replacement parts.