


Back Cane, Spreader Bar and Push Handle Assembly

Manual Tilt (12" - 16" W Nylon Back Uph.) (12" - 15" W Conv. Back) - TDX
SPREE

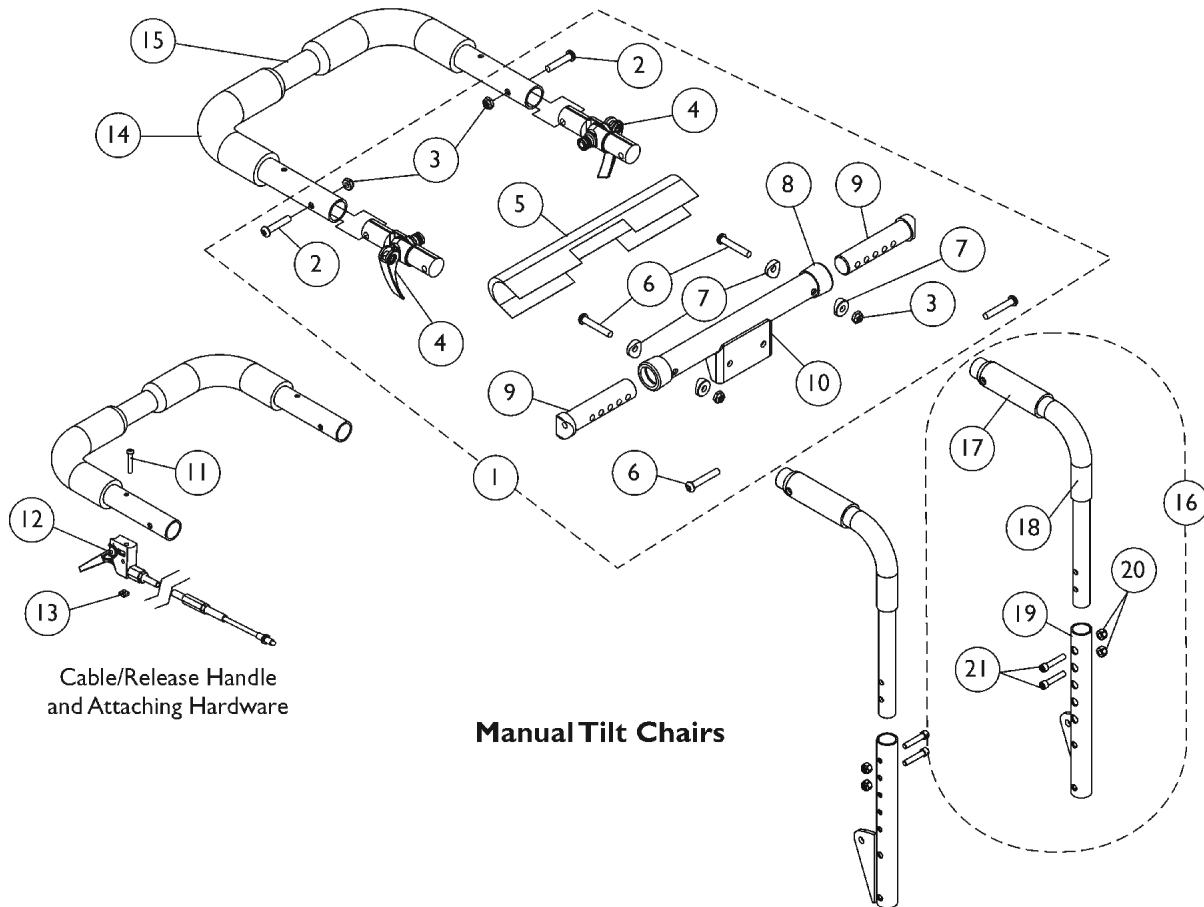

Back Cane, Spreader Bar and Push Handle Assembly

Manual Tilt (12" - 16" W Nylon Back Uph.) (12" - 15" W Conv. Back) - TDX SPREE

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1126148-NLA	NLA - NLA - Kit, Spreader Bar with Headrest Bracket (12"-16" Adj. Width)	1	1
2			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	2
3			Locknut (1/4-20)	1	4
4			Folding Hinge Assembly	1	2
5			Cover, Spreader Bar w/ Headrest Bracket	1	1
6			Screw, Socket Button Head (1/4-20 x 1-1/2")	1	4
7			Spacer, Coved (1/4 x 11/16 x 1/2")	1	4
8			Anti-Rattle (1")	1	2
9			NLA - Insert, Spreader Bar Connector, Black	1	2
10			NLA - Tube, Spreader Bar w/ Headrest Bracket, Black	1	1
11		1028513-NLA	NLA - Screw, Socket Head (#10-32 x 1-1/4")	1	1
12		1126103-NLA	NLA - Release Handle Assembly w/ Cable	1	1
13		1086207	Nut, Square (#10-32)	1	1
14		1133698	Grip, Foam Hand (9" L)	1	2
15		1082620-NLA	NLA - Handle, Push w/ Foam Grip, Black (12" W)	1	1
15		1082621-NLA	NLA - Handle, Push w/ Foam Grip, Black (13" W)	1	1
15		1082622-NLA	NLA - Handle, Push w/ Foam Grip, Black (14" W)	1	1
15		1082623	Handle, Push w/ Foam Grip, Black (15" W)	1	1
15		1082624	Handle, Push w/ Foam Grip, Black (16" W)	1	1
16	B	1152733	Kit, Back Cane Assembly with Hardware (12"-15" Back Height) - Right	1	1
17			Grip, Hand, Foam (4")	1	1
18			Handle, Push, Black	1	1
19			Cane, Back, Lower, Black - Right (12-15" H)	1	1
20			Screw, Socket Head (1/4-20 x 1-1/4")	1	2
21	B	1152734	Locknut (1/4-20)	1	2
16			Kit, Back Cane Assembly with Hardware (12"-15" Back Height) - Left	1	1
17			Grip, Hand, Foam (4")	1	1
18			Handle, Push, Black	1	1
19			Cane, Back, Lower, Black - Left (12-15" H)	1	1
20	B	1152735	Screw, Socket Head (1/4-20 x 1-1/4")	1	2
21			Locknut (1/4-20)	1	2
16			Kit, Back Cane Assembly with Hardware (16"-19" Back Height) - Right	1	1
17			Grip, Hand, Foam (4")	1	1
18			Handle, Push, Black	1	1
19	B	1152736	Cane, Back, Lower, Black - Right (16-19" H)	1	1
20			Screw, Socket Head (1/4-20 x 1-1/4")	1	2
21			Locknut (1/4-20)	1	2
16			Kit, Back Cane Assembly with Hardware (16"-19" Back Height) - Left	1	1
17			Grip, Hand, Foam (4")	1	1
18			Handle, Push, Black	1	1

Back Cane, Spreader Bar and Push Handle Assembly

Manual Tilt (12" - 16" W Nylon Back Uph.) (12" - 15" W Conv. Back) - TDX SPREE

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
19	B	1152737	Cane, Back, Lower, Black - Left (16-19" H)	1	1
20			Screw, Socket Head (1/4-20 x 1-1/4")	1	2
21			Locknut (1/4-20)	1	2
16			Kit, Back Cane Assembly with Hardware (20"-22" Back Height) - Right	1	1
17			Grip, Hand, Foam (4")	1	1
18			Handle, Push, Black	1	1
19			Cane, Back, Lower, Black - Right (20 - 22" H)	1	1
20			Screw, Socket Head (1/4-20 x 1-1/4")	1	2
21	B	1152738	Locknut (1/4-20)	1	2
16			Kit, Back Cane Assembly with Hardware (20"-22" Back Height) - Left	1	1
17			Grip, Hand, Foam (4")	1	1
18			Handle, Push, Black	1	1
19			Cane, Back, Lower, Black - Left (20 - 22" H)	1	1
20			Screw, Socket Head (1/4-20 x 1-1/4")	1	2
21			Locknut (1/4-20)	1	2
NOTE: A - Includes items 2-10 B - Includes items 17-21					