


Back Canes - Adjustable Angle w/ Hardware (14 - 18" H)

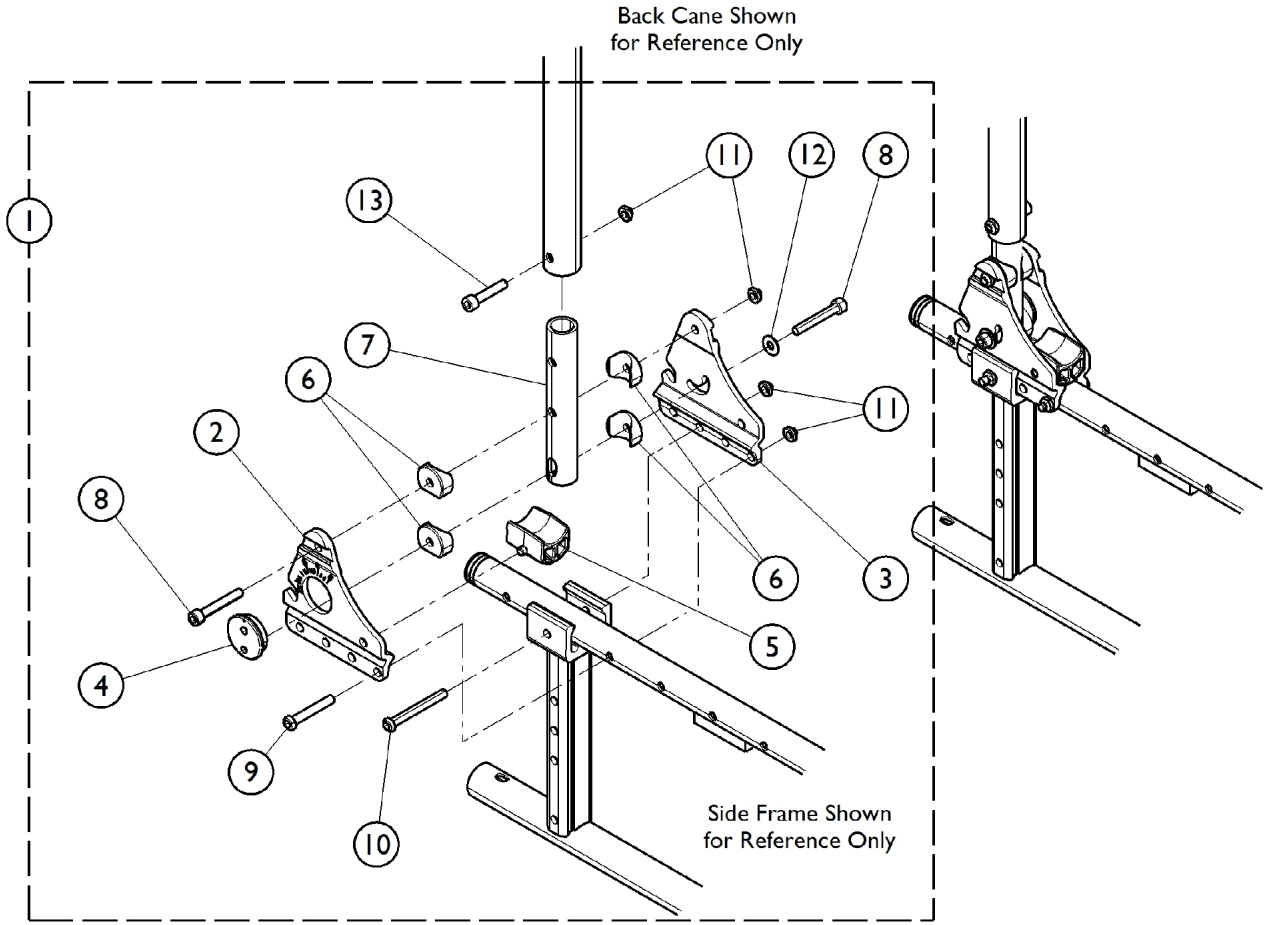

Back Canes - Adjustable Angle w/ Hardware (14 - 18" H)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1169695	Kit, Adjustable Angle Back Canes w/ Hardware, 8 Degree (15 - 19") (Pair)	1	1
			Back Cane, Adjustable Height, 8 Degree (15 - 19" H)	1	2
			Insert, Back Cane, Adjustable Height (14 - 18" H)	1	2
			Plate, Adjustable Angle, Inner - Right	1	1
			Plate, Adjustable Angle, Inner - Left	1	1
			Plate, Adjustable Angle, Outer - Right	1	1
			Plate, Adjustable Angle, Outer - Left	1	1
			Spacer, Coved (1/4 x 1 x 13/32")	1	8
			Support, Seat Tube, Rear	1	2
			Backrest, Excenter	1	2
			Screw, Socket Head Cap (M6 x 1 x 30)	1	2
			NLA - Screw, Socket Head Cap (M6 x 1 x 40mm)	1	4
			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
			Screw, Socket Button Head (M6 x 53mm)	1	2
			Washer (1/4 x 9/16 x 1/16")	1	2
			NLA - Locknut (M6 x 1)	1	8
3		1179667	Plate, Adjustable Angle, Inner - Right	1	1
3		1179668	Plate, Adjustable Angle, Inner - Left	1	1
5		1533913	Support, Seat Tube, Rear	1	2
7		1169321	Insert, Back Cane, Adjustable Height (14 - 18" H)	1	2
NOTE: A - Includes items 2 - 13.					