


Shower Chair - Compact Blow Molded Adjustable Height

Model 75-1

Model 75-3 Discontinued 12/1/04


Without Back

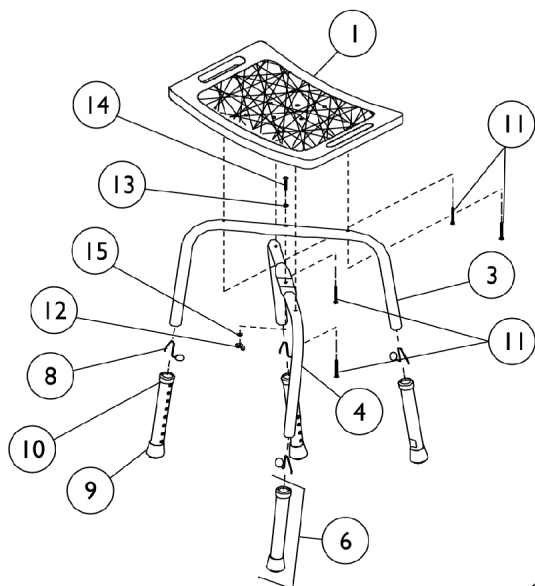

Model 85-1

Model 85-3 Discontinued 12/1/04

With Back

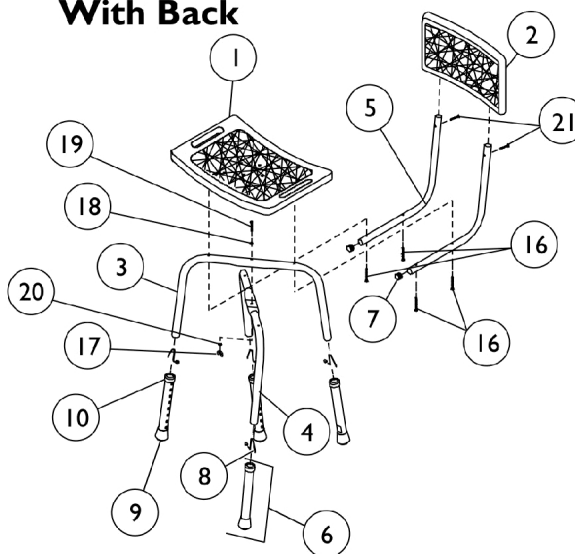

Shower Chair - Compact Blow Molded Adjustable Height

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1079341-NLA	NLA - Seat	1	1
2		1079340-NLA	NLA - Back	1	1
3		1079314-NLA	NLA - Leg, Top	1	1
4		1079315-NLA	NLA - Leg, Bottom	1	1
5	C	1083245-NLA	NLA - Tube, Backrest Support	1	2
6	A,B	1065346-NLA	NLA - Leg Extension Assembly	1	4
7		1027096	Plug Button, Grey (7/8")	1	2
8		1094945	Button, Snap	1	4
9		1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
10		1063124	Anti-Rattle, Grey (1-1/8")	1	4
	D,E,H	1074249-NLA	NLA - Kit, Hardware	1	1
11			NLA - Screw, Slotted Hex Washer Head (1/4-14 x 1-1/2")	1	4
12			NLA - Nut, Wing (1/4-20)	1	1
13			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	1
14			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
15			NLA - Washer, External Tooth Lock (1-1/4")	1	1
	F,G,H	1079912-NLA	NLA - Kit, Hardware	1	1
16			NLA - Screw, Phillips Pan Head (1/4-14 x 2-1/4")	1	4
17			NLA - Nut, Wing (1/4-20)	1	1
18			NLA - Washer, Steel Flat (1/4 x 5/8 x 1/16")	1	1
19			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	1
20			NLA - Washer, External Tooth Lock (1-1/4")	1	1
21			NLA - Screw, Phillips Pan Head Tap (#8-18 x 1/2")	1	2
NOTE: A - Due to style changes, when replacing Leg Extension after 3-2-98, order (4) each, and also replace Item 8 (Snap Button). B - Includes Items 9 and 10. C - Includes Plug Buttons. D - To be used with Model 75-1, 3. E - Kit Includes Items 11-15. F - To be used with Model 85-1, 3. G - Kit Includes Items 16-21. H - Not Shown.					