
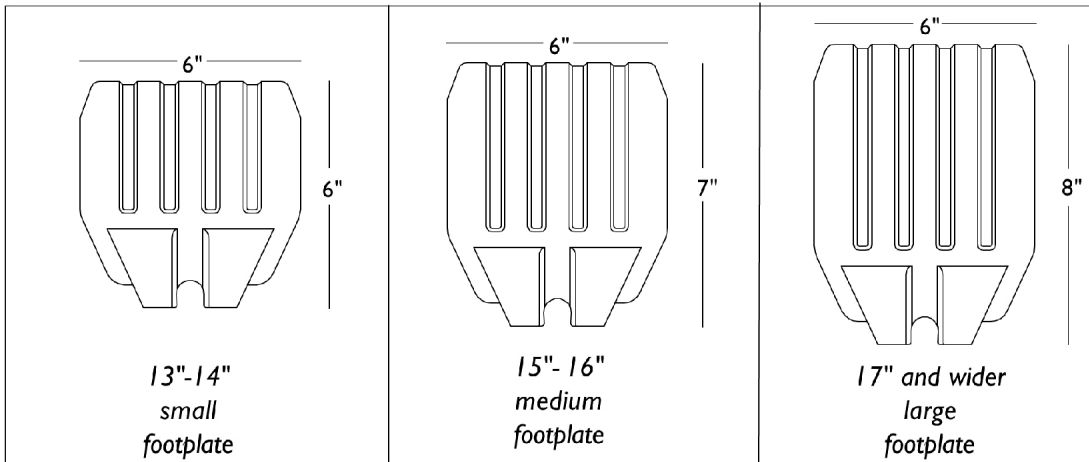


# Pivot/ Slide Tube and Composite Footplates

NOTE: Heel loops are not included. See footrest and legrest accessories.

NOTE: For complete pivot/slide tube footplate assemblies, see front rigging reference charts.

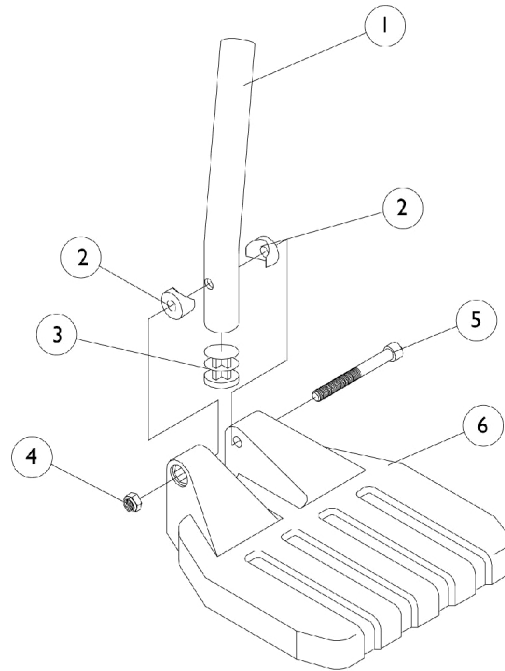

## Pivot/ Slide Tube and Composite Footplates

**NOTE:** Heel loops are not included. See footrest and legrest accessories.

**NOTE:** For complete pivot/slide tube footplate assemblies, see front riggings reference charts.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1057852	Tube, Pivot/Slide, Curved End, Steel, Black (9-7/8")	1	1
1	D	1057854	Tube, Pivot/Slide, Curved End, 2" Longer, Steel, Black	1	1
1	D	1058418	Tube, Pivot/Slide, Curved End, 4" Longer, Steel, Black	1	1
2		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	2
3		1027097	Plug Button, Black (3/4")	1	1
4		7930323-NLA	NLA - Package, Locknut (1/4-20)	1	1
5		7930366	Package, Socket Head Screw (1/4-20 x 2-1/4")	1	1
6	A	1026879	Footplate, Composite, Small, Black (6" W x 6" L)	1	1
6	B	1026880	Footplate, Composite, Medium, Black (7" W x 6" L)	1	1
6	C	1026881	Footplate, Composite, Large, Black (8" W x 6" L)	1	1
<p><b>NOTE:</b> A - 6" W x 6" L (13-14" Wide Chairs) and (15-17" Wide for 70STAPER and PW93ST only).            B - 7" W x 6" L (15-16" Wide Chairs) and (18-19" Wide for 70STAPER and PW93ST only).            C - 8" W x 6" L (17" and Wider Chairs) and (20" and Wider for 70STAPER and PW93ST only).            D - ALPT Option</p>					