


Orbit Seat Option Stroller Handles - 1483

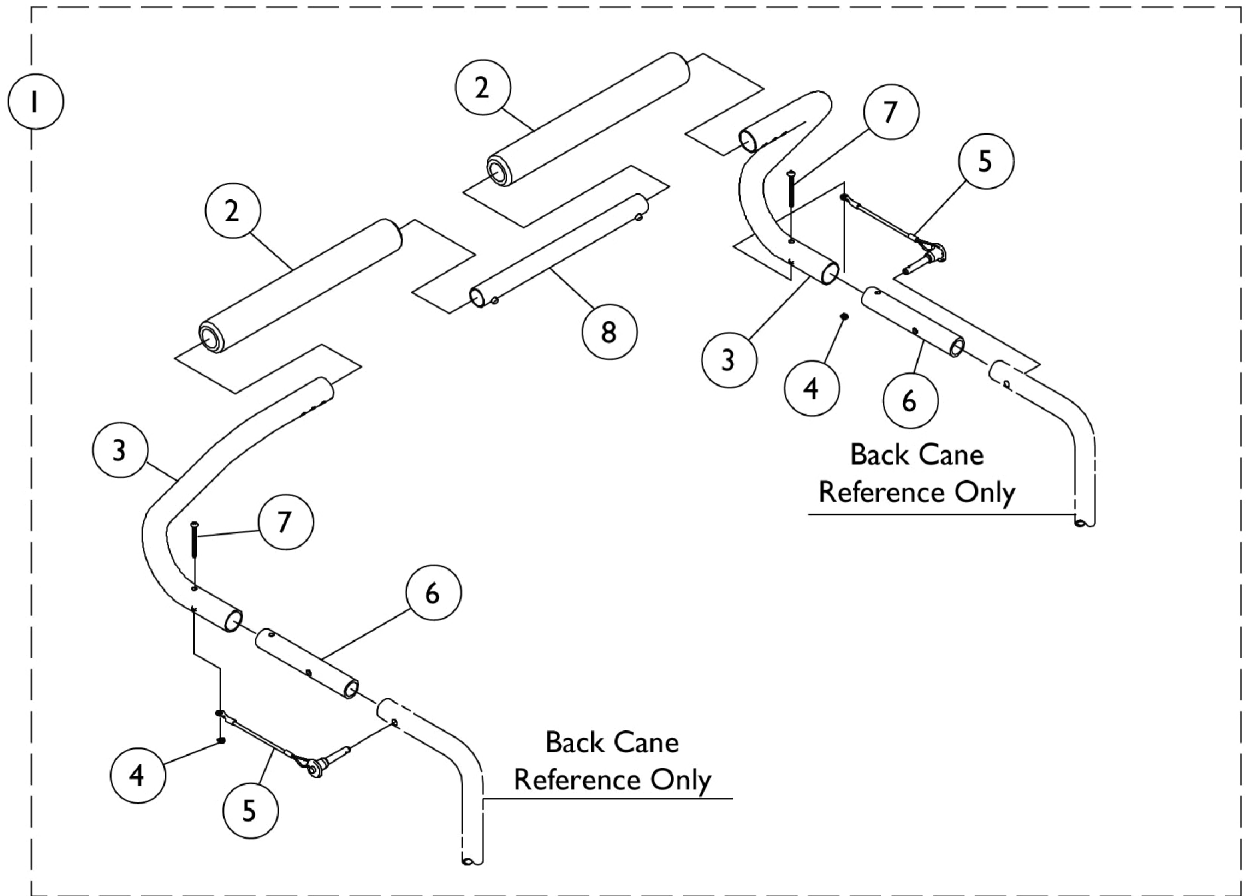

Orbit Seat Option Stroller Handles - 1483

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	1071194-NLA	NLA - Stroller Handle Assembly (10-12")	1	1
1	A,C	1071193-NLA	NLA - Stroller Handle Assembly (13-15")	1	1
1	A,D	1075082	Stroller Handle Assembly (16-18")	1	1
2		1133698	Grip, Foam Hand (9" L)	1	2
3		1075091-NLA	NLA - Handle, Stroller, Black - Right	1	1
3		1075092-NLA	NLA - Handle, Stroller, Black - Left	1	1
4		1025735	Locknut (#10-32)	1	2
5		1045470	Pin, Quick Release with Lanyard	1	2
6		1061993-NLA	NLA - Tube, Insert	1	2
7		1107155	Screw, Phillips Pan Head Tap (#10-32 x 1-1/4")	1	2
8	B	1071175-NLA	NLA - Link Assembly, Stroller Handle	1	1
8	C	1071174-NLA	NLA - Link Assembly, Stroller Handle	1	1
8	D	1075083	Link Assembly, Stroller Handle	1	1
NOTE: A - Includes items 2 - 8. B - 10 - 12" Wide Chairs. C - 13 - 15" Wide Chairs. D - 16" Wide Chairs.					