
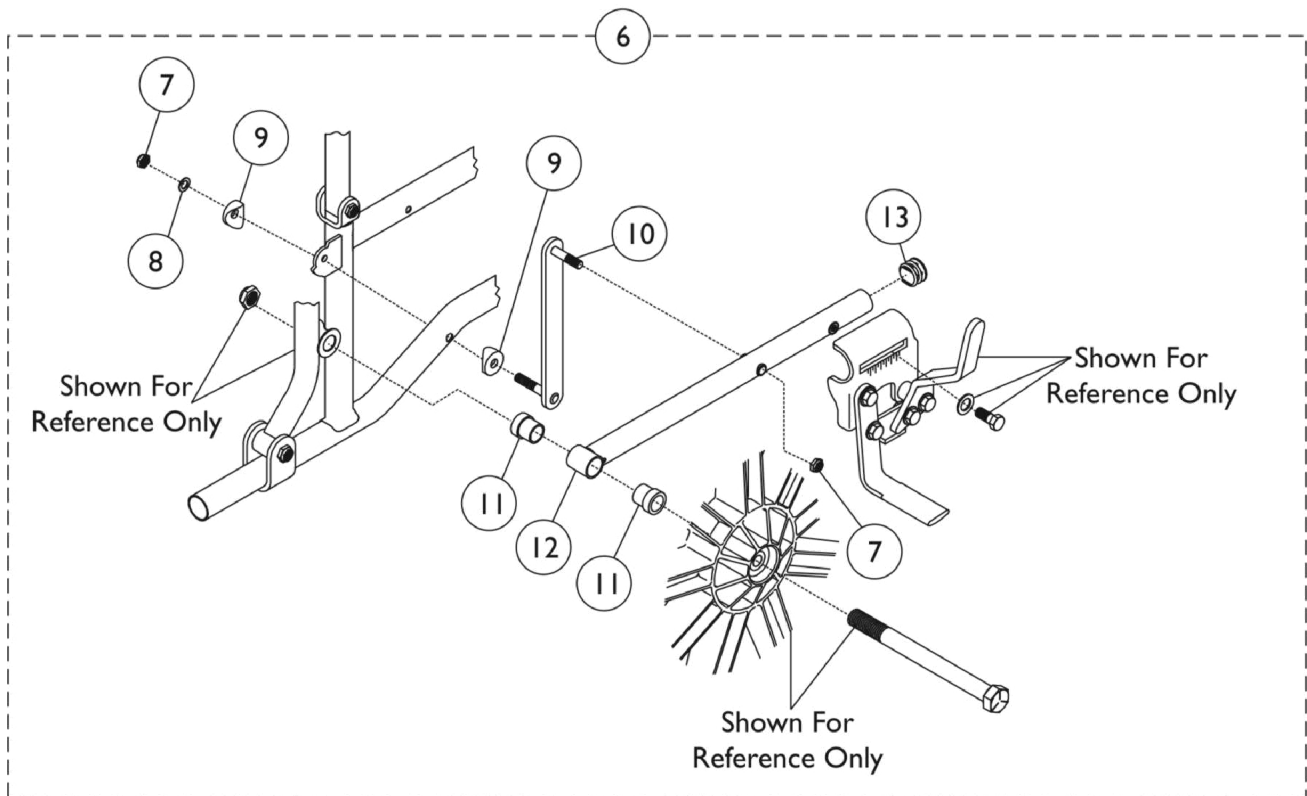
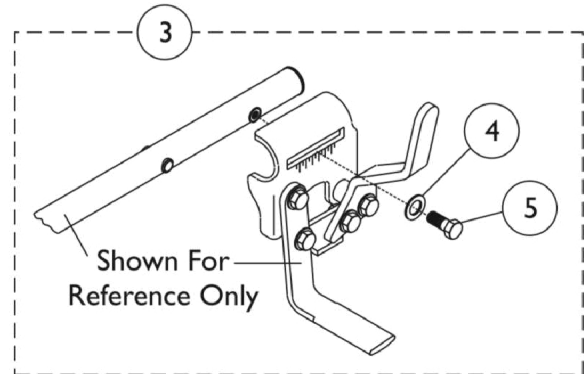


Hill Holder Wheel Locks

9697R, Conventional Arm, 9RC

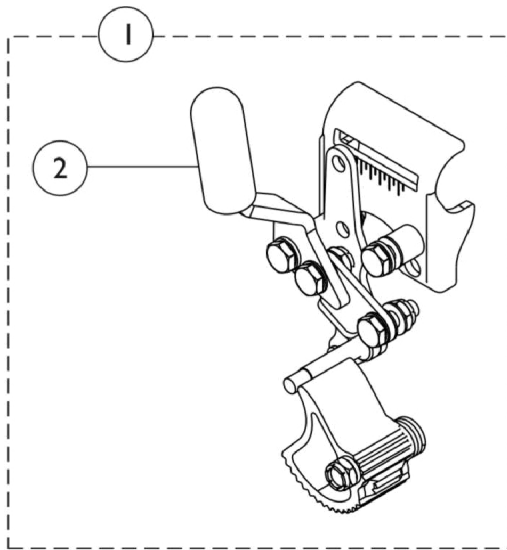

Hill Holder Wheel Locks

9697R, Conventional Arm, 9RC

Item Number	Note	Part Number	Description	Quantity Per		
				Package	Assembly	
1	A	1023495-NLA	NLA - Wheel Lock Assembly, Hill Holder, Push to Lock - Right	1	1	
1		1023496-NLA	NLA - Wheel Lock Assembly, Hill Holder, Push to Lock - Left	1	1	
1		1023499-NLA	NLA - Wheel Lock Assembly, Hill Holder, Pull to Lock - Right	1	1	
1		1023500-NLA	NLA - Wheel Lock Assembly, Hill Holder, Pull to Lock - Left	1	1	
2		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1	
3		1035964	Kit, Hardware, Wheel Lock (Pair)	1	1	
4			Washer (1/4 x 5/8 x 3/32")	1	2	
5			NLA - Screw, Hex Head (1/4-20 x 3/4")	1	2	
6		B	8881181877	Kit, Hardware, Wheel Lock	1	1
7				Locknut (5/16-24)	1	2
8				Washer, Steel (11/32 x 5/8 x 3/64")	1	1
9				Spacer, Coved (3/4 x 1 x 1/2")	1	2
10				Strap and Stud Assembly	1	1
11			Spacer (7/8")	1	2	
12			Arm, Pivot, Color	1	1	
13			Plug Button, Black (7/8")	1	1	
NOTE: A - Includes items 4 and 5. B - Includes items 7-13. Specify Silver Vein (71P) when ordering. Chrome is No Longer Available.						