


# Base Assembly (After 7/1/10)

## Model RPS350-1

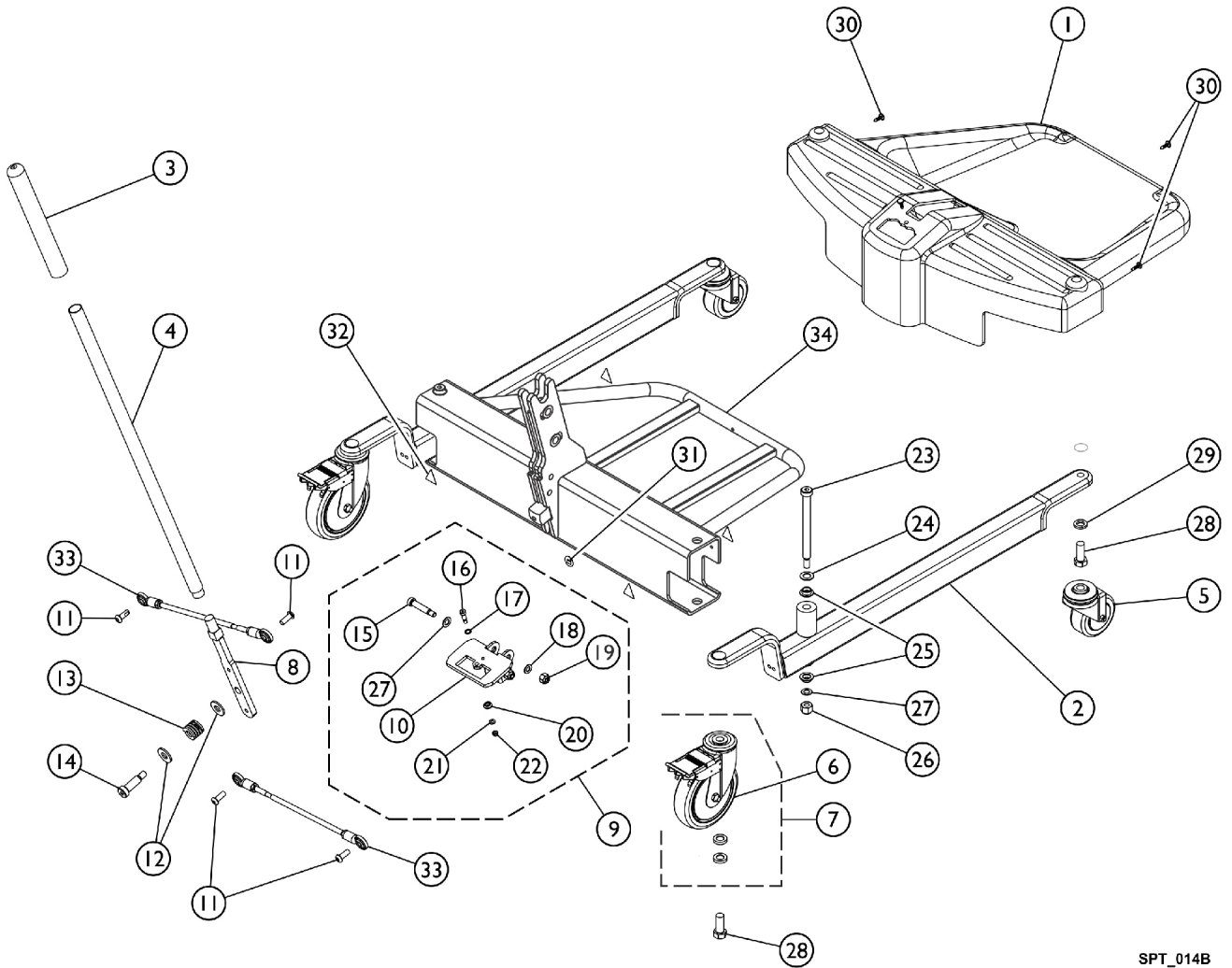

## Base Assembly (After 7/1/10) Model RPS350-1

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1156179	Shroud, Footrest	1	1
2		1095671	Leg, Single	1	2
3		1032246	Grip, Cane	1	1
4		1078287	Handle, Grey	1	1
5		1155118	Caster, Swivel (3")	1	2
6	A	1123539	Caster Assembly, Swivel w/ Brake (5")	1	2
7	B	1110421	Kit, Caster Assembly, Swivel w/ Brake (5") (Pair)	1	1
			Caster Assembly, Swivel w/ Brake (5")	1	2
			Bushing (15/32 x 1-1/16 x 9/32")	1	2
			Instruction Sheet	1	1
8		1156176	Handle, Mounting	1	1
9	C	1169830	Kit, Index Plate w/ Hardware	1	1
10			Bracket, Index Plate	1	1
15			Screw, Shoulder Hex Socket Head (3/8 x 1-1/2", 5/16-18)	1	2
16			Screw, Shoulder, Hex Socket (1/4 x 1/2", 10-32)	1	1
17			Washer, Flat (1/4 x 3/8 x 1/16)	1	1
18			Washer, Flat (5/16 x 5/8 x 1/16")	1	2
19			Locknut (5/16-18)	1	2
20			Roller, Index Plate	1	1
21			Washer, Flat (7/32 x 13/32 x 1/16)	1	1
22			Locknut (#10-32)	1	1
27			Washer, Flat (3/8 x 11/16 x 1/16")	1	2
11		1155720	Screw, Socket, Button Head w/ Patch (5/16-18 x 13/16")	1	4
12		1068305	Washer (1/2")	1	2
13		1064122	Spring, Compression (1 x 1-1/16 x 1/8")	1	1
14		1029134	Screw, Shoulder Hex Socket Head w/ Patch (1/2" x 1-1/4", 3/8-16)	1	1
23		1062424	Screw, Shoulder Hex Socket Head (1/2" x 5", 3/8-16)	1	2
24		1026174	Washer (1/2 x 7/8 x 3/64")	1	4
25		1062419	Bushing, Flanged	1	4
26		1065216	Locknut (3/8-16)	1	2
27		1065214	Washer, Flat (3/8 x 11/16 x 1/16")	1	3
28		1073884	Screw, Hex Head (1/2-13 x 1-1/4")	1	2
29		1014371	Lockwasher (1/2")	1	4
30		1023632	Clip, Shroud	1	4
31		1079113	Label, Year of Manufacture	1	1
32		1079112	Label, Attention	1	4
33		1065152	Tie Rod Assembly	1	2
34		1147175	Cross Support with Footrest	1	1

NOTE: A - Also available as a part of a kit, see item 7 (1110421).  
 B - Kit includes (2) Swivel Casters, (2) Bushings, and an Instruction Sheet.  
 C - Kit includes items 10, 15 - 22, and item 27.