


# Base Assembly

Model - RPA600-1

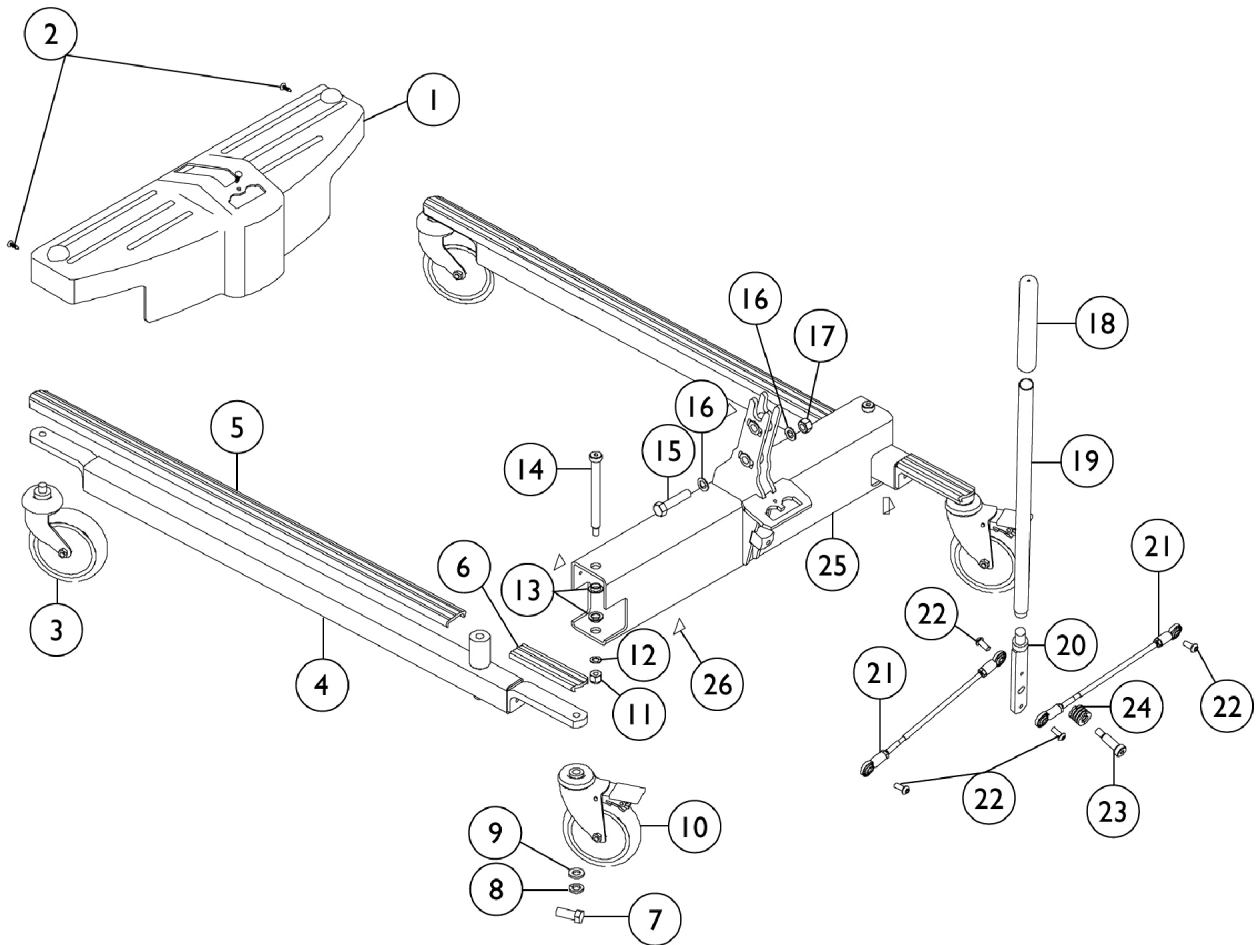

# Base Assembly

## Model - RPA600-1

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E	1078290-NLA	NLA - Shroud	1	1
1		1156185	Shroud	1	1
2		1023632	Clip, Shroud	1	3
3	A	1070957-NLA	NLA - Caster, Swivel, Front - Adjustable Base (5")	1	2
3		1123545	Caster Assembly, Swivel (5")	1	2
	B	1110422	Kit, Caster Assembly, Swivel (5") (Pair)	1	1
3			Caster Assembly, Swivel (5")	2	1
4	D	1095672	Leg, Adjustable, Low Base, Grey	1	2
	B	1095674	Kit, Legs Adjustable, Low Base, Pair with Covers	1	1
4			Leg, Adjustable, Low Base, Grey	1	2
5			Cover, Front Leg (Long)	1	2
6			Cover, Rear Leg (Short) - Low Base/ Standard Base	1	2
5		1070114	Cover, Front Leg (Long)	1	2
6		1073195-NLA	NLA - Cover, Rear Leg - Standard Base	1	2
6		1079137	Cover, Rear Leg (Short) - Low Base/ Standard Base	1	2
7		1073884	Screw, Hex Head (1/2-13 x 1-1/4")	1	2
8		1014371	Lockwasher (1/2")	1	2
9		1073885-NLA	NLA - Washer (1/2 x 15/16 x 1/8")	1	2
10		1123539	Caster Assembly, Swivel w/ Brake (5")	1	2
	B,F	1110421	Kit, Caster Assembly, Swivel w/ Brake (5") (Pair)	1	1
			Caster Assembly, Swivel w/ Brake (5")	1	2
			Bushing (15/32 x 1-1/16 x 9/32")	1	2
			Instruction Sheet	1	1
11		1065216	Locknut (3/8-16)	1	2
12		1065214	Washer, Flat (3/8 x 11/16 x 1/16")	1	2
13		1062419	Bushing, Flanged	1	4
14		1062424	Screw, Shoulder Hex Socket Head (1/2" x 5", 3/8-16)	1	2
15		1078979	Screw, Hex Head (1/2-13 x 2-1/4")	1	1
16		1026174	Washer (1/2 x 7/8 x 3/64")	1	2
17		1078981	Locknut (1/2-13)	1	1
18		1032246	Grip, Cane	1	1
19		1078287	Handle, Grey	1	1
20		1156176	Handle, Mounting	1	1
21		1065152	Tie Rod Assembly	1	2
22		1155720	Screw, Socket, Button Head w/ Patch (5/16-18 x 13/16")	1	4
23		1029134	Screw, Shoulder Hex Socket Head w/ Patch (1/2" x 1-1/4", 3/8-16)	1	1
24		1064122	Spring, Compression (1 x 1-1/16 x 1/8")	1	1
25	C	1078300	Cross Support	1	1
26		1079112	Label, Attention	1	4

# Base Assembly

## Model - RPA600-1

NOTE: A - Item no longer available, refer to part #1123545.  
B - Not shown.  
C - Part is no longer offered as a service part as of 7/1/10, use 1132033 as the replacement. When ordering 1132033, you should also order 1169830 and 1156176.  
D - Both Legs must be replaced on Units built before 7/2/10. Order 1095674 as the replacement.  
E - Part is no longer available. Order 1156185 as a replacement.  
F - Bushings added 7/30/12 and used for 9805/9805P Lifts only to provide clearance for the caster to rotate freely.