


# Base Assembly - RPL600-1 (Before 7/2/10)

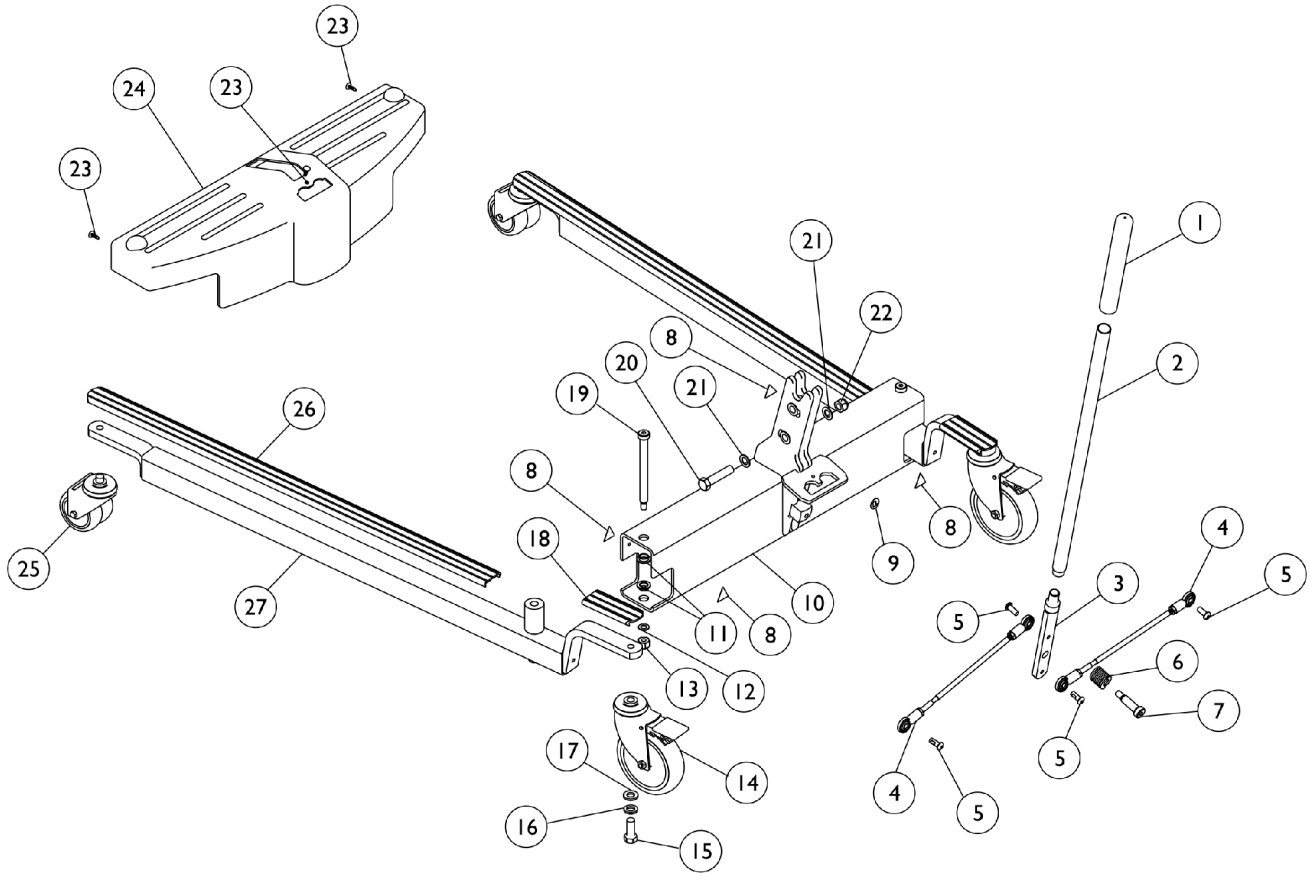

## Base Assembly - RPL600-1 (Before 7/2/10)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1032246	Grip, Cane	1	1
2		1078287	Handle, Grey	1	1
3		1078289-NLA	NLA - Handle, Mount	1	1
4		1065152	Tie Rod Assembly	1	2
5		1065215	Screw, Socket Button Head w/ Patch (5/16-18 x 3/4")	1	2
6		1064122	Spring, Compression (1 x 1-1/16 x 1/8")	1	1
7		1029134	Screw, Shoulder Hex Socket Head w/ Patch (1/2" x 1-1/4", 3/8-16)	1	1
8		1079112	Label, Attention	1	4
9		1079113	Label, Year of Manufacture	1	1
10	F	1078300	Cross Support	1	1
11		1062419	Bushing, Flanged	1	4
12		1065214	Washer, Flat (3/8 x 11/16 x 1/16")	1	2
1		1065216	Locknut (3/8-16)	1	1
14	D	1123539	Caster Assembly, Swivel w/ Brake (5")	1	2
	C,I	1110421	Kit, Caster Assembly, Swivel w/ Brake (5") (Pair)	1	1
			Caster Assembly, Swivel w/ Brake (5")	1	2
			Bushing (15/32 x 1-1/16 x 9/32")	1	2
			Instruction Sheet	1	1
15		1073884	Screw, Hex Head (1/2-13 x 1-1/4")	1	2
16		1014371	Lockwasher (1/2")	1	2
17		1073885-NLA	NLA - Washer (1/2 x 15/16 x 1/8")	1	2
18		1079137	Cover, Rear Leg (Short) - Low Base/ Standard Base	1	2
19		1062424	Screw, Shoulder Hex Socket Head (1/2" x 5", 3/8-16)	1	2
20		1078979	Screw, Hex Head (1/2-13 x 2-1/4")	1	1
21		1026174	Washer (1/2 x 7/8 x 3/64")	1	2
22		1078981	Locknut (1/2-13)	1	1
23		1023632	Clip, Shroud	1	3
24	H	1078290-NLA	NLA - Shroud	1	1
24		1156185	Shroud	1	1
25	E	1123560	Caster, Swivel Dual (3")	1	2
26		1070114	Cover, Front Leg (Long)	1	2
27	A,G	1095672	Leg, Adjustable, Low Base, Grey	1	2
	C	1095674	Kit, Legs Adjustable, Low Base, Pair with Covers	1	1
27			Leg, Adjustable, Low Base, Grey	1	2
26			Cover, Front Leg (Long)	1	2
18			Cover, Rear Leg (Short) - Low Base/ Standard Base	1	2
	B,C	1079810	Kit, Base - Low, Adjustable	1	1

NOTE: A - For Domestic Models only.  
 B - Kit includes items 1 - 19, 23 - 27, and Owner's Manual.  
 C - Not shown.  
 D - Also available as a pair, refer to part number 1110421.  
 E - Caster stem is threaded, no additional hardware needed.  
 F - Part is no longer serviceable in the field as of 7/1/10, use 1132033 as the replacement. When ordering 1132033, you should also order 1169830 and 1156176.  
 G - Both Legs must be replaced on Units built before 7/2/10. Order 1095674 as the replacement.  
 H - Part is no longer available. Order 1156185 as a replacement.  
 I - Bushings added 7/30/12 and used for 9805/9805P Lifts only to provide clearance for the caster to rotate freely.