


Swing Arms and Suspension Attaching Hardware

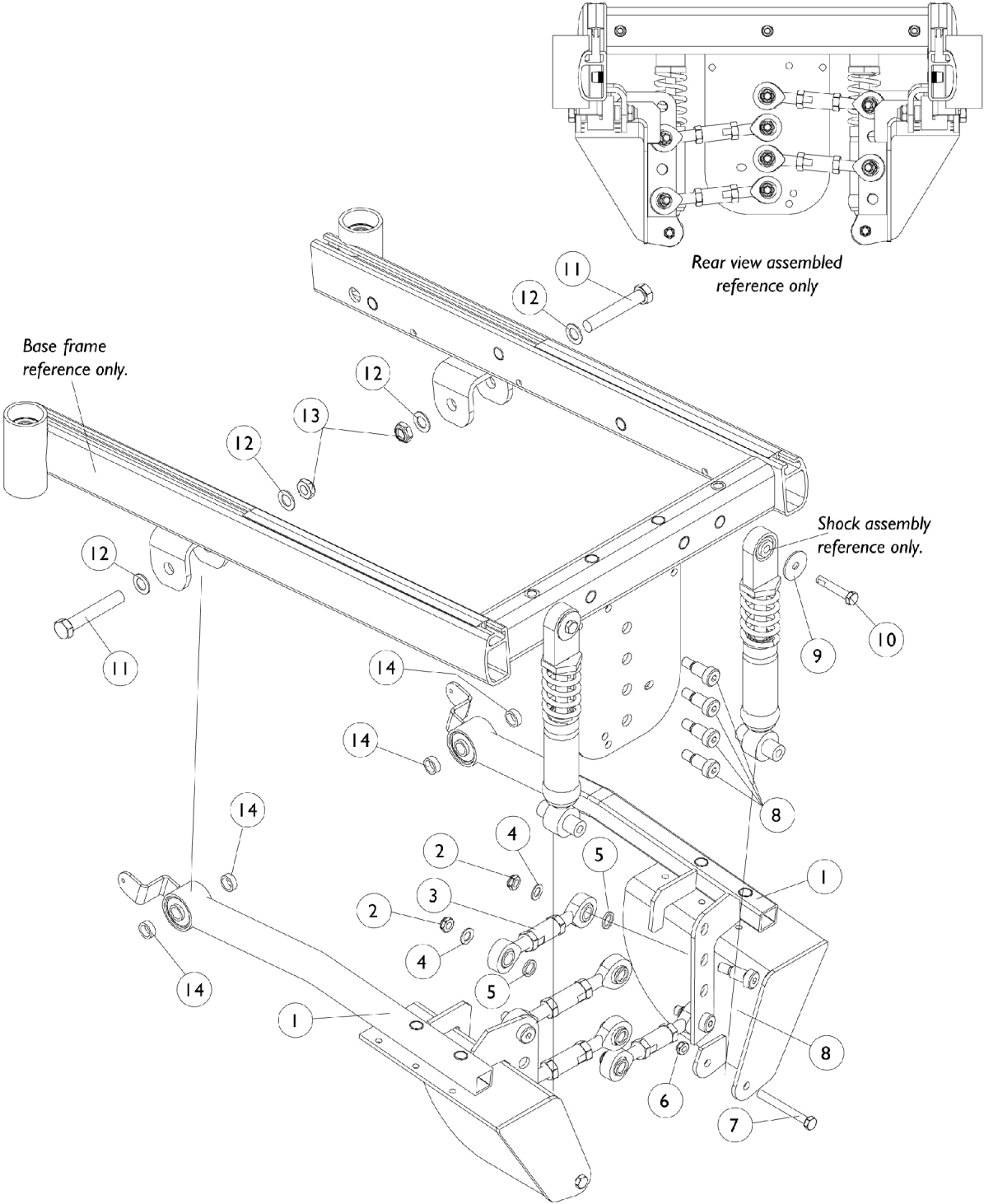

Swing Arms and Suspension Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1086287-NLA	NLA - Swing Arm w/ Rubber Bushing Assembly, Black - Right	1	1
1		1086288-NLA	NLA - Swing Arm w/ Rubber Bushing Assembly, Black - Left	1	1
2		1028486	Locknut, Low Profile (3/8-16)	1	8
3		1086324-NLA	NLA - Tie Rod Assembly	1	4
4		1065214	Washer, Flat (3/8 x 11/16 x 1/16")	1	8
5		1092408-NLA	NLA - Washer, Steel (1/2 x 11/16 x 3/32")	1	8
6		1028769	Locknut (5/16-18)	1	2
7		1042486	Screw, Hex Head (5/16-18 x 2-3/4")	1	2
8		1086313-NLA	NLA - Screw, Shoulder Hex Socket Head (1/2 x 3/4", 3/8-16)	1	8
9		1092409	Washer, Steel (5/16 x 1-1/4 x 1/16")	1	2
10		1043627-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1-3/4")	1	2
11		1090783-NLA	NLA - Screw, Hex Head (1/2-13 x 3")	1	2
12		1026174	Washer (1/2 x 7/8 x 3/64")	1	4
13		1050169-NLA	NLA - Locknut (1/2-13)	1	2
14		1086328-NLA	NLA - Spacer (1/2 x 11/16 x 1/4")	1	4