

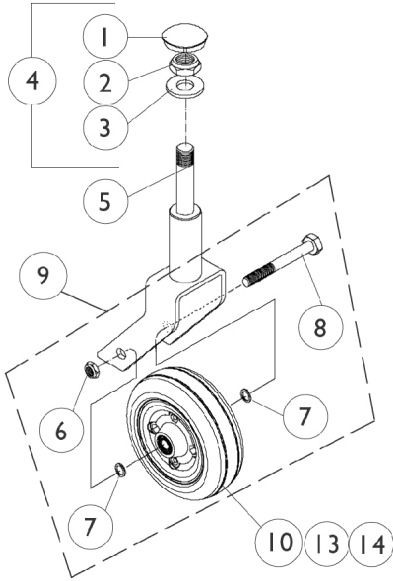


6" and 8" Caster

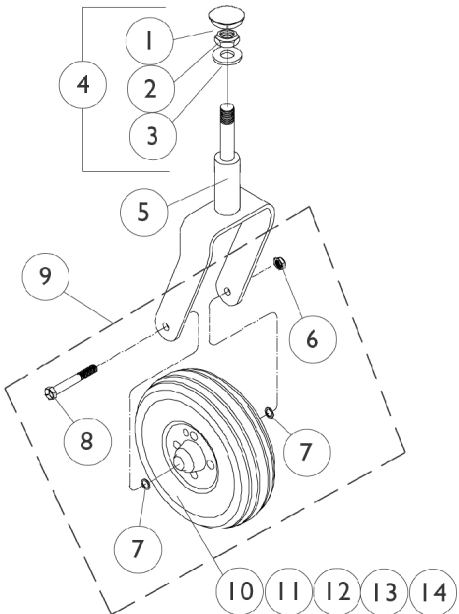

6" Caster Assembly

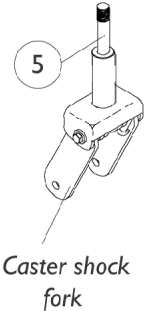

8" Caster Assembly

6" and 8" Caster

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027600-NLA	NLA - Cap, Black	1	1
2		1028550	Locknut (5/8-18)	1	1
3		1041444	Washer, Nylon (5/8 x 1-5/16 x 1/8")	1	1
4	A	1050173-NLA	NLA - Hardware, Caster Fork Mounting	1	1
5	B	8881039740-NL	NLA - Fork and Stem Assembly (8" Caster)	1	1
		A			
5	B	8881056370-NL	NLA - Fork, Shock, Color (8")	1	1
		A			
5	B	8881041458	Fork and Stem Assembly (6" Caster)	1	1
5	B	8881056379	Fork, Shock, Color (6")	1	1
6		7930331-NLA	NLA - Locknut (7/16-20")	25	1
7		7930329	Washer, Steel (15/32 x 5/8 x 3/64")	25	2
8		1025726	Screw, Hex Head (7/16-20 x 3-1/4")	1	1
8		1058744-NLA	NLA - Screw, Hex Head (7/16-20 x 3-1/2")	1	1
9		1042489	Hardware, Caster Axle	1	2
9		1058890-NLA	NLA - Hardware, Caster, Shock Fork (Pair)	2	1
10		1002002	Wheel Assembly w/ Bearings, Pneumatic (8" x 2")	1	1
10		1070811	Wheel Assembly, Semi-Pneumatic, Urethane, Light Grey (8" x 1-3/4")	1	1
10	C	1072014-NLA	NLA - Wheel Assembly, Semi-Pneumatic, Urethane (6" x 2")	1	1
10	D	1115179	Wheel Assembly, Semi-Pneumatic, Urethane, Deep Profile, Grey Tire/Black Hub (6" x 2")	1	1
10	D	1122066	Wheel Assembly, Semi-Pneumatic, Urethane, Deep Profile, Grey Tire/Black Hub (6" x 2") (Pair)	2	1
11		40176X034	Tire, Pneumatic (8" x 2")	1	1
12		1034942	Tube, Pneumatic Inner (8 x 2")	1	1
13		1006049	Bearing (7/16")	1	2
14		1020592	Spacer, Bearing (7/16 x 5/8 x 1-3/4")	1	1

NOTE: A - Includes Items 1-3.
 B - Specify frame color when ordering.
 C - Service part No Longer Available 8/15/03. Due to product improvement, order (pair) part number 1122066 dated after 8/14/03 so each side matches. The deep profile caster has 2 tread grooves w/ round corners instead of 4 tread grooves w/ square corners.
 D - Service part after 8/14/03