


Drive Wheels and Attaching Hardware

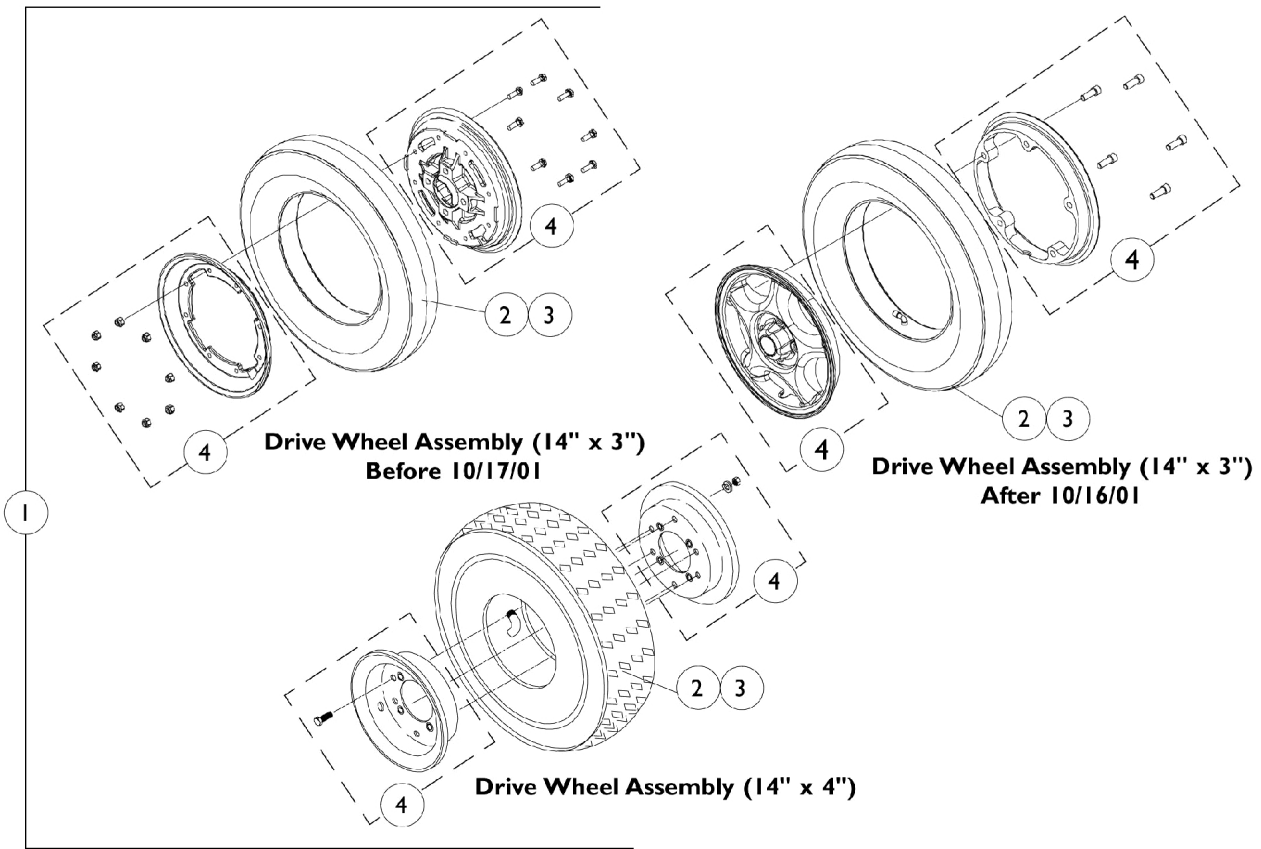

Drive Wheels and Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1-3	C	See Chart	#'s 1-3 Drive Wheels and Attaching Hardware	1	1
4	A	1090651-NLA	NLA - Kit, 8" Rim w/ Hardware (14" x 3" Wheel)	1	1
4	B	1104882-NLA	NLA - Kit, 8" Rim w/ Hardware (14 x 3" Wheel)	1	1
4		1042453-NLA	NLA - Kit, 6" Rim w/ Hardware (14" x 4" Wheel)	1	1
	A,D,E	7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	8
	A,D,E	7930333	Locknut (1/4-20)	25	8
	B,D,F	1101094	Screw, Socket Head w/ Patch (5/16-18 x 7/8")	1	5
	B,D,G	1024443-NLA	NLA - Screw, Hex Head (5/16-24 x 5/8")	1	6
	B,D,G	1024442-NLA	NLA - Washer, Lock (5/16 x 9/16 x 1/16")	1	6
	B,D,G	1024426-NLA	NLA - Nut, Hex (5/16-24)	1	6
NOTE: A - Before 10/17/01. B - After 10/16/01. C - See chart titled "Item #'s 1-3 Drive Wheels and Attaching Hardware." D - Not Shown E - Included in rim with hardware part number #1090651 before 10/17/01. F - Included in rim with hardware part number #1104882 after 10/16/01. G - Included in rim with hardware part number #1042453.					

Item #'s 1-3 Drive Wheels and Attaching Hardware

Description	#1 Wheel Complete	#1 Wheel Complete w/ Flat Free	#2 Tire	#2 Tire w/ Flat Free	#3 Tube
14" x 4"	1094017-NLA	1094016-NLA	1042474	1042478	1042475
14" x 3" Before 10/17/01	1090649-NLA	1090650-NLA	1039652	1101754	1039651
14" x 3" After 10/16/01	1104438	1104437	1039652	1101754	1039651