


Rear Battery Box

Group 24 Batteries

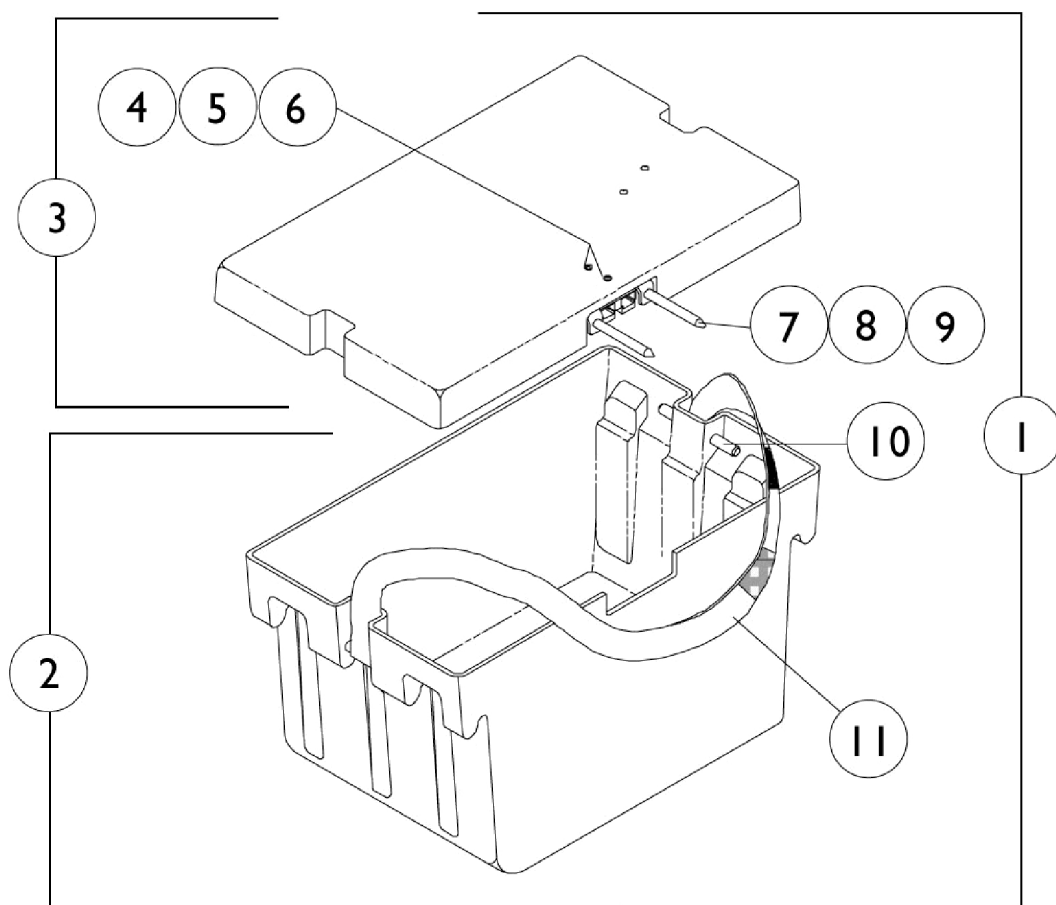

Rear Battery Box Group 24 Batteries

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1041440	Battery Box Assembly w/ Black Battery Connector, Rear	1	1
2		1041438	Battery Box Bottom Assembly, Rear	1	1
3		1085414	Battery Box Lid Assembly w/ Black Battery Connector, Rear	1	1
4		1039773-NLA	NLA - Screw, Phillips Pan Head (#6-32 x 1-1/8")	1	2
5		1043599-NLA	NLA - Washer, Flat (5/32 x 3/8 x 1/32")	1	2
6		P0870X050-NLA	NLA - Nut, Keps (#6-32)	1	2
7		1041421	Pin, Connector Guide	1	2
8		1041422	Ring, Retaining	1	4
9		1041420	Bracket, Connector Guide Mounting	1	1
10		1041433-NLA	NLA - Pin, Handle	1	2
11		1041434	Handle, Strap, Rear Battery Box, Black	1	1
	B,C	50000X166-NLA	NLA - Fuse - Buss 2-1/4" L (60 AMP)	1	1
	B,D	1089385	Fuse, MIDI (Non-G/B & G/B) (80 AMP)	1	1
	A,B	1039726-NLA	NLA - Harness, Rear Wiring	1	1
NOTE: A - Replace with Item 3 part # 1085414 battery box lid assembly. B - Not Shown C - Before 6/15/00 D - After 6/14/00					