
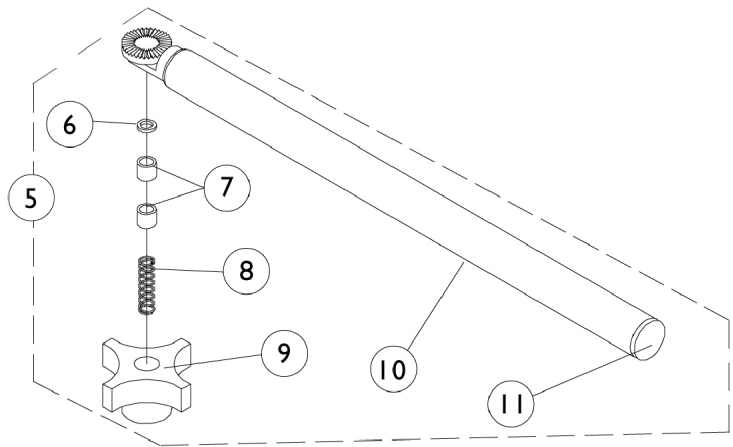


# MKIV Swivel Joystick Hardware

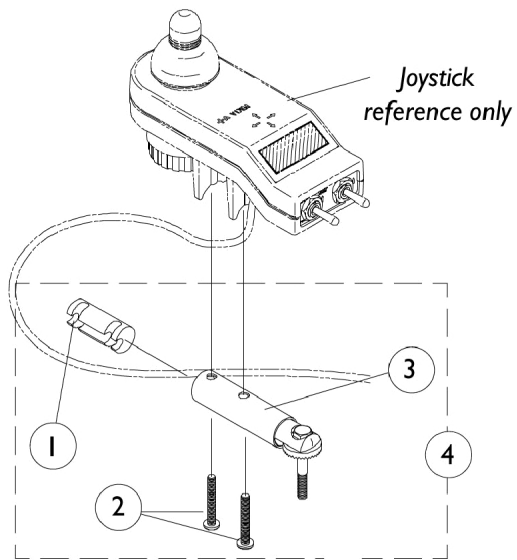

## MKIV Swivel Joystick Hardware

| Item Number | Note | Part Number | Description   | Quantity Per |          |
|-------------|------|-------------|---|--------------|----------|
|             |      |             |   | Package      | Assembly |
| 1           |      | 1093063     | Spacer, Joystick Tube, Black                              | 1            | 1        |
| 2           |      | 7930327-NLA | NLA - Package, Screw, Phillips Pan Head (#10-32 x 1-1/8") | 1            | 3        |
| 3           |      | 1049962-NLA | NLA - Bracket, Swivel Joystick, MKIV                      | 1            | 1        |
| 4           |      | 1049963-NLA | NLA - Bracket, Swivel Joystick with Hardware              | 1            | 1        |
| 5           |      | 1035986-NLA | NLA - Joystick Swivel Tube with Hardware, Black           | 1            | 1        |
| 6           |      | 1020551-NLA | NLA - Washer, Flat (1/4 x 3/8 x 1/16")                    | 1            | 1        |
| 7           |      | 1005844-NLA | NLA - Bushing   | 1            | 2        |
| 8           |      | 1002511-NLA | NLA - Spring, Hitch Bracket                               | 1            | 1        |
| 9           |      | 1023670-NLA | NLA - Knob, Clamping, Grey                                | 1            | 1        |
| 10          |      | 1035990-NLA | NLA - Tube, Swivel Joystick, Black                        | 1            | 1        |
| 11          |      | 1027095     | Plug Button, Black (7/8")                                 | 1            | 1        |