


# MKIV Mid-Line Swing Bracket Assembly

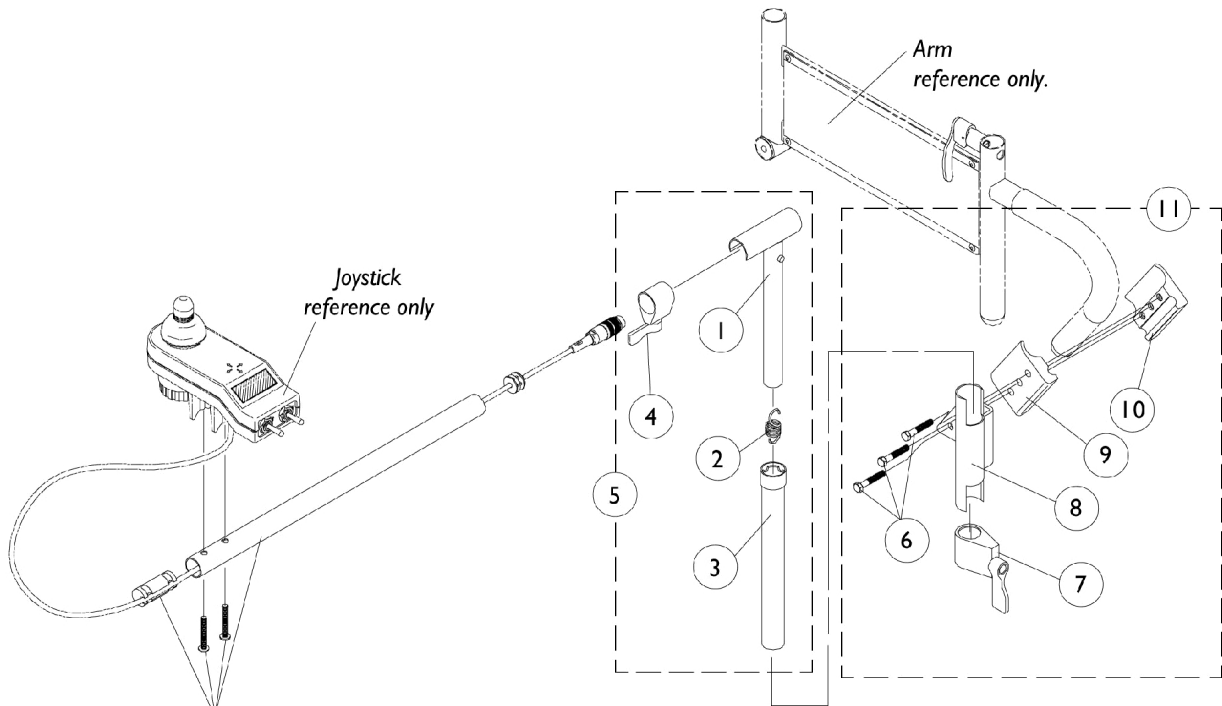

**NOTE:** Use existing joystick tube, screws and plug button.

## MKIV Mid-Line Swing Bracket Assembly

| Item Number | Note | Part Number | Description                                      | Quantity Per |          |
|-------------|------|-------------|--|--------------|----------|
|             |      |             |  | Package      | Assembly |
| 1           |      | 1035528-NLA | NLA - Tube, Adjustment, Inside, Mid-Line, Black  | 1            | 1        |
| 2           |      | 1016778-NLA | NLA - Spring, Extension (9/16 x 1-3/8" L)        | 1            | 1        |
| 3           |      | 1035860-NLA | NLA - Tube, Adjustment, Outside, Mid-Line, Black | 1            | 1        |
| 4           |      | 1027274     | Adjustment Lock Assemnbly, Black                 | 1            | 1        |
| 5           | C    | 1035852-NLA | NLA - Swing Bracket Assembly, Mid-Line, Black    | 1            | 1        |
| 6           |      | 7930349-NLA | NLA - Screw, Hex Head (1/4-20 x 1-1/8")          | 1            | 3        |
| 7           |      | 1027274     | Adjustment Lock Assemnbly, Black                 | 1            | 1        |
| 8           | A    | 1071879-NLA | NLA - Bracket, Joystick Mounting, Black          | 1            | 1        |
| 9           |      | 1027266     | Half Clamp, Non-Threaded Holes                   | 1            | 1        |
| 10          |      | 1027267-NLA | NLA - Half Clamp, Threaded Holes                 | 1            | 1        |
| 11          | A,B  | 1071880-NLA | NLA - Hardware, Attaching - Mid-Line             | 1            | 1        |

NOTE: A - Left side joystick mounting bracket or hardware must be ordered as a special.  
 B - Includes Items 6-10.  
 C - Includes items 1-4.