


Leg Link Assembly Components

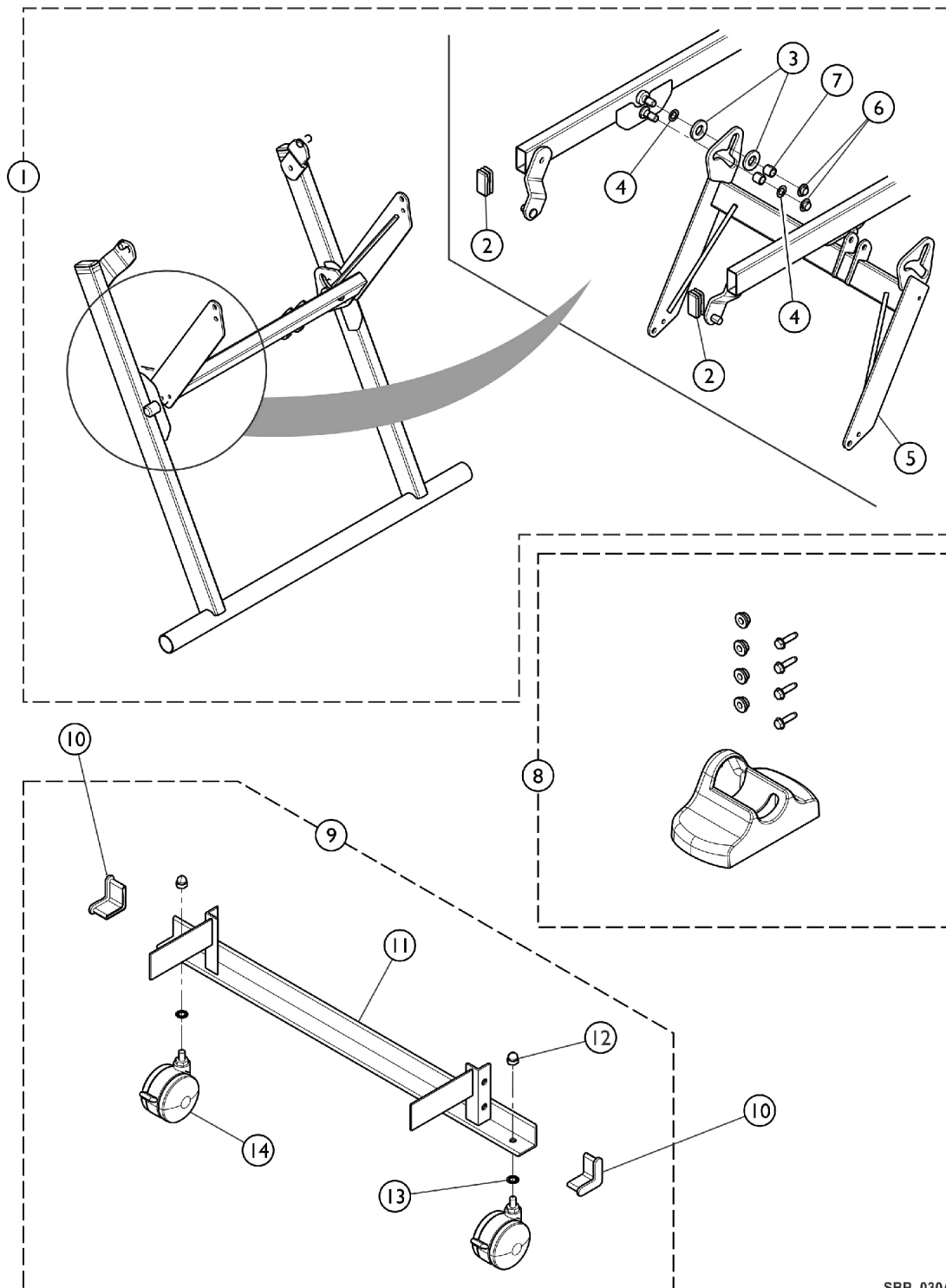

Leg Link Assembly Components

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1147534	Leg and Link Assembly	1	1
2		1186878-NLA	NLA - Plug, Universal (1" x 2" - PLG-1-2-14)	1	2
3		1147541	Washer, Flat - Nylon	1	2
4		1147612-NLA	NLA - Washer, Flat - Plain	1	2
5		1149704-NLA	NLA - Link, Stabilizer	1	1
6		1147519-NLA	NLA - Nut, Push	1	2
7		1145036-NLA	NLA - Bushing, Cam - Steel	1	2
8		1153739	Foot Stabilizer w/ Hardware	4	16
9	B	1156547-NLA	NLA - Caster Bracket Assembly, Bed	1	1
10		1156558-NLA	NLA - Cap, Angle Iron (1-1/2 x 1-1/2 x 11 ga.)	1	4
11		1156543	Caster Bracket, Bed	1	2
12		1147523-NLA	NLA - Nut, Acorn	1	2
13		1152559	Washer, Lock, Internal Tooth	1	2
14		1147527	Caster, Locking (3")	1	2
NOTE: A - Includes items 2 - 7. B - Includes items 11 - 14.					