
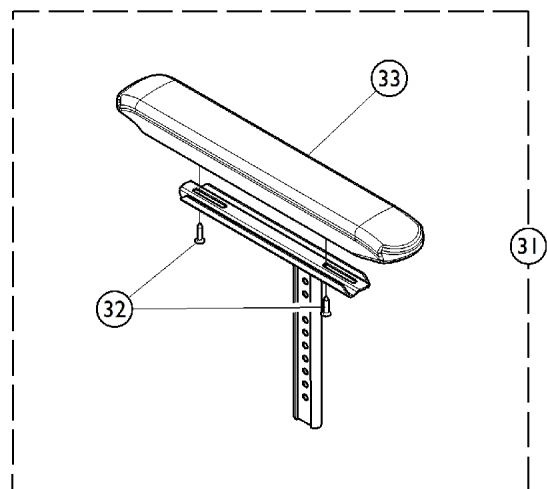
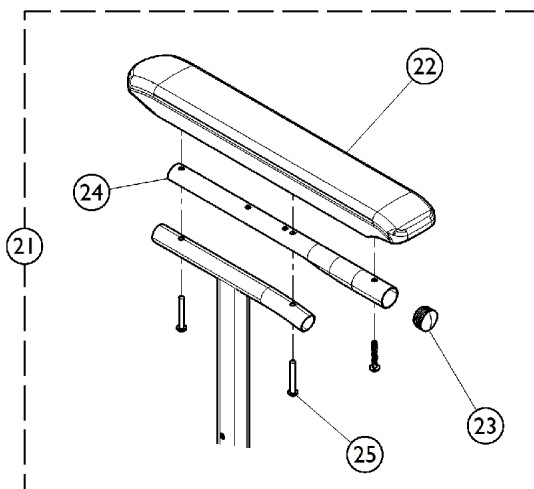
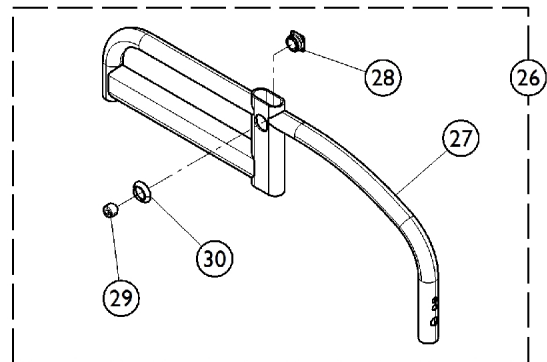
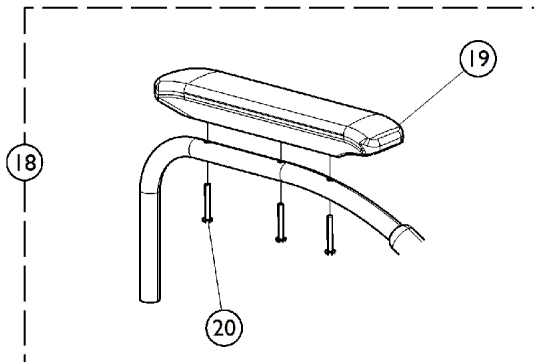
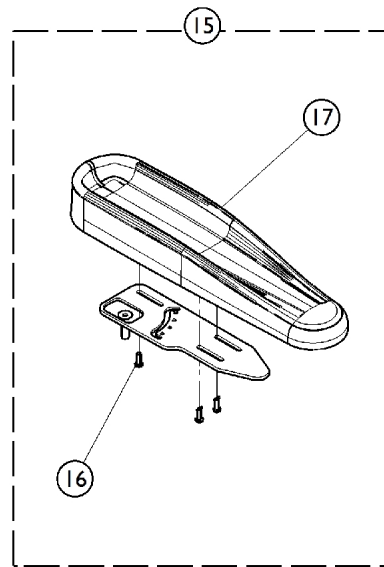
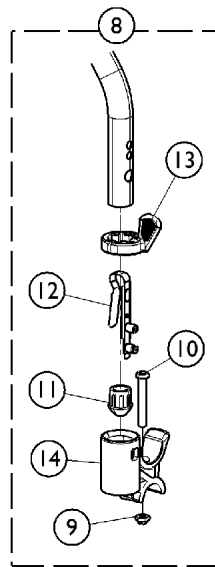


Armpad and Armrest Kits

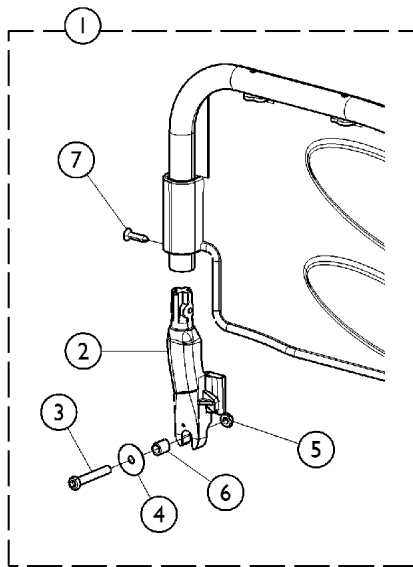

Armpad and Armrest Kits

Item Number	Note	Part Number	Description	Quantity Per		
				Package	Assembly	
1	A	1430423	Kit, Swingaway Armrest Support - Right	1	1	
2			Support, Armrest - Right	1	1	
3			NLA - Screw, Socket Button Head (M6 x 40mm)	1	1	
4			NLA - Locknut (M6 x 1)	1	1	
5			Washer (6.6 x 22 x 2)	1	1	
6			Bushing, Black	1	1	
7			Screw, Countersunk (M4.8)	1	1	
2	A	1425466	Support, Armrest - Right	1	1	
1		1431284	Kit, Swingaway Armrest Support - Left	1	1	
2			Support, Armrest - Left	1	1	
3			NLA - Screw, Socket Button Head (M6 x 40mm)	1	1	
4			NLA - Locknut (M6 x 1)	1	1	
5			Washer (6.6 x 22 x 2)	1	1	
6			Bushing, Black	1	1	
7	B		1182259	Screw, Countersunk (M4.8)	1	1
2		Support, Armrest - Left		1	1	
8		Kit, Socket Arm, Front w/ Hardware		1	1	
9		NLA - Locknut (M6 x 1)		1	1	
10		NLA - Screw, Socket Button Head (M6 x 40mm)		1	1	
11		Plug, Armrest		1	1	
12		Button, Spring		1	1	
13	C	1182271	Lever, Release	1	1	
14			Support, Armrest, Front	1	1	
15			Kit, Armpad, Hemi, Dartex	1	1	
16			Screw, Socket, Button Head (M6 x 16mm)	1	3	
17			Armpad Assembly, Hemi	1	1	
18			1162413	Kit, Armpad, Desk Length (Pair) (CNAD, TAD)	1	1
19				Arm Pad, Desk Length	1	2
20	Screw, Socket, Button Head (M5 x 35mm)	1		6		
21	E	1432595	Kit, Desk to Full Armpad w/ Tube and Hardware	1	1	
22			Button, Plug, Reflector (22mm)	1	1	
23			Arm Pad, Full Length	1	1	
24			Tube, Support, Armpad, Full Length	1	1	
25	E	8881185942	Screw, Socket, Button Head (M5 x 35mm)	1	4	
26			Kit, Armrest Tube w/ Hardware - Right	1	1	
27			Tube, Armrest, Color - Right	1	1	
28			Nut, Square, Armrest	1	1	
29			Screw, Socket Head (M12 x 12.5)	1	1	
30			Bushing, Armrest	1	1	
26	F	8881185943	Kit, Armrest Tube w/ Hardware - Left	1	1	
27			Tube, Armrest, Color - Left	1	1	
28			Screw, Socket Head (M12 x 12.5)	1	1	
29			Bushing, Armrest	1	1	
30	G	1182266	Nut, Square, Armrest	1	1	
31			Kit, Armpad, Full w/ Hardware	1	1	
32			Screw, Countersunk (M4.8)	1	2	
33			Pad, Arm, Full Length	1	1	

Armpad and Armrest Kits

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
31	G	1182263	Kit, Arm Pad, Desk Length w/ Hardware	1	1
32			Screw, Countersunk (M4.8)	1	2
33			Pad, Arm, Desk Length	1	1
NOTE:	A - Kit includes items 2 - 7. B - Kit includes items 9 - 14. C - Kit includes items 16 and 17. D - Kit includes items 19 and 20. E - Kit includes items 22 - 25. F - Kit includes items 27 - 30. G - Kit includes items 32 and 33.				