


# T Arm Component Kits

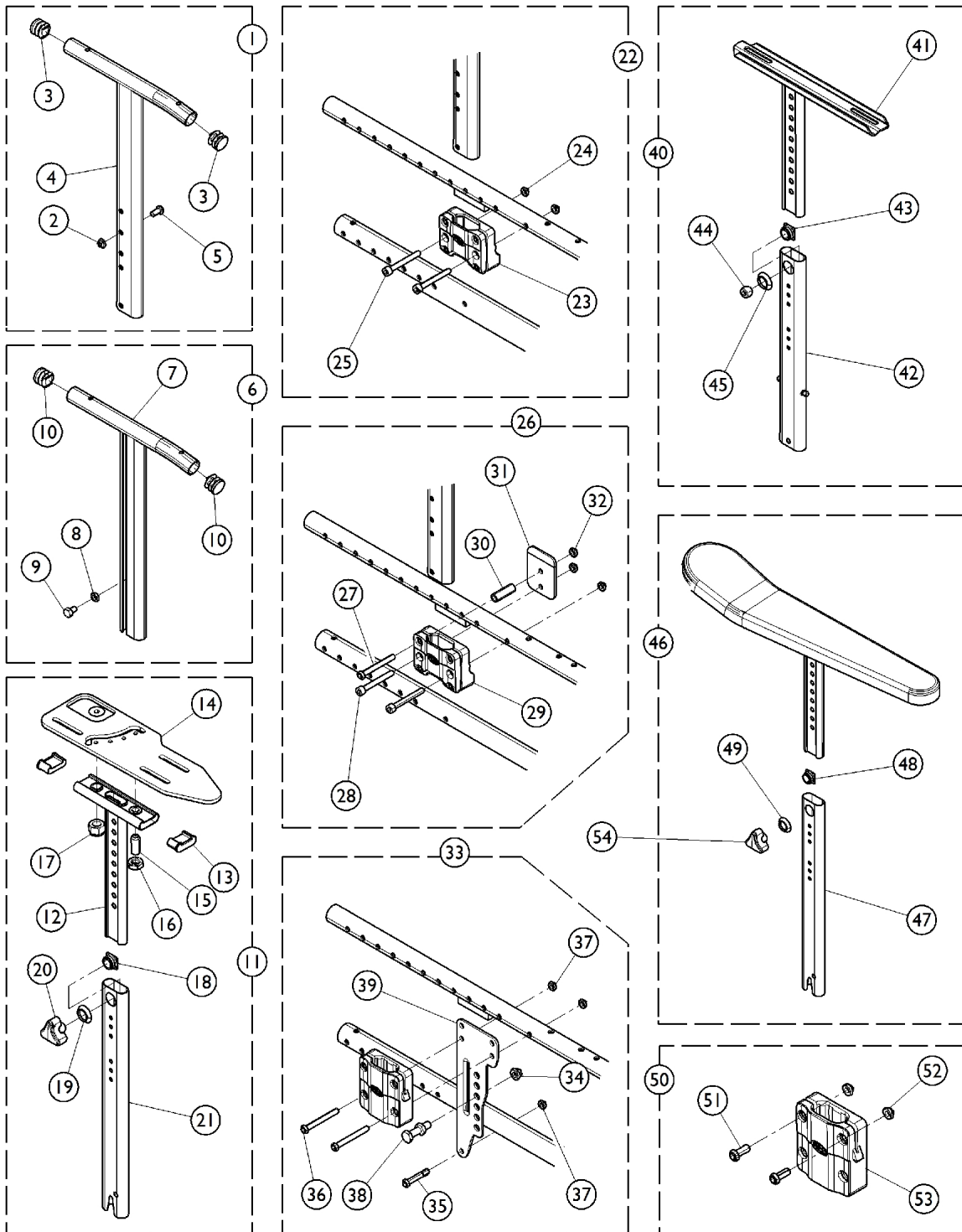

## T Arm Component Kits

| Item Number | Note | Part Number | Description                               | Quantity Per |          |
|-------------|------|-------------|-------------------------------------------|--------------|----------|
|             |      |             |                                           | Package      | Assembly |
| 1           | A    | 1182260     | Kit, T-Arm Post - Right                   | 1            | 1        |
| 2           |      |             | NLA - Locknut (M6 x 1)                    | 1            | 1        |
| 3           |      |             | Plug Button, Black (3/4")                 | 1            | 2        |
| 4           |      |             | Tube, Armrest - Right                     | 1            | 1        |
| 5           |      |             | Screw, Socket Button Head (M6 x 1 x 12mm) | 1            | 1        |
| 1           | A    | 1182261     | Kit, T-Arm Post - Left                    | 1            | 1        |
| 2           |      |             | NLA - Locknut (M6 x 1)                    | 1            | 1        |
| 3           |      |             | Plug Button, Black (3/4")                 | 1            | 2        |
| 4           |      |             | Tube, Armrest, Black - Left               | 1            | 1        |
| 5           |      |             | Screw, Socket Button Head (M6 x 1 x 12mm) | 1            | 1        |
| 6           | B    | 1182267     | Kit, T-Arm Post w/ Hardware - Left        | 1            | 1        |
| 7           |      |             | Plug Button, Black (3/4")                 | 1            | 2        |
| 8           |      |             | Nut (M6)                                  | 1            | 1        |
| 9           |      |             | Screw (M6 x 1 x 10mm)                     | 1            | 1        |
| 10          |      |             | Support, T-Arm                            | 1            | 1        |
| 6           | B    | 1182269     | Kit, T-Arm Post w/ Hardware - Right       | 1            | 1        |
| 7           |      |             | Support, T-Arm - Right                    | 1            | 1        |
| 8           |      |             | Nut (M6)                                  | 1            | 1        |
| 9           |      |             | Screw (M6 x 1 x 10mm)                     | 1            | 1        |
| 10          |      |             | Plug Button, Black (3/4")                 | 1            | 2        |
| 11          | C    | 1182272     | Kit, Hemi Arm Post w/ Hardware - Right    | 1            | 1        |
| 12          |      |             | Support, Hemi Arm - Right                 | 1            | 1        |
| 13          |      |             | Cap, Plastic, Arm Support                 | 1            | 2        |
| 14          |      |             | Support, Armpad, Hemi - Right             | 1            | 1        |
| 15          |      |             | Screw, Stop (M10 x 1.5 x 23mm)            | 1            | 1        |
| 16          |      |             | Nut, Low (M10 x 1.5)                      | 1            | 1        |
| 17          |      |             | Nut, Cap, Self Lock                       | 1            | 1        |
| 18          |      |             | Nut, Square, Armrest                      | 1            | 1        |
| 19          |      |             | Bushing, Armrest                          | 1            | 1        |
| 20          |      |             | Screw, Ball (7.5 mm)                      | 1            | 1        |
| 21          |      |             | Tube, Support, Comfort/Hemi Arm           | 1            | 1        |
| 11          | C    | 1182273     | Kit, Hemi Arm Post w/ Hardware - Left     | 1            | 1        |
| 12          |      |             | Support, Armpad, Hemi - Left              | 1            | 1        |
| 13          |      |             | Cap, Plastic, Arm Support                 | 1            | 2        |
| 14          |      |             | Support, Hemi Arm - Left                  | 1            | 1        |
| 15          |      |             | Screw, Stop (M10 x 1.5 x 23mm)            | 1            | 1        |
| 16          |      |             | Nut, Low (M10 x 1.5)                      | 1            | 1        |
| 17          |      |             | Nut, Cap, Self Lock                       | 1            | 1        |
| 18          |      |             | Nut, Square, Armrest                      | 1            | 1        |
| 19          |      |             | Bushing, Armrest                          | 1            | 1        |
| 20          |      |             | Screw, Ball (7.5 mm)                      | 1            | 1        |
| 21          |      |             | Tube, Support, Comfort/Hemi Arm           | 1            | 1        |
| 22          | D    | 1430421     | Kit, Attaching Hardware, T-Arm            | 1            | 1        |
| 23          |      |             | NLA - Support, Armrest                    | 1            | 1        |
| 24          |      |             | NLA - Locknut (M6 x 1)                    | 1            | 2        |
| 25          | E    | 1532575     | Screw, Socket Head (M6 x 50)              | 1            | 2        |
| 26          |      |             | Kit, Attaching Hardware, T-Arm            | 1            | 1        |

## T Arm Component Kits

| Item Number                                                                                                                                                                                                                                                                                                                   | Note                             | Part Number | Description                                        | Quantity Per |                                    |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------|-------------|----------------------------------------------------|--------------|------------------------------------|
|                                                                                                                                                                                                                                                                                                                               |                                  |             |                                                    | Package      | Assembly                           |
| 27                                                                                                                                                                                                                                                                                                                            | F                                | 1440953     | Screw, Socket Button Head (M6 x 53mm)              | 1            | 1                                  |
| 28                                                                                                                                                                                                                                                                                                                            |                                  |             | Screw, Socket Head (M6 x 50)                       | 1            | 2                                  |
| 29                                                                                                                                                                                                                                                                                                                            |                                  |             | NLA - Support, Armrest                             | 1            | 1                                  |
| 30                                                                                                                                                                                                                                                                                                                            |                                  |             | Spacer, Nylon                                      | 1            | 1                                  |
| 31                                                                                                                                                                                                                                                                                                                            |                                  |             | Support, T Arm, Heavy Duty                         | 1            | 1                                  |
| 32                                                                                                                                                                                                                                                                                                                            |                                  |             | NLA - Locknut (M6 x 1)                             | 1            | 3                                  |
| 33                                                                                                                                                                                                                                                                                                                            |                                  |             | Kit, Hardware, Attaching, Comfort Arm (Right/Left) | 1            | 1                                  |
| 34                                                                                                                                                                                                                                                                                                                            |                                  |             | NLA - Locknut (M8 x 1.25)                          | 1            | 1                                  |
| 35                                                                                                                                                                                                                                                                                                                            |                                  |             | Screw, Socket Head, Cap (M6 x 35mm)                | 1            | 1                                  |
| 36                                                                                                                                                                                                                                                                                                                            |                                  |             | Screw, Socket Button Head (M6 x 53mm)              | 1            | 2                                  |
| 37                                                                                                                                                                                                                                                                                                                            | G                                | 1182264     | NLA - Locknut (M6 x 1)                             | 1            | 3                                  |
| 38                                                                                                                                                                                                                                                                                                                            |                                  |             | Bolt, Double Headed                                | 1            | 1                                  |
| 39                                                                                                                                                                                                                                                                                                                            |                                  |             | Plate, Attaching                                   | 1            | 1                                  |
| 40                                                                                                                                                                                                                                                                                                                            |                                  |             | Kit, T-Arm Post w/ Hardware - Right                | 1            | 1                                  |
| 41                                                                                                                                                                                                                                                                                                                            |                                  |             | Arm Pad Mounting Assembly - Right                  | 1            | 1                                  |
| 42                                                                                                                                                                                                                                                                                                                            |                                  |             | Tube, Mounting, Adjustable                         | 1            | 1                                  |
| 43                                                                                                                                                                                                                                                                                                                            |                                  |             | Nut, Square, Armrest                               | 1            | 1                                  |
| 44                                                                                                                                                                                                                                                                                                                            |                                  |             | Screw, Socket Head (M12 x 12.5)                    | 1            | 1                                  |
| 45                                                                                                                                                                                                                                                                                                                            |                                  |             | Bushing, Armrest                                   | 1            | 1                                  |
| 40                                                                                                                                                                                                                                                                                                                            |                                  |             | G                                                  | 1182265      | Kit, T-Arm Post w/ Hardware - Left |
| 41                                                                                                                                                                                                                                                                                                                            | Arm Pad Mounting Assembly - Left | 1           |                                                    |              | 1                                  |
| 42                                                                                                                                                                                                                                                                                                                            | Tube, Mounting, Adjustable       | 1           |                                                    |              | 1                                  |
| 43                                                                                                                                                                                                                                                                                                                            | Nut, Square, Armrest             | 1           |                                                    |              | 1                                  |
| 44                                                                                                                                                                                                                                                                                                                            | Screw, Socket Head (M12 x 12.5)  | 1           |                                                    |              | 1                                  |
| 45                                                                                                                                                                                                                                                                                                                            | H                                | 1440895     | Bushing, Armrest                                   | 1            | 1                                  |
| 46                                                                                                                                                                                                                                                                                                                            |                                  |             | Kit, Comfort Arm Post w/ Hardware                  | 1            | 1                                  |
| 47                                                                                                                                                                                                                                                                                                                            |                                  |             | Tube, Support, Comfort/Hemi Arm                    | 1            | 1                                  |
| 48                                                                                                                                                                                                                                                                                                                            |                                  |             | Nut, Square, Armrest                               | 1            | 1                                  |
| 49                                                                                                                                                                                                                                                                                                                            | I                                | 1440952     | Bushing, Armrest                                   | 1            | 1                                  |
| 50                                                                                                                                                                                                                                                                                                                            |                                  |             | Kit, Central Arm Support, Comfort Arm              | 1            | 1                                  |
| 51                                                                                                                                                                                                                                                                                                                            |                                  |             | Screw, Socket, Button Head (M6 x 16mm)             | 1            | 2                                  |
| 52                                                                                                                                                                                                                                                                                                                            |                                  |             | NLA - Locknut (M6 x 1)                             | 1            | 2                                  |
| 53                                                                                                                                                                                                                                                                                                                            |                                  |             | Support, Armrest                                   | 1            | 1                                  |
| 54                                                                                                                                                                                                                                                                                                                            |                                  | 1434623     | Screw, Ball (7.5 mm)                               | 1            | 1                                  |
| NOTE: A - Includes items 2, 3, 4, and 5.<br>B - Includes items 7, 8, 9, and 10.<br>C - Includes items 12 - 21.<br>D - Includes items 23, 24, and 25.<br>E - Includes items 27 - 32.<br>F - Includes items 34 - 39.<br>G - Includes items 41 - 45.<br>H - Includes items 47, 48, and 49.<br>I - Includes items 51, 52, and 53. |                                  |             |                                                    |              |                                    |