
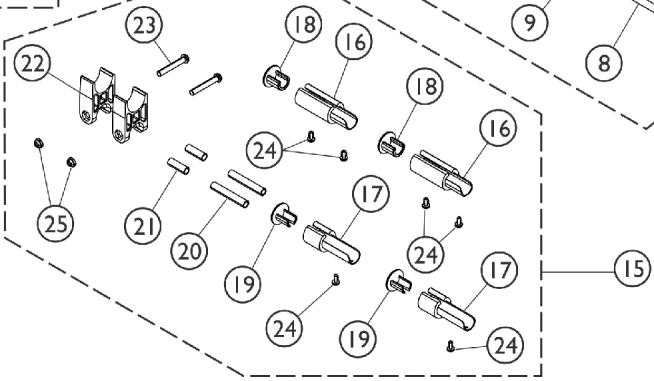
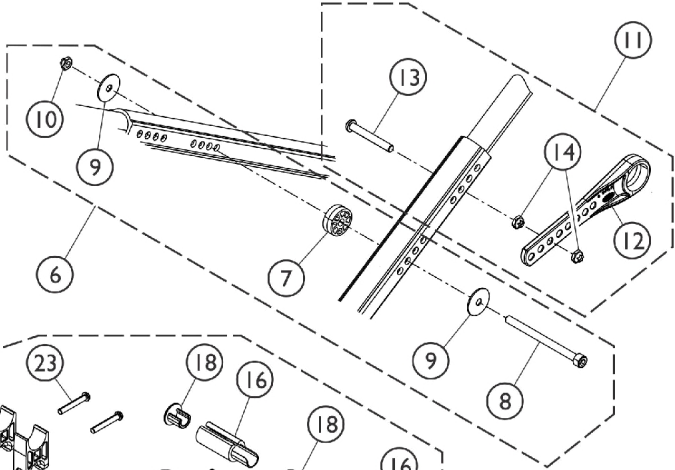
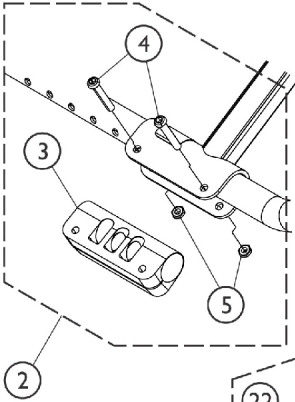


Crossbrace Assembly - Myon Junior (8 - 15" W)

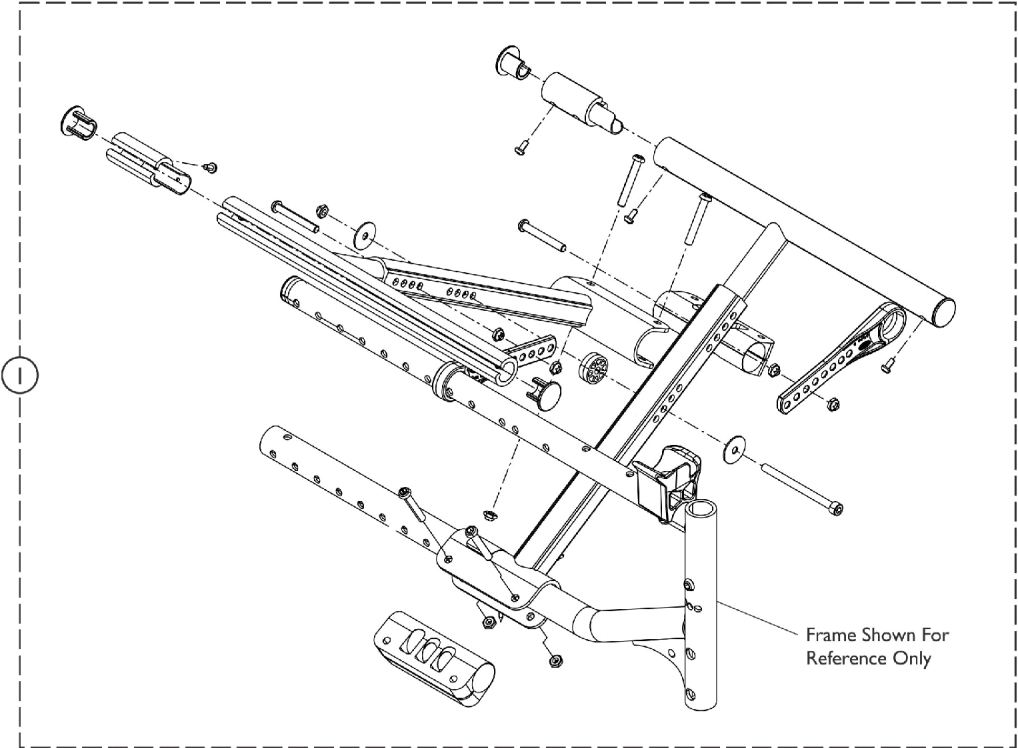

Crossbrace Assembly - Myon Junior (8 - 15" W)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Crossbrace Assembly - Myon Junior	1	1
2	A	1173765	Kit, Saddle Clamp w/ Hardware (Pair)	1	1
3			Clamp, Saddle, Crossbar	1	1
4			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
5			NLA - Locknut (M6 x 1)	1	2
6	B	1173777	Kit, Crossbrace w/ Hardware (Pair)	1	1
7			Bolt (M6 x 80mm)	1	1
8			Washer, Plastic	1	1
9			NLA - Locknut (M6 x 1)	1	1
10			Washer (6.6 x 22 x 2)	1	2
11	C	1173776	Kit, Pivot Link w/ Hardware (12 - 15" W)	1	1
12			Link, Pivot, Crossbrace (12-15" W)	1	2
13			Screw (M6 x 45mm)	1	2
14			NLA - Locknut (M6 x 1)	1	4
15	D	1172274	Kit, Seat Rail Extensions (16 - 18" D)	1	1
16			Extension, Seat Rail (2")	1	2
17			Extension, Seat Rail (1")	1	2
18			NLA - Plug Button	1	2
19			Cap, Extension, Seat Rail (1")	1	2
20			Rod, Upholstery (2")	1	2
21			Rod, Upholstery (1")	1	2
22			Guide, Seat	1	2
23			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
24			NLA - Screw, Phillips Pan Head (M3.5 x .6 x 9.5mm)	1	6
25			NLA - Locknut (M6 x 1)	1	2
			Instruction Sheet, Seat Rail Extensions	1	1
NOTE: A - Includes items 3, 4, and 5. B - Includes items 7 - 10. C - Includes items 12, 13, and 14. D - Kit Includes items 16 - 25 and Instruction Sheet.					

#1 Crossbrace Assembly - Myon Junior

Seat Width	SDR810 8-10" D	SDR1012 10-12" D	SDR1214 12-14" D	SDR1113 11-13" D	SDR1315 13-15" D
8" W	1511958	1511959	1513260	N/A	N/A
9" W	1511958	1511959	1513260	N/A	N/A
10" W	1511958	1511959	1513260	N/A	N/A
11" W	1511958	1511959	1513260	N/A	N/A
12" W	N/A	N/A	N/A	1512008	1512009
13" W	N/A	N/A	N/A	1512008	1512009
14" W	N/A	N/A	N/A	1512008	1512009
15" W	N/A	N/A	N/A	1512008	1512009