


# Crossbrace Assembly - Myon (15 - 20" W)

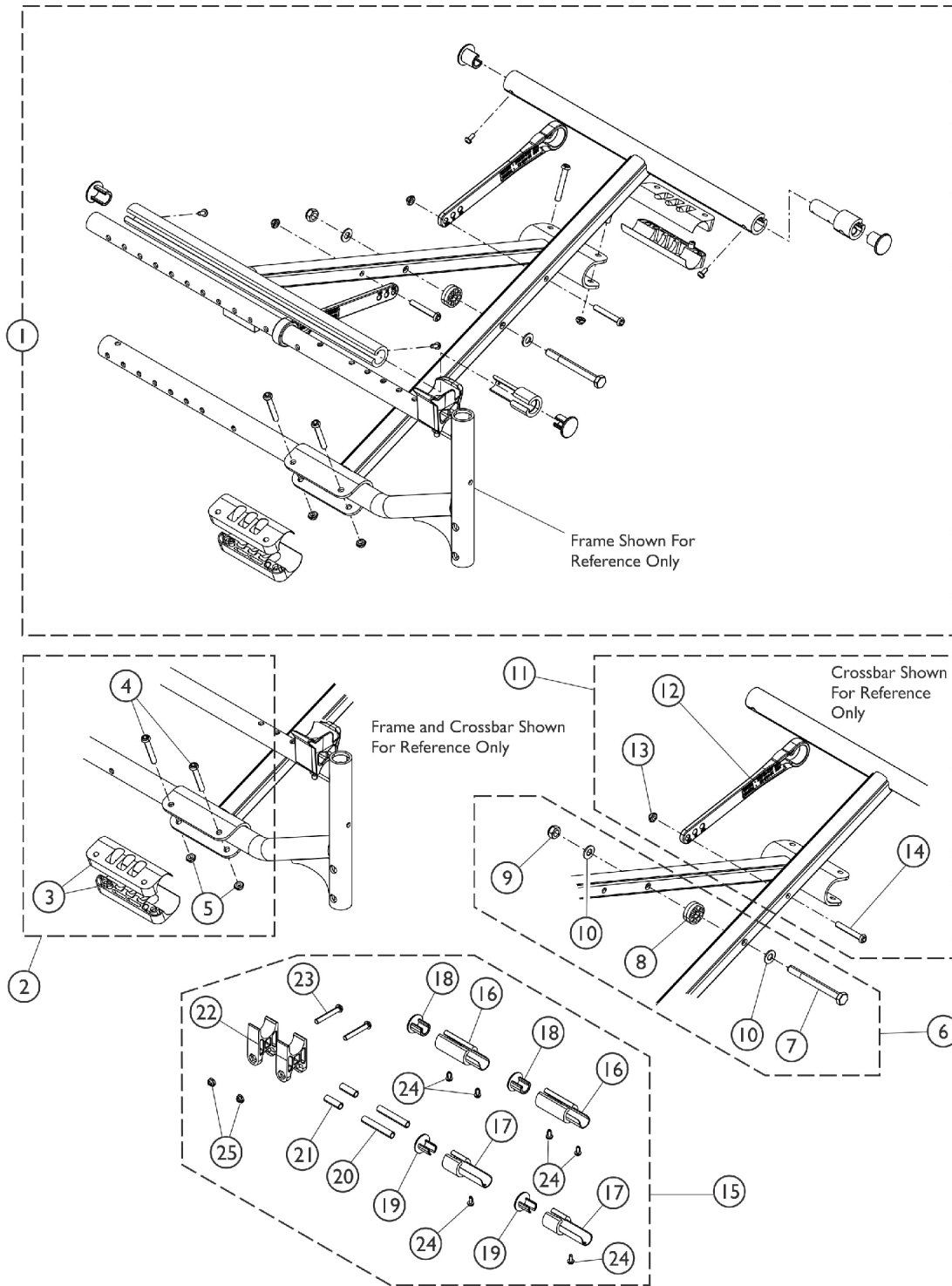

## Crossbrace Assembly - Myon (15 - 20" W)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Crossbrace Assembly w/ Hardware - Myon	1	1
2	A	1173765	Kit, Saddle Clamp w/ Hardware (Pair)	1	1
3			Clamp, Saddle, Crossbar	1	2
4			NLA - Screw, Socket Button Head (M6 x 40mm)	1	4
5			NLA - Locknut (M6 x 1)	1	4
6	B	1173771	NLA - Kit, Crossbrace Bolt w/ Hardware (Pair)	1	1
7			Washer, Plastic	1	2
8			NLA - Screw, Hex Head (M8 x 1.25 x 80mm)	1	2
9			NLA - Washer (M8 x 16mm x 1.5mm)	1	4
10			NLA - Locknut (M8 x 1.25)	1	2
11	C	1173779-NLA	NLA - Kit, Pivot Link w/ Hardware (15 - 17" W) (Pair)	1	1
12			NLA - Link, Pivot, Crossbrace (15 - 17" W)	1	2
13			NLA - Locknut (M6 x 1)	1	2
14			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
11	C	1173767-NLA	NLA - Kit, Pivot Link w/ Hardware (18 - 20" W) (Pair)	1	1
12			Link, Pivot, Crossbrace (18 - 20" W)	1	2
13			NLA - Locknut (M6 x 1)	1	2
14			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
15	D	1172274	Kit, Seat Rail Extensions (16 - 18" D)	1	1
16			Extension, Seat Rail (2")	1	2
17			Extension, Seat Rail (1")	1	2
18			NLA - Plug Button	1	2
19			Cap, Extension, Seat Rail (1")	1	2
20			Rod, Upholstery (2")	1	2
21			Rod, Upholstery (1")	1	2
22			Guide, Seat	1	2
23			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
24			NLA - Screw, Phillips Pan Head (M3.5 x .6 x 9.5mm)	1	6
25			NLA - Locknut (M6 x 1)	1	2
			Instruction Sheet, Seat Rail Extensions	1	1
15	E	1173066	Kit, Seat Rail Extensions (18 - 20" D)	1	1
16			Extension, Seat Rail (2")	1	2
17			Extension, Seat Rail (1")	1	2
18			NLA - Plug Button	1	2
19			Cap, Extension, Seat Rail (1")	1	2
20			Rod, Upholstery (20" D)	1	2
21			Rod, Upholstery (19" D)	1	2
24			NLA - Screw, Phillips Pan Head (M3.5 x .6 x 9.5mm)	1	6

NOTE: A - Includes items 3, 4, and 5.  
 B - Kit includes items 7 - 10.  
 C - Kit includes items 12, 13, and 14.  
 D - Kit includes items 16 - 25.  
 E - Kit includes items 16 - 21, and item 24.

# Crossbrace Assembly - Myon (15 - 20" W)

## #1 Crossbrace Assembly w/ Hardware - Myon

### Fixed Depth (FXDDPTH) and Adjustable Depth (ADJDPTH)

Seat Width	15" W	16" W	17" W	18" W	19" W	20" W
Fixed Depth (FXDDPTH)						
D16	1530032	1530033	1530034	1530035	1530036	1530037
D17	1168733	1168734	1168735	1168736	1168737	1168738
D18	1530038	1530052	1530053	1530054	1530055	1530056
D19	1172255	1172256	1172257	1172258	1172259	1172260
D20	1172261	1172262	1172263	1172264	1172265	1172266
Adjustable Depth (ADJDPTH)						
SDR1820	1530038	1530052	1530053	1530054	1530055	1530056