


Crossbrace Assembly - Myon Active HD (20 - 24" W)

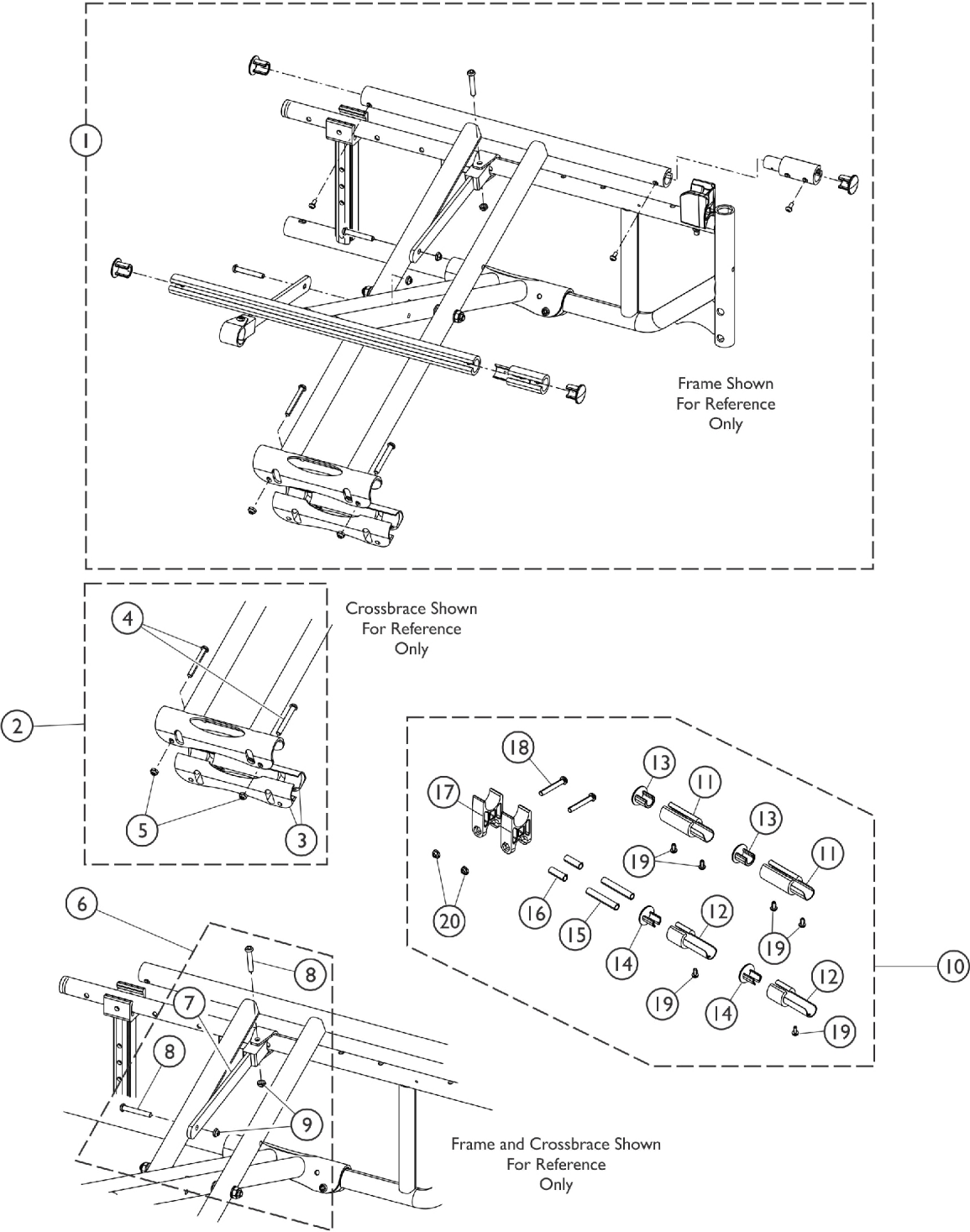

Crossbrace Assembly - Myon Active HD (20 - 24" W)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 Crossbrace Assembly w/ Hardware - Myon Active HD	1	1
2	A	1173766	Kit, Saddle Clamp w/ Hardware (Pair)	1	2
3			Clamp, Saddle, Crossbar	1	1
4			Screw (M6 x 45mm)	1	2
5			NLA - Locknut (M6 x 1)	1	2
6	D	1423416	Link, Pivot, Crossbrace (18 - 20" W)	1	2
6	B	1173769	Kit, Pivot Link w/ Hardware (22" W)	1	2
7			Link, Pivot, Crossbrace (22" W)	1	2
8			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
9			NLA - Locknut (M6 x 1)	1	2
6	B	1173770	Kit, Pivot Link w/ Hardware (24" W)	1	2
7			Link, Pivot, Crossbrace (24" W)	1	2
8			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
9			NLA - Locknut (M6 x 1)	1	2
10	C	1172274	Kit, Seat Rail Extensions (16 - 18" D)	1	1
11			Extension, Seat Rail (2")	1	2
12			Extension, Seat Rail (1")	1	2
13			NLA - Plug Button	1	2
14			Cap, Extension, Seat Rail (1")	1	2
15			Rod, Upholstery (2")	1	2
16			Rod, Upholstery (1")	1	2
17			Guide, Seat	1	2
18			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
19			NLA - Screw, Phillips Pan Head (M3.5 x .6 x 9.5mm)	1	6
20			NLA - Locknut (M6 x 1)	1	2
			Instruction Sheet, Seat Rail Extensions	1	1
NOTE: A - Kit includes items 3, 4, and 5. B - Kit includes items 7, 8, and 9. C - Kit includes items 11 - 20 and Instruction Sheet. D - NOTE: Kit #1173768 is No Longer Available. Part # 1423416 is the Crossbrace Link only!. Customer must use existing Attaching Hardware.					

#1 Crossbrace Assembly w/ Hardware - Myon Active HD

Seat Width	20" W	22" W	24" W
D16	1488618	1488620	N/A
D17	1169097	1169098	N/A
D18-SDR1618	1488619	1488621	1488622
D19	N/A	1169105	1169106
D20	1169112	1169113	1169114
SDR1820	N/A	1169113	1169114