
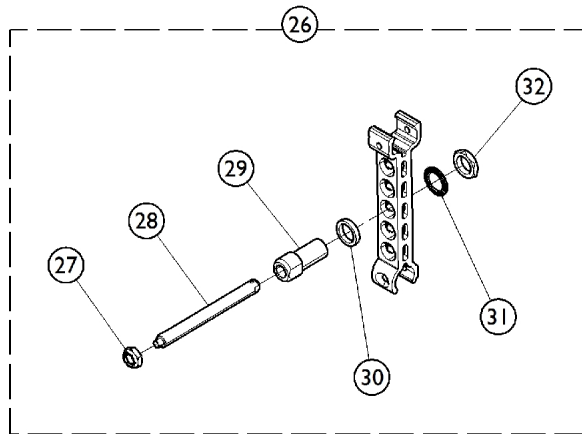
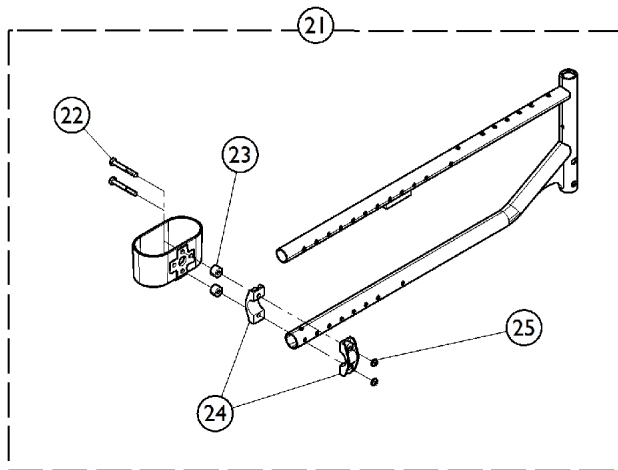
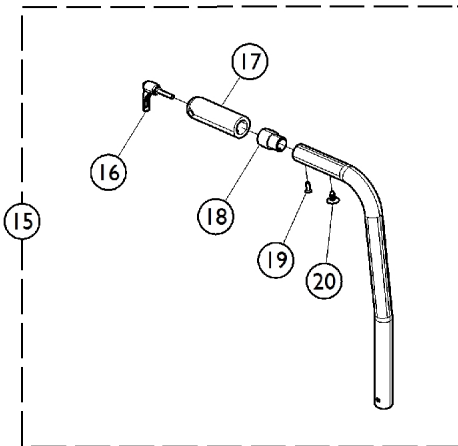
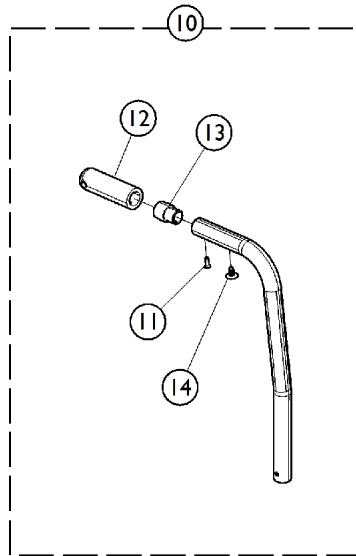
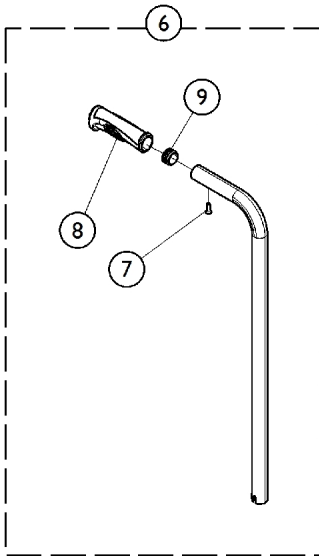


Accessories - Back Canes, Oxygen Holder, and Axle Hardware

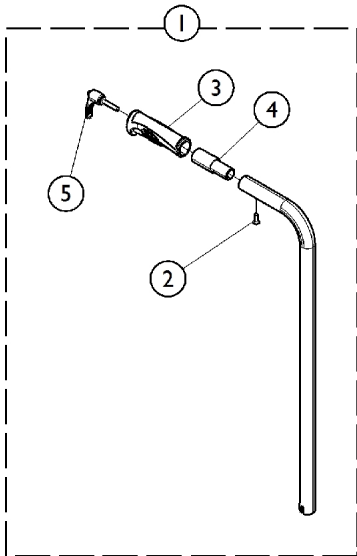

Accessories - Back Canes, Oxygen Holder, and Axle Hardware

Item Number	Note	Part Number	Description	Quantity Per		
				Package	Assembly	
1	A	1185945	Kit, Backbrace Lever w/ Hardware (Pair)	1	1	
2			Screw, Phillips Pan Head (M4.8 X 16mm)	1	2	
3			Grip, Cane (22 mm)	1	2	
4			Insert, Rigidizer	1	2	
5			Lever, Indexable	1	2	
5	B	1514326	Lever, Indexable	1	2	
6		1185946	Kit, Grip w/ Hardware, w/o Backbrace (Pair)	1	1	
7			Screw, Phillips Pan Head (M4.8 X 16mm)	1	2	
8			Grip, Cane (22 mm)	1	2	
9			Button, Plug (22mm)	1	2	
10	C	1185947	Kit, Grip w/ Hardware, w/o Backbrace (Pair)	1	1	
11			Screw, Phillips Pan Head (M4.8 X 16mm)	1	2	
12			Grip, Hand, Foam (4")	1	2	
13			Insert, Backbrace	1	2	
14			Panel Fastener, Snap In	1	2	
12	D	1167666	Grip, Hand, Foam (4")	1	2	
15		1185941	Kit, Back Brace Lever w/ Hardware (Pair)	1	1	
16			Lever, Indexable	1	2	
17			Grip, Hand, Foam (4")	1	2	
18			Insert, Backbrace	1	2	
19	E	1185940	Screw, Phillips Pan Head (M4.8 X 16mm)	1	2	
20			Panel Fastener, Snap In	1	2	
17			1167666	Grip, Hand, Foam (4")	1	1
21			Kit, Clamp, Cane Holder w/ Hardware	1	1	
22			Bolt, Hex Head (M6 x 45mm)	1	2	
23	F	1185939	Spacer, Cane Clamp Holder	1	2	
24			Clamp, Cane Holder	1	2	
25			NLA - Locknut (M6 x 1)	1	2	
26			Kit, Quick Release Axle w/ Hardware	1	1	
27			Locknut (1/2-20)	1	1	
28			Quick Release Axle	1	1	
29			Bushing, Axle	1	1	
30			Washer (20 x 27 x 5)	1	1	
31			Washer, Serrated (M20.3 x 30mm x 1.5mm)	1	1	
32			Nut (3/4-16)	1	1	
NOTE: A - Kit includes items 2 - 5. B - Kit includes items 7, 8, and 9. C - Kit includes items 11 - 14. D - Kit includes items 16 - 20. E - Kit includes items 22 - 25. F - Kit includes items 27 - 32.						