


2" Higher Seat-to-Floor Mounting Plate Assembly

Basic Tilt and Tilt Only

NOTE: Storm Arrow heavy duty bases come standard with 2" higher seat-to-floor assemblies. Non-heavy duty bases are done as a "special" when ordered "on-chair" with the original order.

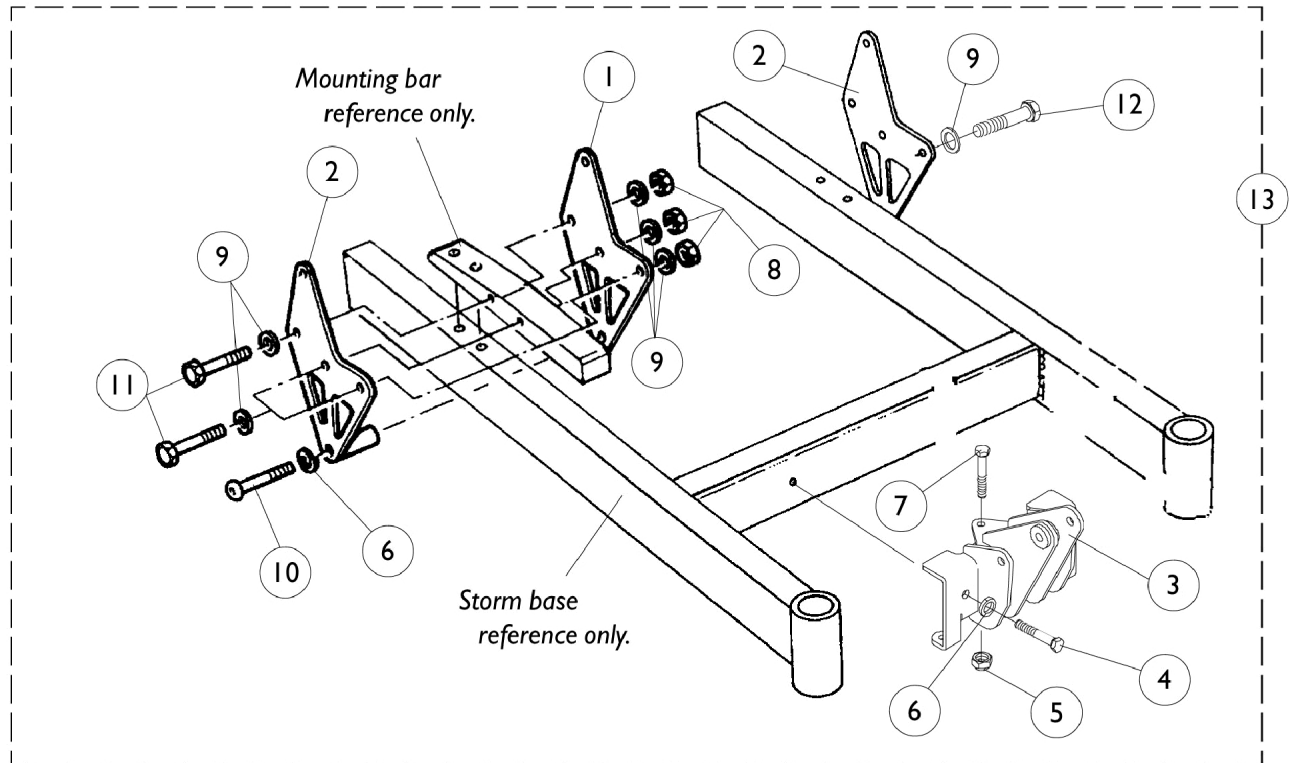

2" Higher Seat-to-Floor Mounting Plate Assembly

Basic Tilt and Tilt Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		0195302-NLA	NLA - Plate, Seat Frame Inner Mounting, Right (20" S.T.F.)	1	1
1		0195202-NLA	NLA - Plate, Seat Frame Inner Mounting, Left (20" S.T.F.)	1	1
2		0195502-NLA	NLA - Plate, Seat Frame Outer Mounting, Right (20" S.T.F.)	1	1
2		0195402-NLA	NLA - Plate, Seat Frame Outer Mounting, Left (20" S.T.F.)	1	1
3		0188202	Plate, Link Mounting (20" S.T.F.)	1	1
4		0139606-NLA	NLA - Screw, Hex Head (5/16-18 x 1")	1	2
5		0054906	Locknut, High Profile (5/16-18)	1	2
6		1072692-NLA	NLA - Lockwasher, Split (5/16")	1	4
7		0139806-NLA	NLA - Screw, Hex Head (5/16-18 x 3")	1	1
8		7930333	Locknut (1/4-20)	25	6
9		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	12
10		0140406-NLA	NLA - Screw, Socket Head, Cap (5/16-18 x 1-1/2")	1	2
11		0052306-NLA	NLA - Screw, Hex Head (1/4-20 x 2-1/4")	1	5
12		0052206	Screw, Hex Head (1/4-20 x 2-1/2")	1	1
13	A	1072683	Retrofit Assembly, Raise Seat-To-Floor 2"	1	1
NOTE: A - Includes items 1-12					