
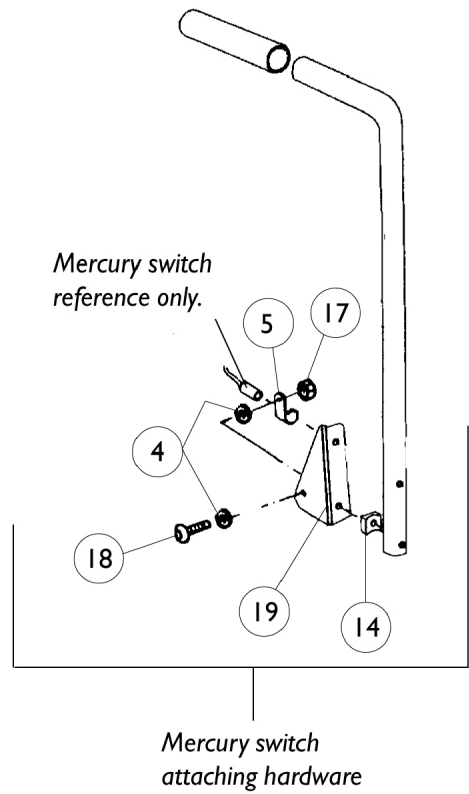


Back Cane Assembly

Basic Tilt and Tilt Only

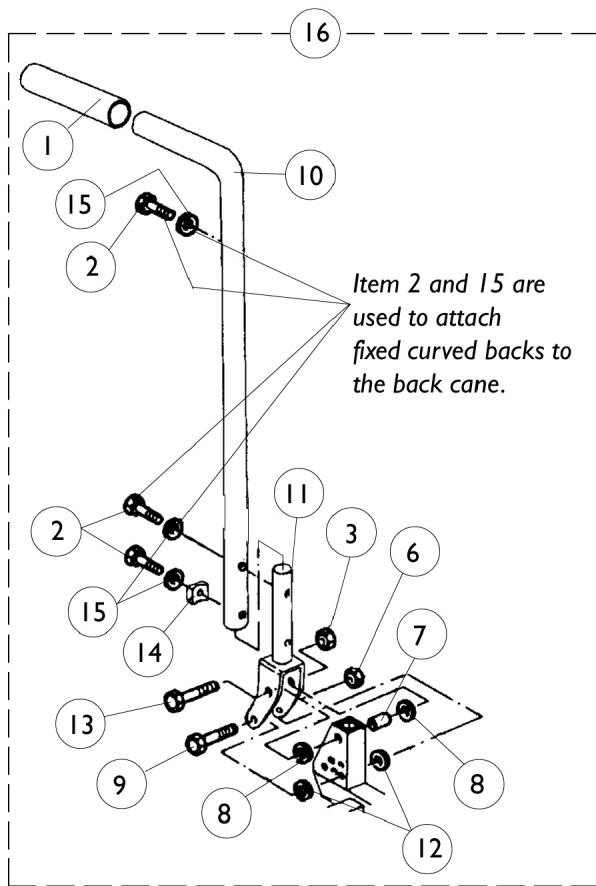

Back Cane Assembly Basic Tilt and Tilt Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		0014500-NLA	NLA - Grip, Hand	1	1
2		0053006-NLA	NLA - Screw, Hex Head (1/4-20 x 3/4")	1	3
3		7930333	Locknut (1/4-20)	25	1
4		0050006-NLA	NLA - Lockwasher, External Star (1/4")	2	2
5		0062800-NLA	NLA - Bracket, Cable/Wire Mounting, Black	1	1
6		0054906	Locknut, High Profile (5/16-18)	1	1
7		0060600	Bushing (7/16 x 5/16 x 1-1/4")	1	1
8		0055800-NLA	NLA - Washer, Nylon (7/16 x 3/4 x 3/32")	1	4
9		0055506-NLA	NLA - Screw, Button Head, Socket (5/16-18 x 2")	1	1
10		0309902-NLA	NLA - Back Cane Assembly	1	1
11		0116702-NLA	NLA - Post, Mounting	1	1
12		0054700-NLA	NLA - Washer, Nylon (5/16 x 1/2 x 1/16")	1	4
13		0052606-NLA	NLA - Screw, Hex Head (1/4-20 x 2")	1	1
14		0049824-NLA	NLA - Spacer, Coved (1/4 x 3/4 x 7/16")	1	1
15		0050506-NLA	NLA - Lockwasher, High Collar, Split (1/4")	1	3
16		1072788-NLA	NLA - Back Cane Assembly	1	1
17		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
18		0139306-NLA	NLA - Screw, Socket Button Head (#10-32 x 3/4")	1	1
19		0002902-NLA	NLA - Bracket, Mounting, Black - Left	1	1