



Transfer Bench - Large Blow Molded

Model 9670 - Without Commode

Model 9670U - Unassembled

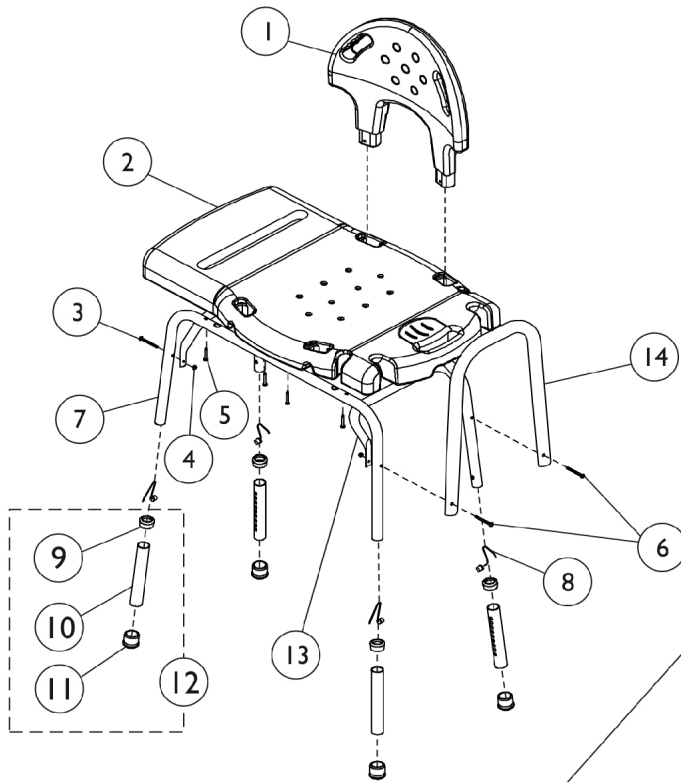

Model 9670C
With Commode

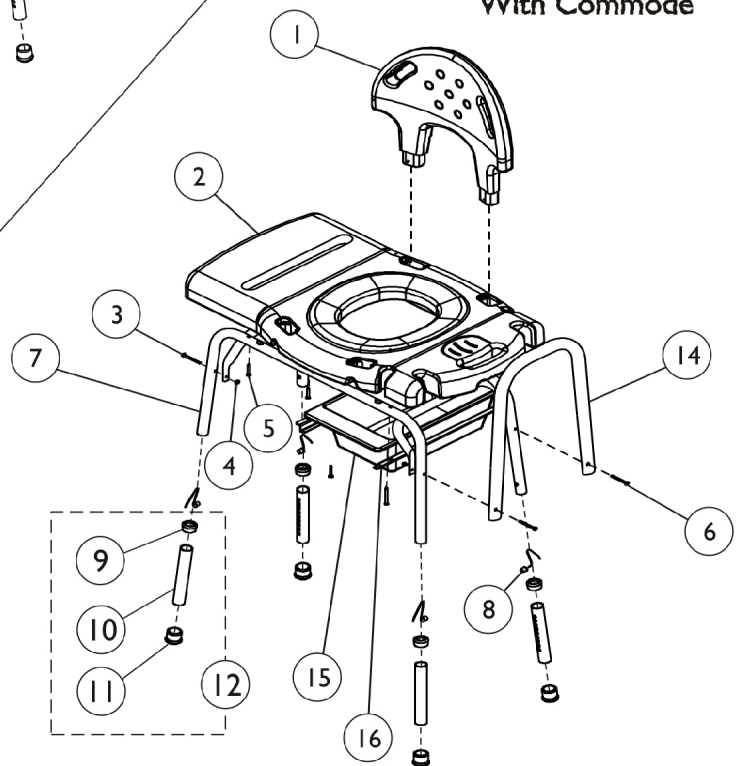

Transfer Bench - Large Blow Molded

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1060375-NLA	NLA - Back	1	1
2	A	1058356	Bench, Transfer with Commode	1	1
2	B	1060372	Bench, Transfer	1	1
	G,H	1069168	Kit, Hardware	1	1
3			Screw, Phillips Pan Head (1/4-20 x 1-3/8")	1	2
4			Locknut (1/4-20)	1	4
5			Screw, Slotted Hex Washer Head Tap (#10-16 x 1-1/2")	1	8
6			Screw, Phillips Pan Head (1/4-20 X 1-5/8")	1	2
7		1048897	Leg, Bench	1	2
8	C	1094945	Button, Snap	1	4
9		1063124	Anti-Rattle, Grey (1-1/8")	1	4
10	C	1022925	Tube, Leg Extension	1	4
11		1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
12	C,D	1058806-NLA	NLA - Leg Extension Assembly	1	1
13		1062820	Rail, End	1	2
14		1060379	Tube, Seat Arm	1	1
15	E	13245X000	Pail, Commode, White	1	1
15	F	1074676	Pan	1	1
16	E	1058357-NLA	NLA - Holder, Pan	1	2
16	F	1075045	Holder, Pan	1	2
	G,J	1068492-NLA	NLA - Back With Hardware	1	1
	G,I	1064507-NLA	NLA - Plug, End	1	2
NOTE: A - To be used with Model 9670C. B - To be used with Model 9670. C - It is recommended when replacing Leg Extension after 3/20/98, also replace Item 8 (Snap button). D - Includes Items 9-11. E - Before 4/20/98. F - After 4/19/98. G - Not Shown. H - Kit Includes Items 3-6. I - Before 5/15/97. J - Includes Items 1, 8, and 9.					