


# 1" Forward Rear Wheel Positioner

## 14" Deep Non-Reclining Chairs Only

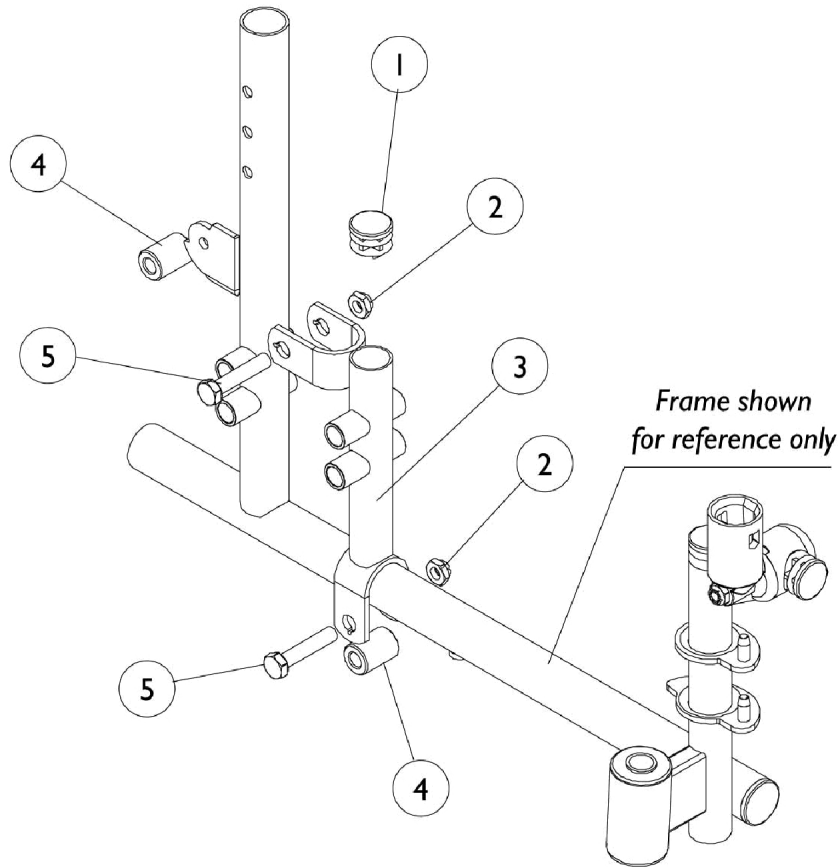

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027095	Plug Button, Black (7/8")	1	1
2		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	2
3	D	1101940-NLA	NLA - Bracket, Forward, Chrome - Right (1")	1	1
3	D	1101941-NLA	NLA - Bracket, Forward, Chrome - Left (1")	1	1
4		1102577	Spacer, Black (5/16 x 11/16 x 15/16")	1	2
5		11239X001	Screw, Hex Head (5/16-24 x 1-1/2")	1	2
	A,B,C	1101942	Hardware, Forward Option	2	1
NOTE: A - Not Shown B - Anti-Tippers must be used with 1" Forward Rear Wheel Positioner. See category titled, "Accessories." Wheel Lock Shoe Spacer is needed with rear wheels. See category titled, "Wheel Lock - Conventional Arms." C - Includes items 1, 2, 4, and 5. D - Item #3 is not available for color frame models.					