


# Headrest and Push Handles - For Recliner Chairs Only - Chrome Finish

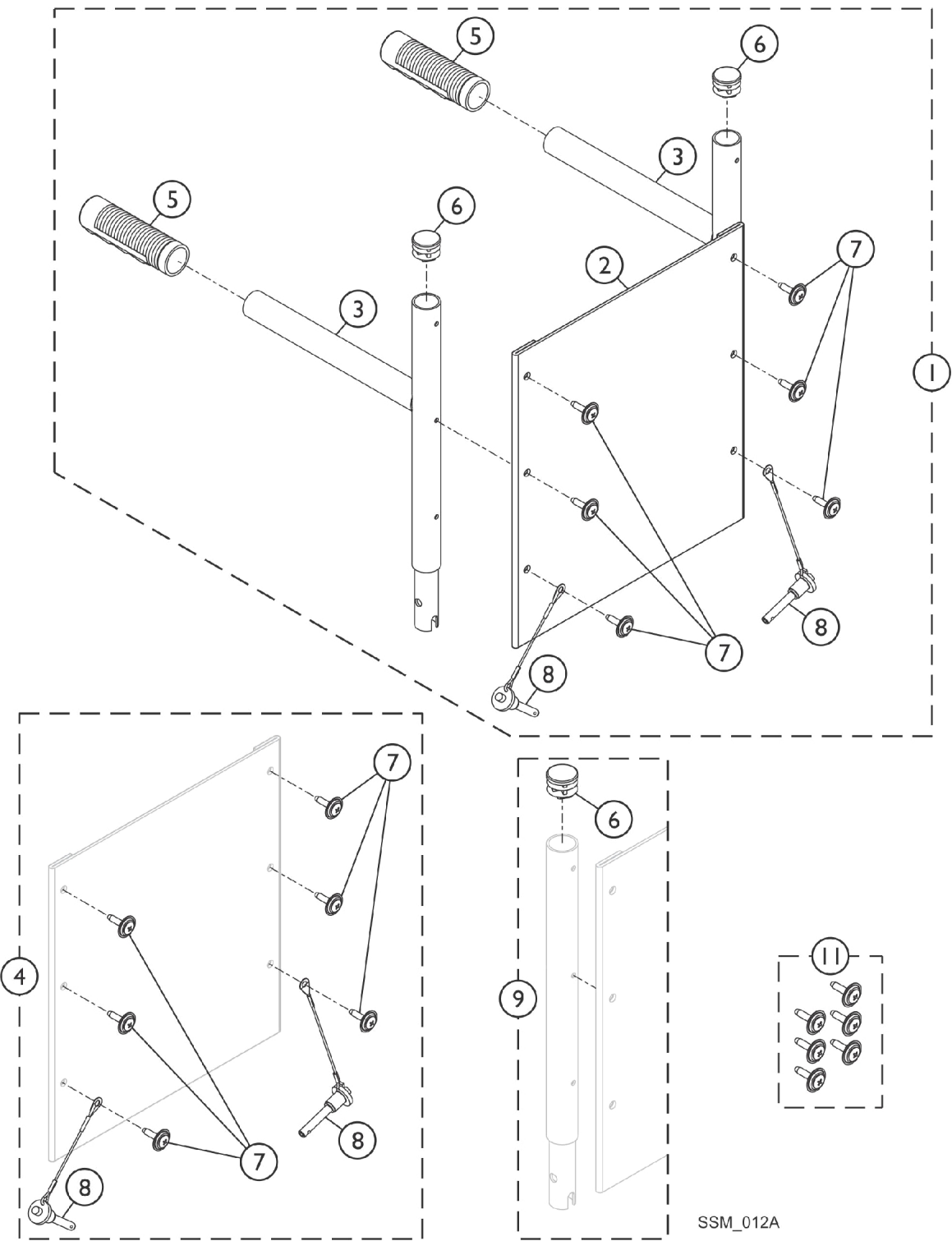

## Headrest and Push Handles - For Recliner Chairs Only - Chrome Finish

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1107053	Kit, Headrest with Handle Extension, Black (10" W)	1	1
1	A	1107052	Kit, Headrest with Handle Extension, Black (12" W)	1	1
1	A	1107051	Kit, Headrest with Handle Extension, Black (14" W)	1	1
2		1102536	Upholstery, Headrest (10" W)	1	1
2		1102535	Upholstery, Headrest (12" W)	1	1
2		1054627	Upholstery, Headrest, Recliner, Nylon, U240 (14" W)	1	1
2		1054628-NLA	NLA - Upholstery, Headrest, Recliner, Nylon, U240 (16" W)	1	1
3	B,F	1106218-NLA	NLA - Headrest Extension Tube Assembly with Handle, Chrome	1	2
4	C	1107050	Hardware, Attaching	1	1
5		1021261	Grip, Hand (6")	1	2
6		1027095	Plug Button, Black (7/8")	1	2
7		1076284	Screw w/ Washer (#10 x 7/8")	1	6
8		1045470	Pin, Quick Release with Lanyard	1	2
9	D	1054595-NLA	NLA - Headrest Extension Tube Assembly, Black	1	1
10	E	1102538	Cushion, Headrest (10" W)	1	1
10	E	1102537	Cushion, Headrest (12" W)	1	1
10	E	1054894	Cushion, Headrest (14" W)	1	1
10	E	1054896	Cushion, Headrest (16" W)	1	1
11		1030441	Package, Headrest Upholstery, Screws w/ Washers (#10 x 7/8")	6	1

NOTE: A - Includes items 2-4.  
B - Includes Headrest Extension Tube w/ Handle and items 5 and 6.  
C - Includes items 7 and 8.  
D - Includes Headrest Extension Tube and item 6.  
E - Not Shown  
F - No Longer Available. See Category titled "Accessories - Recliner Headrest & Push Handles - Color Finish" and order item #1 part number 8881182764 as a replacement.