
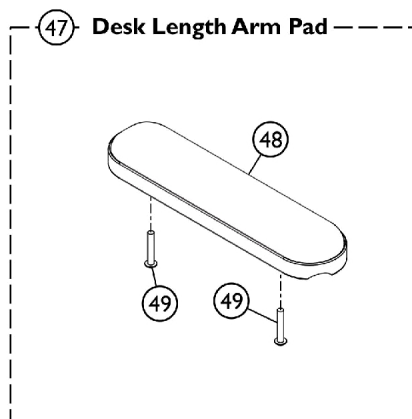
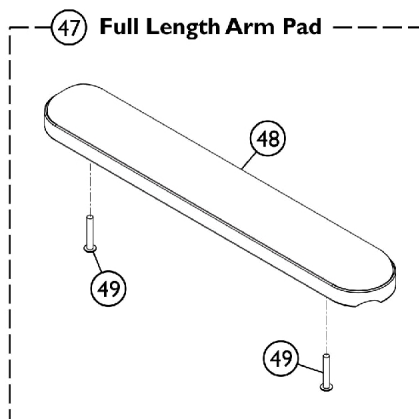
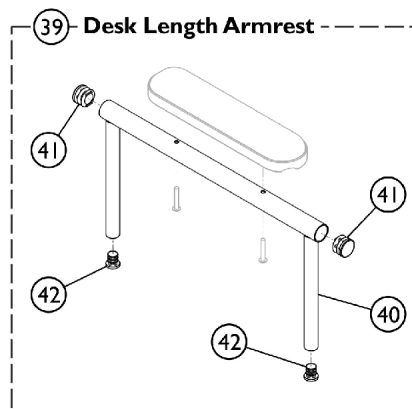
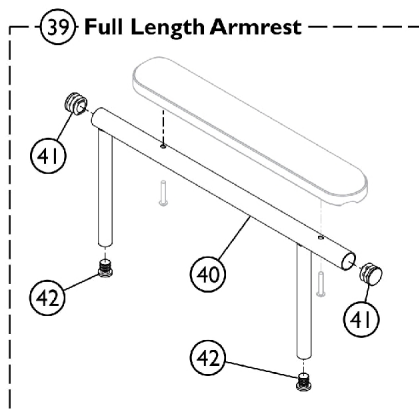
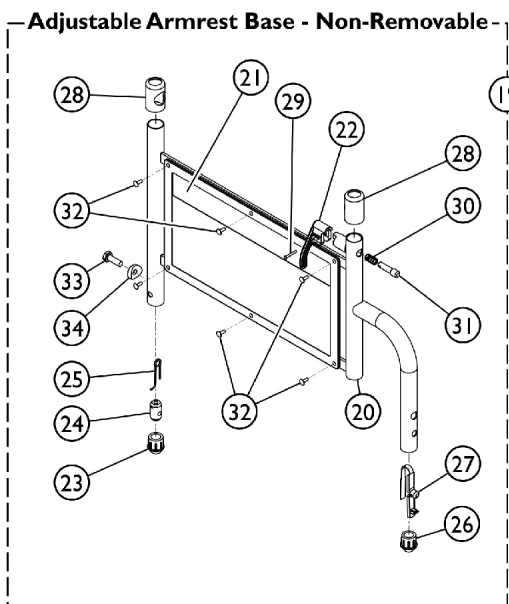


Conventional Adjustable Height Arms - Color Finish

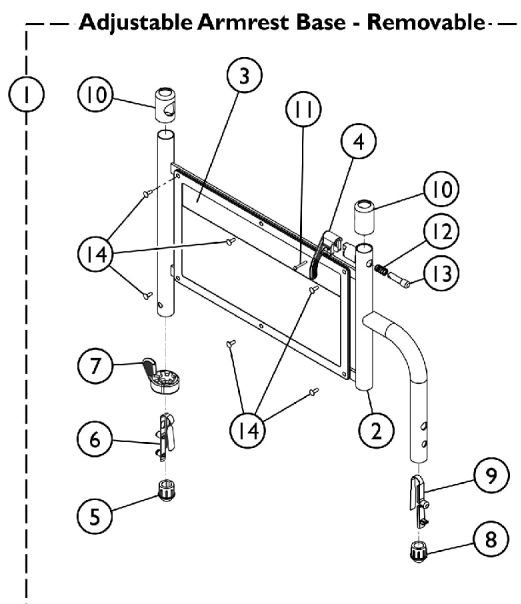

Conventional Adjustable Height Arms - Color Finish

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	E,A	8881181883	Kit, Armrest Base, Adjustable, Removable, Color - Right	1	1
2			Armrest, Adjustable Height, Color	1	1
3			Guard, Clothing	1	1
4			Handle, Lock, Black - Right	1	1
5			Plug Button, Arm Tube, Black (3/4")	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Plug, Armrest, Black (3/4") (18GA)	1	1
9			Spring, Torsion w/ Button Lock	1	1
10			Anti-Rattle	1	2
11			Rivet, Tubular	1	1
12			Spring, Arm Lock	1	1
13			Pin, Locking	1	1
14			Rivet	1	6
1	E,A	8881181884	Kit, Armrest Base, Adjustable, Removable, Color - Left	1	1
2			Armrest, Adjustable Height, Color	1	1
3			Guard, Clothing	1	1
4			Handle, Lock, Black - Left	1	1
5			Plug Button, Arm Tube, Black (3/4")	1	1
6			Spring, Torsion with Button Lock	1	1
7			Handle, Quad Release	1	1
8			Plug, Armrest, Black (3/4") (18GA)	1	1
9			Spring, Torsion w/ Button Lock	1	1
10			Anti-Rattle	1	2
11			Rivet, Tubular	1	1
12			Spring, Arm Lock	1	1
13			Pin, Locking	1	1
14			Rivet	1	6
19	E,B	8881181885	Kit, Armrest Base, Adjustable, Non-removable, Color - Right	1	1
20			Armrest, Adjustable Height, Color	1	1
21			Guard, Clothing	1	1
22			Handle, Lock, Black - Right	1	1
23			Plug, Armrest, Black (3/4") (20GA)	1	1
24			Insert, Threaded	1	1
25			Spring, Release	1	1
26			Plug, Armrest, Black (3/4") (18GA)	1	1
27			Spring, Torsion w/ Button Lock	1	1
28			Anti-Rattle	1	2
29			Rivet, Tubular	1	1
30			Spring, Arm Lock	1	1
31			Pin, Locking	1	1
32			Rivet	1	6
33			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1
34			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1

Conventional Adjustable Height Arms - Color Finish

Item Number	Note	Part Number	Description	Quantity Per		
				Package	Assembly	
19	E,B	8881181886	Kit, Armrest Base, Adjustable, Non-removable, Color - Left	1	1	
20			Armrest, Adjustable Height, Color	1	1	
21			Guard, Clothing	1	1	
22			Handle, Lock, Black - Left	1	1	
23			Plug, Armrest, Black (3/4") (20GA)	1	1	
24			Insert, Threaded	1	1	
25			Spring, Release	1	1	
26			Plug, Armrest, Black (3/4") (18GA)	1	1	
27			Spring, Torsion w/ Button Lock	1	1	
28			Anti-Rattle	1	2	
29			Rivet, Tubular	1	1	
30			Spring, Arm Lock	1	1	
31			Pin, Locking	1	1	
32			Rivet	1	6	
33			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	1	
34			Spacer, Coved (1/4 x 11/16 x 1/2")	1	1	
39		C,E	8881181879	Kit, Armrest w/ Plug Buttons, Desk Length, Color - Right/Left	1	1
40				Tube, Arm, Desk Length, Color - Right/Left	1	1
41				Plug Button, Black (7/8")	1	2
42			Plug Button, Black (5/8")	1	2	
39			8881181881	Kit, Armrest w/ Plug Buttons, Full Length, Color - Right/Left	1	1
40				Tube, Arm, Full Length, Color - Right/Left	1	1
41		Plug Button, Black (7/8")		1	2	
42		D	8881127509	Plug Button, Black (5/8")	1	2
47				Kit, Arm Pad w/Screws - Full Length	1	1
48				NLA - Pad, Arm, Full Length w/ Law Label	1	1
49		D	8881127510	Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	1
47				Kit, Arm Pad w/Screws - Desk Length	1	1
48				Arm Pad, Desk Length w/ Law Label	1	1
49				Package, Screw Phillips Pan Head (#10-32 x 1-1/8")	2	1
NOTE: A - Includes items 2-14. B - Includes items 20-34. C - Includes items 40-42. D - Includes items 48 and 49. Specify upholstery color when ordering, see category titled "Frame Finishes, Packaging, Literature and Upholstery". E - Specify 71P Silver Vein when ordering.						