



Trapeze

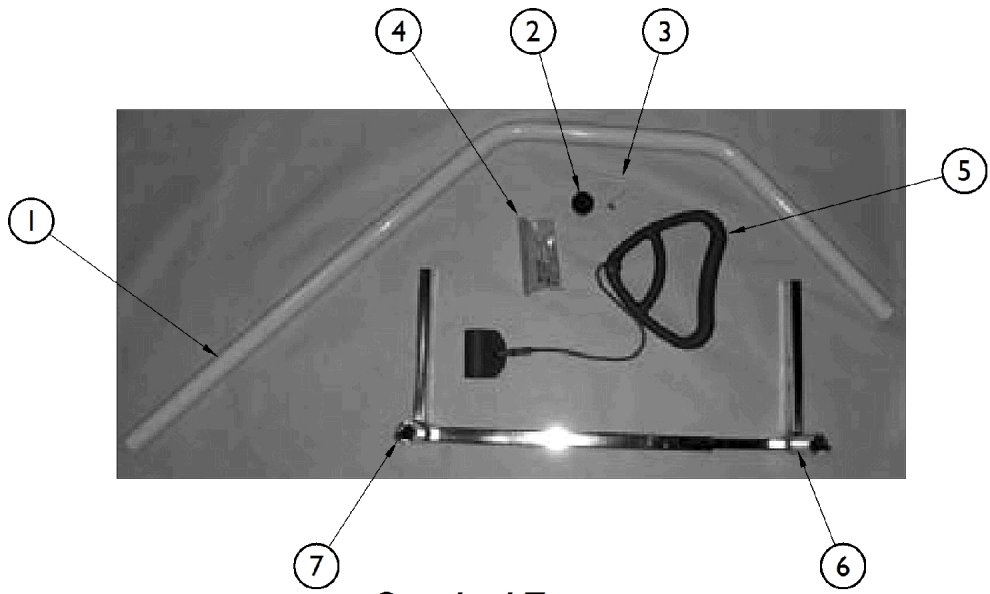

Standard Trapeze

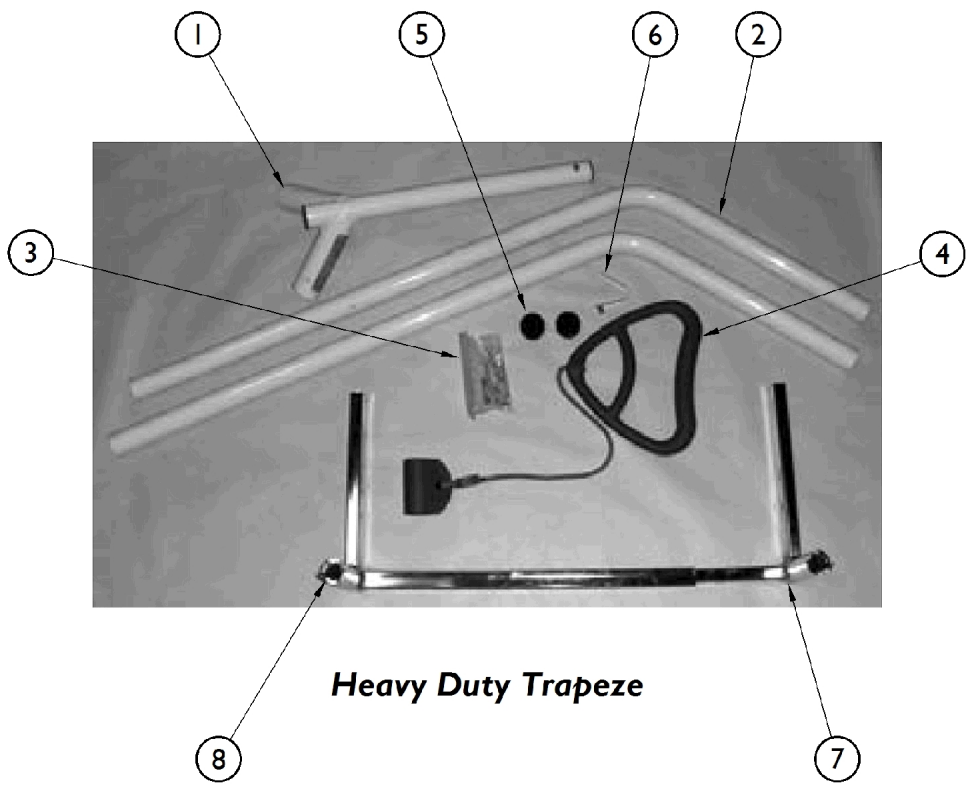

Heavy Duty Trapeze

Trapeze

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	IHCSTRP-NLA	NLA - Standard Trapeze	1	1
2		1153383-NLA	NLA - Standard Trapeze Bar	1	1
3		1146345-NLA	NLA - Plug Button, Round, Black (1-1/2")	1	1
4		1147186-NLA	NLA - Button, Snap	1	1
5	C	1153381-NLA	NLA - Kit, Hardware - Standard Trapeze	1	1
6			NLA - Lock Nut, Purple Insert (1/4-20)	2	1
7			Washer (3/4 x 1 x 11/64)	2	1
8		1147563-NLA	Handle, Trapeze Bar	1	1
9	D	1147610-NLA	NLA - Bracket, Mounting, Trapeze - Right	1	1
10	D	1147611-NLA	NLA - Bracket, Mounting, Trapeze - Left	1	1
11	E	1152551	Wireform, Trapeze - 80" (14-7/16")	1	1
12	B	IHCSTRPHD	Heavy Duty Trapeze	1	1
13		1153384	NLA - Boom, Heavy Duty Trapeze	1	1
14		1147211	NLA - Heavy Duty Trapeze Bar	1	2
15	C	1153382-NLA	NLA - Kit, Hardware - Heavy Duty Trapeze	1	1
16			Screw, Hex Head Cap (1/4-20 x 2-1/4")	2	1
17			NLA - Lock Nut, Purple Insert (1/4-20)	2	1
18		1147563	Handle, Trapeze Bar	1	1
19		1146345-NLA	NLA - Plug Button, Round, Black (1-1/2")	1	2
20		1147186-NLA	NLA - Button, Snap	1	2
21	D	1147610-NLA	NLA - Bracket, Mounting, Trapeze - Right	1	1
22	D	1147611-NLA	NLA - Bracket, Mounting, Trapeze - Left	1	1
23	E	1152551	Wireform, Trapeze - 80" (14-7/16")	1	1
<p>NOTE: A - Includes items 1 - 8. B - Includes items 1 - 9. C - Includes (2) Screws and (2) Lock Nuts (2). D - Order as a pair only. E - Not shown in pictures.</p>					