



Transfer Bench - Compact Blow Molded (Models 98070 and 990-1)

Model 98070

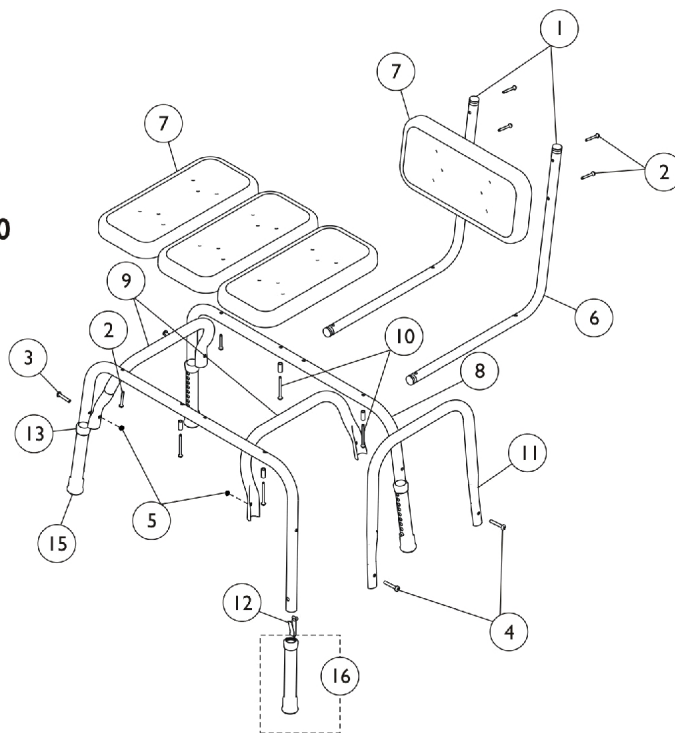

Model 990-1

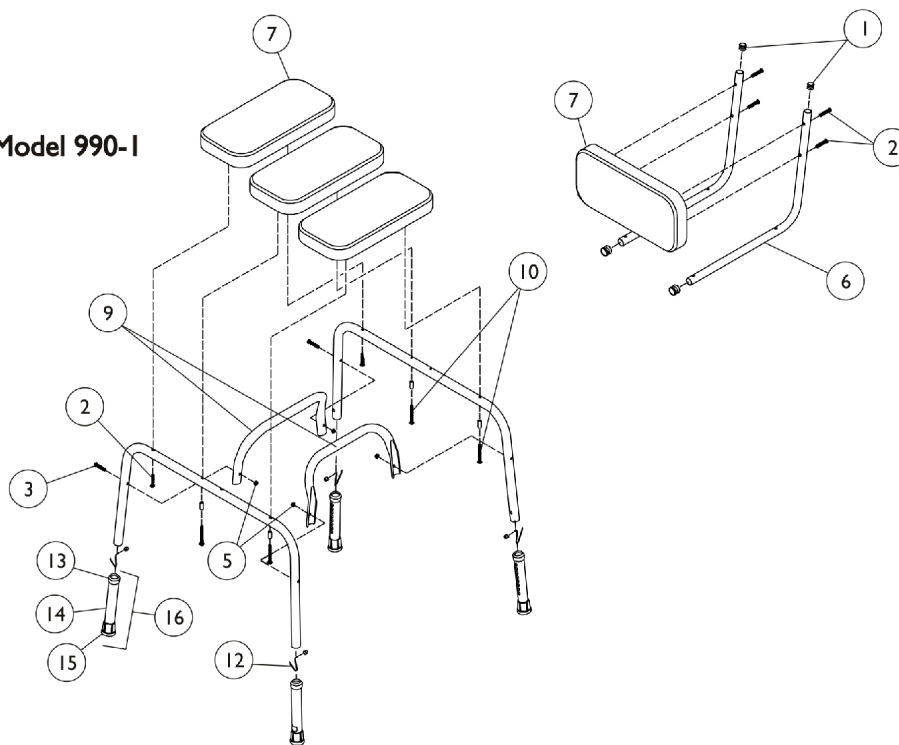

Transfer Bench - Compact Blow Molded (Models 98070 and 990-1)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D B,C,D, F	1027096	Plug Button, Grey (7/8")	1	4
		1075930	NLA - Kit, Hardware	1	1
2			Screw, Slotted Hex Washer Head Tap (#10-16 x 1-1/2")	1	6
3			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/4")	1	2
4			NLA - Screw, Phillips Pan Head (1/4-20 x 1-1/2")	1	2
5			Locknut (1/4-20)	1	4
6	D	1075036	Support, Back	1	2
7	D	1074987-NLA	NLA - Pad, Composite	1	4
7		1075183	Pad, Vinyl, Grey	1	4
8	D	1048897	Leg, Bench	1	2
9	D	1062820	Rail, End	1	2
10	D	1102582	Screw, Slot Hex Washer Head Tap (#10-16 x 2-3/8")	1	4
11		1060379	Tube, Seat Arm	1	1
12	D	1094945	Button, Snap	1	4
13	D	1063124	Anti-Rattle, Grey (1-1/8")	1	4
14	D	1022925	Tube, Leg Extension	1	4
15	D	1057797	Tip, Rubber, Light Grey (1-1/8")	1	4
16	A,D	1058806-NLA	NLA - Leg Extension Assembly	1	4
NOTE: A - Includes Items 12-14. B - Not Shown. C - Kit Includes Items 2-5. D - Used with 98070 and 990-1. F - Item 4 is not needed on Model 990-1.					