


Vent Tray Assembly (Before 5/1/96)

Basic Tilt and Tilt Only

38

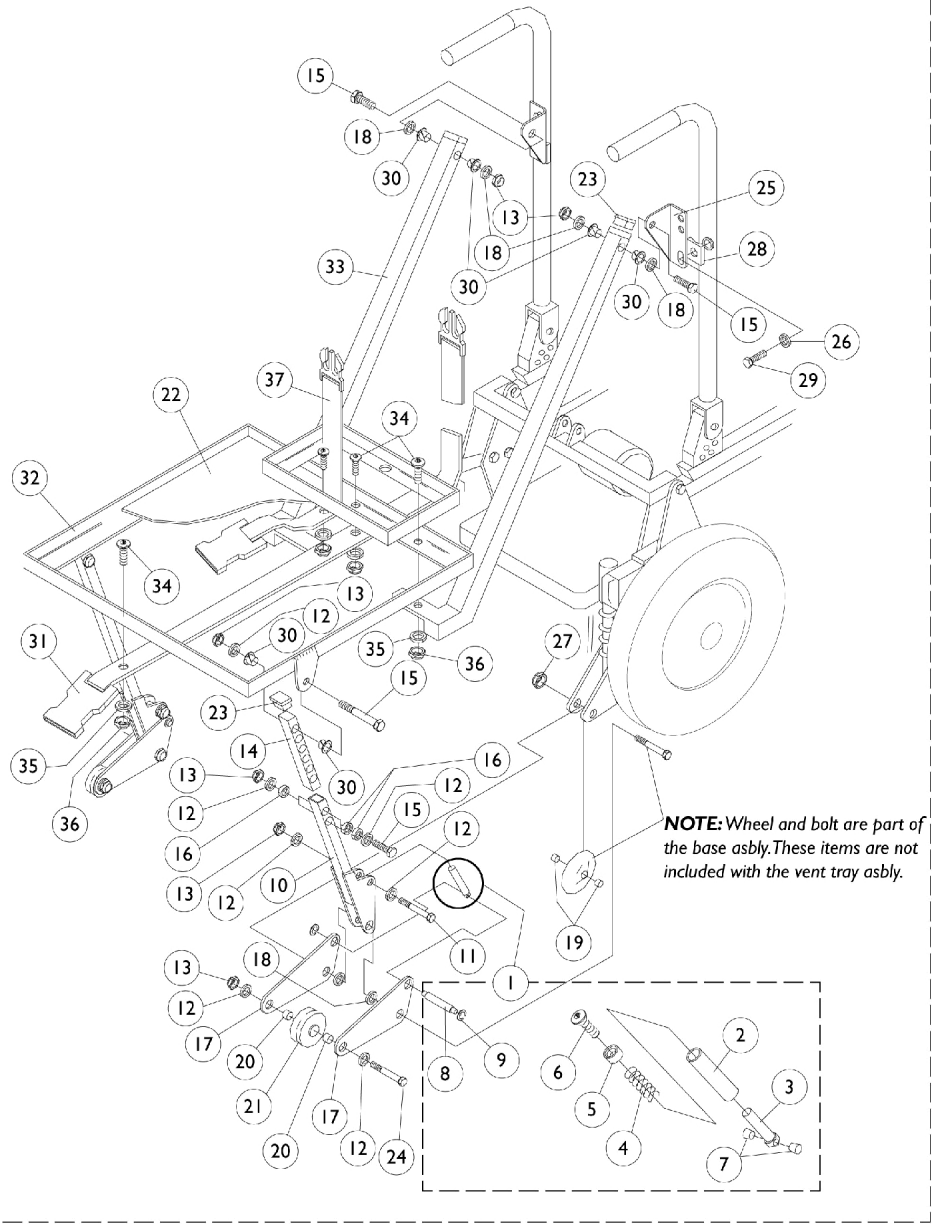

Vent Tray Assembly (Before 5/1/96) Basic Tilt and Tilt Only

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1072928-NLA	NLA - Mechanism, Anti-Tip Spring	1	2
2		0057202-NLA	NLA - Housing, Spring	1	2
3		0057305-NLA	NLA - Plunger, Spring	1	2
4		0159200-NLA	NLA - Spring, Compression	1	2
5		0057405-NLA	NLA - Retainer, Spring Plunger	1	2
6		1077248-NLA	NLA - Screw, Socket Head (1/4-20 x 1")	1	2
7		0184522-NLA	NLA - Spacer, Spring Plunger	1	4
8		0184422-NLA	NLA - Shaft, Spring Plunger Retaining	1	2
9		0226300-NLA	NLA - Ring, Retaining (1/4")	1	4
10		0181602-NLA	NLA - Leg, Vent Battery Tray, Lower, Black	1	2
11		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
12		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	18
13		7930333	Locknut (1/4-20)	25	12
14		0028302-NLA	NLA - Tube, Lower Support	1	2
15		0051006-NLA	NLA - Screw, Hex Head Full Thread (1/4-20 x 1-1/2")	1	8
16		0050506	Lockwasher, High Collar, Split (1/4")	1	12
17		0181502-NLA	NLA - Plate, Anti-Tip Wheel	1	4
18		0050100-NLA	NLA - Washer, Nylon Flat (1/4 x 1/2 x 1/16")	1	8
19		0185500-NLA	NLA - Spacer, Anti-Tip	1	4
20		0186000-NLA	NLA - Spacer, Wheel	1	4
21		1055684-NLA	NLA - Wheel, Anti-Tip 2"	1	2
22		0282619-NLA	NLA - Pan, Vent Tray, Black	1	1
23		0062200-NLA	NLA - Plug Button, Square (3/4")	1	4
24		0052306-NLA	NLA - Screw, Hex Head (1/4-20 x 2-1/4")	1	2
25		0003002-NLA	NLA - Bracket, Mounting, Black - Right	1	1
25		0002902-NLA	NLA - Bracket, Mounting, Black - Left	1	1
26		1007453	Lockwasher, Split (1/4")	1	4
27		0055006-NLA	NLA - Locknut, Low Profile (5/16-18)	1	2
28		0049824-NLA	NLA - Spacer, Coved (1/4 x 3/4 x 7/16")	1	2
29		0051706-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	4
30		0059800-NLA	NLA - Bearing, Nylon	1	8
31		0004900-NLA	NLA - Strap, Battery (65" Long)	1	1
32		0181402-NLA	NLA - Tray, Vent/ Battery	1	1
33		0181802-NLA	NLA - Frame, Vent Hanger - Right	1	1
33		0181702-NLA	NLA - Frame, Vent Hanger - Left	1	1
34		1077249-NLA	NLA - Screw, Socket Head (#10-32 x 3/4")	1	8
35		0224806-NLA	NLA - Washer (#10)	1	8
36		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	8
37		0004800-NLA	NLA - Strap, Battery (55" Long)	1	1
38	A	1077273-NLA	NLA - Vent/Battery Tray Assembly	1	1

NOTE: A - See vent tray assembly dated after 4/30/96.