


Mechanical Shear Reduction Linkage

M.S.R.

Basic Recline 4XP

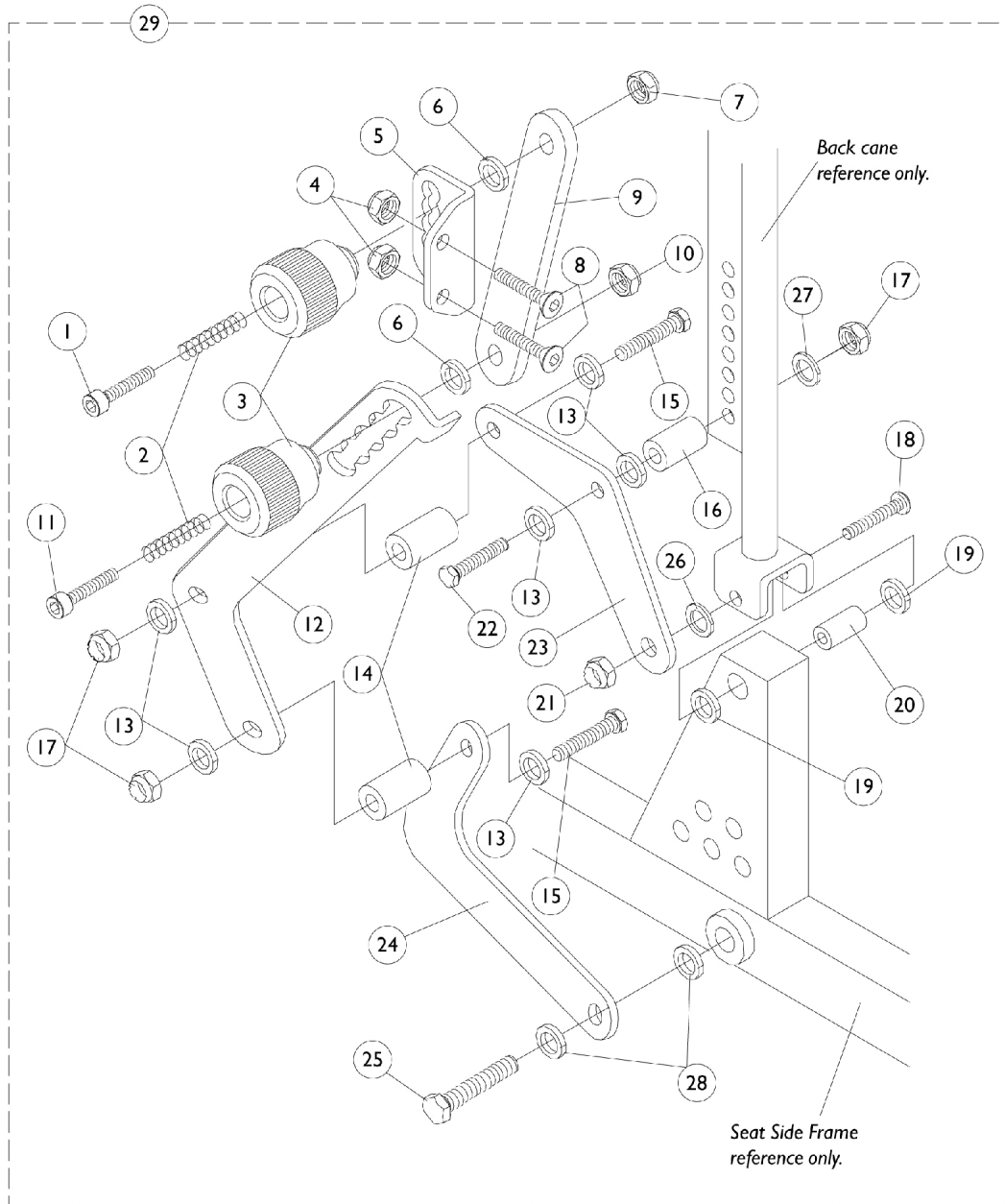

Mechanical Shear Reduction Linkage

M.S.R.

Basic Recline 4XP

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		0258406-NLA	NLA - Screw, Socket Head, Cap (#10-32 x 1-1/4")	1	1
2		0146500-NLA	NLA - Spring, Compression (1/32 WT x 15/16" L)	1	2
3		0146622-NLA	NLA - Knob, MSR Adjustment	1	2
4		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	2
5		0149624-NLA	NLA - Bracket, MSR Back Pan, Black	1	1
6		0146700-NLA	NLA - Washer, Nylon (3/16 x 5/8 x 1/16")	1	2
7		7930332-NLA	NLA - Locknut, High Profile (#10-32)	25	1
8		0195606-NLA	NLA - Screw, Socket, Cap, Flat Head (1/4-20 x 3/4")	1	2
9		0148806-NLA	NLA - Link, Upper Adjustment, Black	1	1
10		317601-NLA	NLA - Locknut, Low Profile (#10-32)	1	1
11		0149506-NLA	NLA - Screw, Socket Head, Hex (#10-32 x 1")	1	1
12		0175524-NLA	NLA - Link, Lower Adjustment, Black - Right	1	1
12		0175424-NLA	NLA - Link, Lower Adjustment, Black - Left	1	1
13		0050100-NLA	NLA - Washer, Nylon Flat (1/4 x 1/2 x 1/16")	1	6
14		0216901-NLA	NLA - Bushing, Upper/ Lower	1	2
15		0052606-NLA	NLA - Screw, Hex Head (1/4-20 x 2")	1	2
16		0146422-NLA	NLA - Spacer, Leg Link	1	1
17		7930333	Locknut (1/4-20)	25	3
18		0055506-NLA	NLA - Screw, Button Head, Socket (5/16-18 x 2")	1	1
19		0055800-NLA	NLA - Washer, Nylon (7/16 x 3/4 x 3/32")	1	2
20		0060300-NLA	NLA - Bushing, Bronze (5/16 x 7/16 x 1-1/4")	1	1
21		0055006-NLA	NLA - Locknut, Low Profile (5/16-18)	1	1
22		0051006-NLA	NLA - Screw, Hex Head Full Thread (1/4-20 x 1-1/2")	1	1
23		0217224-NLA	NLA - Link, MSR Back Post Lower, Black	1	1
24		0165424-NLA	NLA - Link, MSR Lower, Black	1	1
25		0139606-NLA	NLA - Screw, Hex Head (5/16-18 x 1")	1	1
26		0054806-NLA	NLA - Washer, Flat (11/32 x 11/16 x 1/16")	1	1
27		0050406	Washer, Flat (1/4 x 5/8 x 1/16")	1	1
28		0054700-NLA	NLA - Washer, Nylon (5/16 x 1/2 x 1/16")	1	2
29		1077512	Mechanical Shear Reduction Linkage Assembly - Right	1	1
29		Mechanical	Mechanical Shear Reduction Linkage Assembly - Left	1	1