
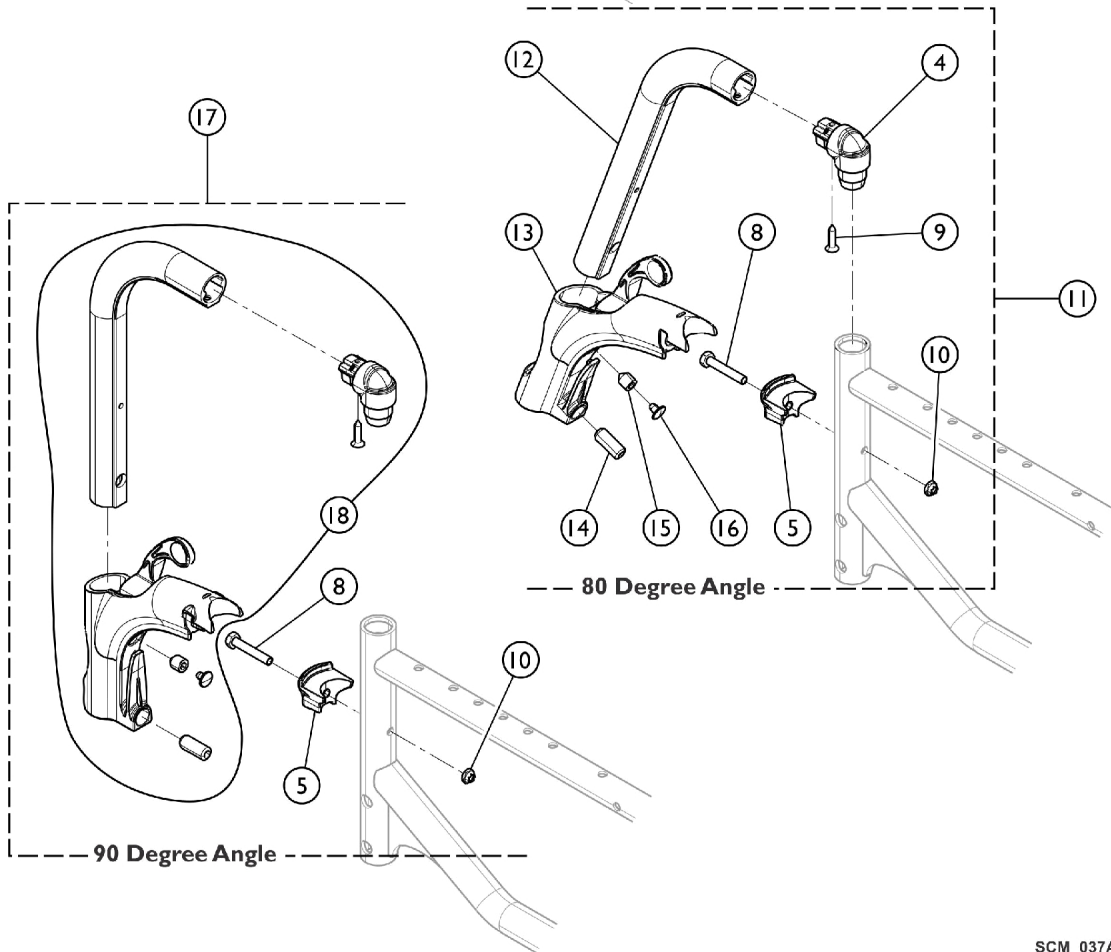


# Front Rigging - Swing Out Hangers 70, 80, and 90 Degree

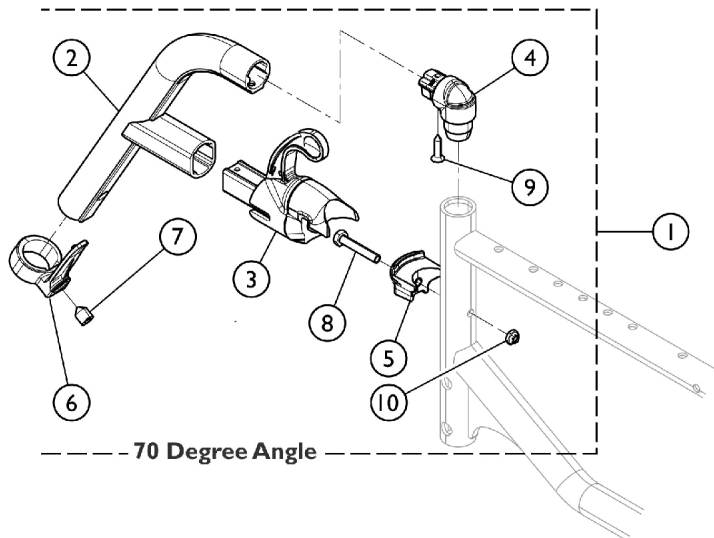

## Front Rigging - Swing Out Hangers 70, 80, and 90 Degree

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881173077	Hanger, Front Rigging, 70 Degree, Color	1	1
2			Tube, Rigging, 70 Degree, Color	1	1
3			Housing, Locking Pin	1	1
4			Support, Hanger, Footrest	1	1
5			Bracket, Front Rigging	1	1
6			Calf Support Strap	1	1
7			Screw, Socket Head Cone PT Set (M10 x 1.5 x 16)	1	1
8			Screw, Socket Head, Cap (M6 x 35mm)	1	1
9			Screw, Countersunk (M4.8)	1	1
10			NLA - Locknut (M6 x 1)	1	1
11	B	8881169050	Hanger, Front Rigging, 80 Degree, Color	1	1
4			Support, Hanger, Footrest	1	1
5			Bracket, Front Rigging	1	1
8			Screw, Socket Head, Cap (M6 x 35mm)	1	1
9			Screw, Countersunk (M4.8)	1	1
10			NLA - Locknut (M6 x 1)	1	1
12			Tube, Upper, Front Rigging, Color	1	1
13			Housing, Legrest, 80 Degree	1	1
14	Screw, Stop (M10 x 1.5 x 23mm)	1	1		
15	Screw, Stop (M10 x 1.5 x 12mm)	1	1		
16	Key, Cap	1	1		
17	C	8881172207	Hanger, Front Rigging, 90 Degree, Color	1	1
5			Bracket, Front Rigging	1	1
8			Screw, Socket Head, Cap (M6 x 35mm)	1	1
10			NLA - Locknut (M6 x 1)	1	1
18			Tube, 90 Degree Footrest, Color	1	1

NOTE: A - Includes items 2 - 10.  
 B - Includes items 4, 5, 8, 9, 10, and 12 - 16.  
 C - Includes items 5, 8, 10, and 18.